



COVIDIEN

positive results for life

Microstream™ Capnography

Competency Statement: *Demonstrate proper awareness, choice, and applications of Microstream capnography monitors and sampling lines. This is a suggested checklist for identifying areas of training as to the physical use and operation of the device. The checklist does not address issues of medical practice or clinical judgment related to use of this product.*

User is able to demonstrate:	Completed (Y / N)
1. Connection of power cord and pressing power - On/Off switch	
2. Purpose and operation of each front panel control (e.g., control buttons/knob, alarm silence, pump off, etc.)	
3. Understanding of monitor display (e.g., etCO ₂ , RR, waveform, etc.)	
4. Navigation of the monitor's menu screens to perform operational functions including:	
- Home Screen / etCO ₂ / SpO ₂ (as applicable) Setup	
- System Screen (e.g., clear trend, D/A Setup, Cal Check, etc.) including use of password to enter	
- Service Screen (e.g., institutional defaults, software support, CO ₂ calibration, etc.)	
- Alarm Limits Screen (e.g., set alarm limits, set caution enable/disable, default reset, trend screen, etc.)	
- Print functions and printer paper replacement (as applicable)	
- Adjusting alarm and pulse volumes	
5. Proper use of sampling lines and sensors (as applicable) including:	
- Attachment to monitor and application to patient	
- Understanding of single patient use sampling lines and sensors (as applicable)	
- Importance of not attempting to clear a blocked sampling line	

Name: _____

Title: _____

Unit: _____

Date: _____