

Dialysis Catheter Cleaning Agents

Please Note: Mixing of these solutions has not been tested and is not recommended.

Chronic Catheters

Cleaning Agent	Tandem Cath™ 10Fr Dual Lumen Catheter System	MAHURKAR™™ 13.5 Fr Chronic Catheter and PermCath Chronic Catheter	MAHURKAR™™ 14.5 Fr Chronic Catheter	Palindrome™™ 14.5 Fr Chronic Catheter Pre-curved Chronic Catheter H- Heparin Coated Catheter	Palindrome™™ SI- Sliver Ion Coated Catheter HSI- Heparin & Silver Ion Coated Catheter	Palindrome™™ RT- Reverse Tunnel Catheter
Alcavis™™ 50 (fka Amukin 50%) Disinfectant	Yes	Yes	Yes	Yes	Yes	Yes
Aqueous-Based Povidone Iodine	Yes	Yes	Yes	Yes	Yes	Yes
Bacitracin Ointment	No	Yes	Yes	Yes	Yes	No
Bactroban™™ Crème	No	Yes	Yes	Yes	Yes	No
Chlorhexidine (Hibiclens™™) Antiseptic	Yes	Yes	Yes	Yes	Yes	Yes
Chloraprep™™ Antimicrobial Solution	No	Yes	Yes	Yes	Yes	Yes
Dilutis™™ Solution	No	Yes	Yes	Yes	Yes	Yes
ExSept™™ (Sodium HypoChlorite) Antiseptic	Yes	Yes	Yes	Yes	Yes	Yes
Hydrogen Peroxide	Yes	Yes	Yes	Yes	Yes	Yes
Isopropyl Alcohol 70%	No	Yes	Yes	Yes	Yes	Yes
Neosporin™™ Antibiotic	Yes	Yes	Yes	Yes	Yes	No
Shur-Cleans™™ Cleansing Solution	No	Yes	Yes	Yes	Yes	Yes

Acute Catheters

Cleaning Agent	MAHURKAR™		
	MAHURKAR™ 8Fr Dual Lumen Catheter	MAHURKAR™ 10Fr Dual Lumen Catheter 11.5Fr Dual Lumen Catheter 13.5Fr Dual Lumen Catheter	MAHURKAR™ 12Fr TL Catheter 12Fr HPTL Catheter
Alcavis™ 50 (fka Amukin 50%) Disinfectant	Yes	Yes	Yes
Aqueous-Based Povidone Iodine	Yes	Yes	Yes
Bacitracin Ointment	Yes	Yes	Yes
Bactroban™ Crème	Yes	Yes	Yes
Chlorhexidine (Hibiclens™) Antiseptic	Yes	Yes	Yes
Chloraprep™ Antimicrobial Solution	Yes	Yes	Yes
Dilutis™ Solution	No	Yes	No
ExSept™ (Sodium HypoChlorite) Antiseptic	Yes	Yes	Yes
Hydrogen Peroxide	Yes	Yes	Yes
Isopropyl Alcohol 70%	No	No (10Fr, 11.5Fr) Yes (13.5Fr)	No
K-Y™ Jelly	Yes	Yes	Yes
Neosporin™ Antibiotic	Yes	Yes	Yes

Peritoneal Catheters

Cleaning Agent	Argyle™ Peritoneal Dialysis Catheters (ALL)
	Alcavis™ 50 (fka Amukin 50%) Disinfectant
Aqueous-Based Povidone Iodine	Yes
Bacitracin Ointment	Yes
Bactroban™ Crème	Yes
Chlorhexidine (Hibiclens™) Antiseptic	Yes
Chloraprep™ Antimicrobial Solution	Yes
Dilutis™ Solution	Yes
ExSept™ (Sodium HypoChlorite) Antiseptic	Yes
Hydrogen Peroxide	Yes
Isopropyl Alcohol 70%	Yes
K-Y™ Jelly	Yes
Neosporin™ Antibiotic	Yes



™MAHURKAR is a U.S. registered trademarks of Sakharam D. Mahurkar, used under license.

™ Trademark of its respective owners.

Other brands are trademarks of a Covidien Company.

COVIDIEN, COVIDIEN with Logo, Covidien logo, and *positive results for life* are U.S. and/or internationally registered trademarks of Covidien AG. ©2013 Covidien.

12MAN0152 04/13