


Hook & Loop 3D Adult Brief Product Application

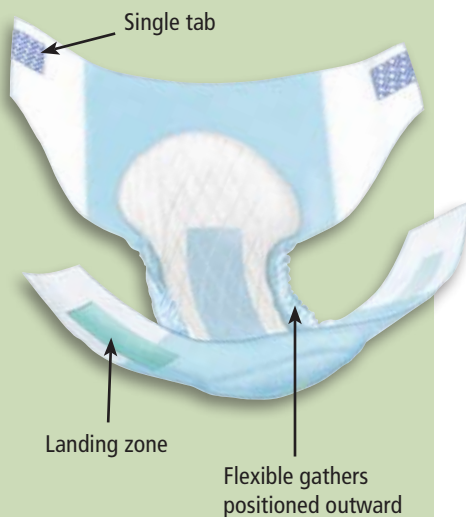

- Make sure you have the correct size. Using too large a brief will result in leakage, discomfort, skin breakdown, and added cost.
- When using the Hook & Loop 3D adult brief, the precision-placed landing zone ensures proper fit. Fasten the single-tab closure to the landing zone on the front of the brief. If the tab goes beyond the landing zone, the brief is too large. If the tab does not reach the landing zone, the brief is too small. The Hook & Loop 3D adult brief is the correct size if the tab is pulled **SNUGLY** around the waist and lands on the landing zone.
- Make sure the elastic gathers fit snugly in the creases of the legs. This reduces the likelihood of leakage.
- Make sure the flexible gathers are positioned outward. If turned in, leakage and skin irritation can result.
- Place brief beneath resident. Create a log roll by folding the front wings of the brief back with the soft, inner area facing the resident. Bring roll up between legs and over abdomen. Unroll over the abdomen, smooth side panels. Make sure the brief is fitted snugly.
- When using the Hook & Loop 3D adult brief, gently pull the brief upwards until the crotch area fits snugly to the groin and inner leg. Re-adjust the tabs as required by simply lifting up on the tab and repositioning it onto the landing zone. A correctly applied brief fits securely around the legs and waist and provides comfortable protection for the resident.
- Use only one absorbent product at a time.

Hook & Loop 3D Adult Brief Troubleshooting Hints

PROBLEM	CAUSE	CORRECTION
Redness in groin area	Poor fit with gathers not turned out away from the skin Brief too large	Good fit with brief pulled up snugly in groin area and leg gathers turned out and away from skin
Skin irritation/redness in brief area	Infrequent changes; poor peri care Concentrated urine	Change, cleanse, pat skin dry when soiled Use Barrier Cream per policy Increase fluid intake
Redness inner thighs	Poor cleansing techniques Baggy brief – sized too large; hanging between legs	Keep skin clean & dry Use correct size
Wet linen/clothing	Resident super saturated Brief not fitted properly Brief too large	More frequent changes Snug fit & proper size
Picking & tearing of brief	Brief may be fitted improperly Confused resident	Apply brief as instructed Cover brief with resident's underwear
Tearing of the brief	Using brief to lift or re-position resident	Use lift sheet