


**Medtronic**

# Aurora EV-ICD™ system

The extravascular system with  
transvenous ICD benefits¹

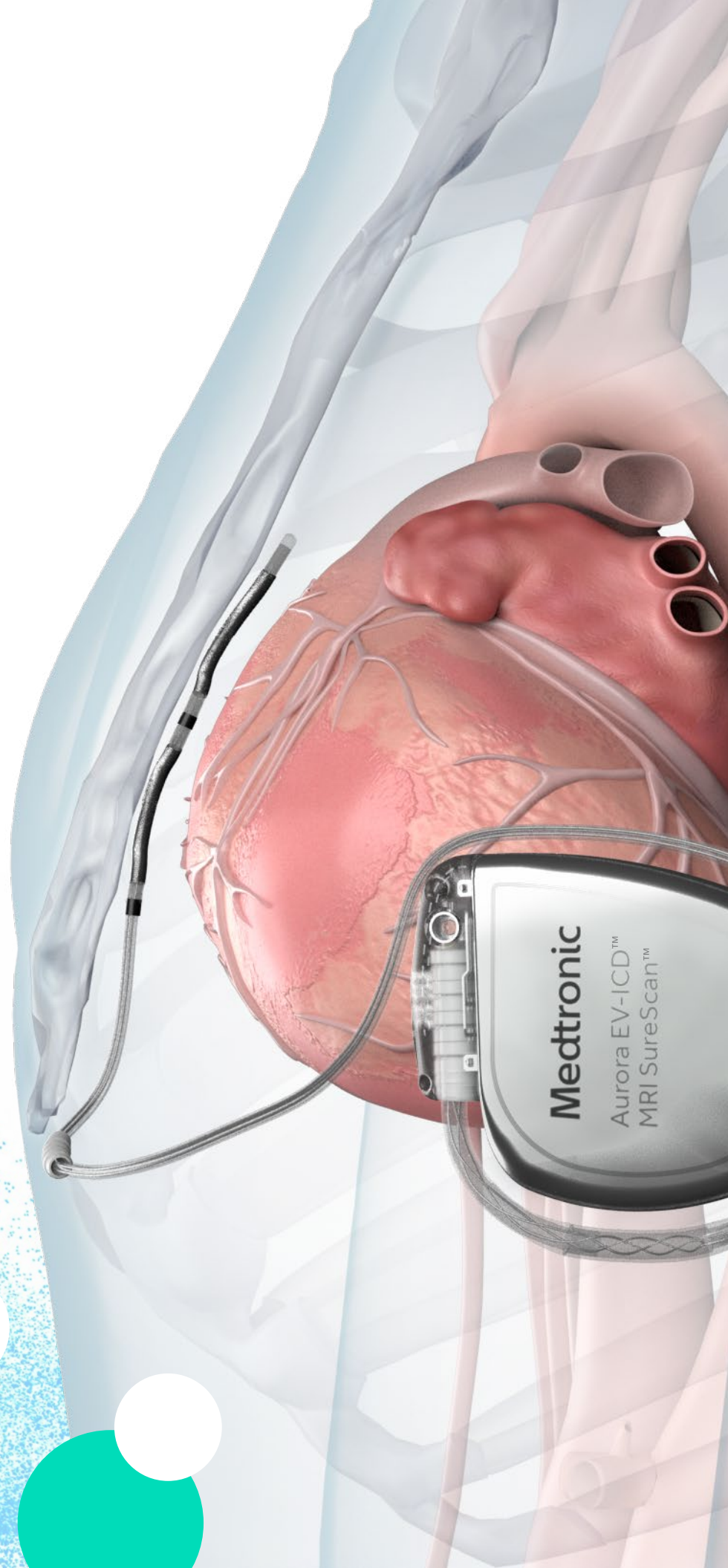

Establishing the rhythm.  
Pioneering what's possible.

# Why Aurora EV-ICD?

## Aurora EV-ICD offers the advantages of an extravascular system

- Avoids certain complications associated with transvenous leads<sup>†</sup>
- Preserves the vasculature and reduces potential for vascular injury



## Aurora EV-ICD provides features available in transvenous ICD systems



Antitachycardia pacing (ATP)



Small size (33 cm<sup>3</sup>) and PhysioCurve™ design – same size and shape as Medtronic transvenous single-chamber ICDs



11.7-year projected longevity<sup>2</sup>



Pause Prevention pacing (backup bradycardia pacing)



Programmable VF detection intervals and multiple therapy zones



Monitor zone which allows for documentation of slow VTs, including non-sustained VTs

<sup>†</sup>The Aurora EV-ICD lead is not intended for implantation within the heart or vasculature, and, thus, Aurora is expected to avoid vascular complications associated with transvenous leads. There were no major intraprocedural complications observed in the EV ICD Pivotal clinical study.<sup>1</sup>

# What makes the Aurora EV-ICD advantages possible?

Because the lead is placed close to the heart, the energy required for pacing and defibrillation is lower than if it were further away and separated from the heart by bone.<sup>1,3</sup>

As a result, the EV-ICD system is able to incorporate important features: ATP, Pause Prevention pacing (backup bradycardia pacing), and 40 Joule defibrillation, all in a device the same size as transvenous ICDs and with similar projected longevity.

## System safety

Specialized implant tools and techniques were developed for this procedure to help ensure the safety of the procedure and the performance of the system.<sup>1,3-6</sup>

The system has been evaluated in five clinical studies, involving more than 80 sites and more than 400 patient subjects.

The global, prospective EV ICD Pivotal clinical study exceeded its primary safety objective<sup>1</sup> and found the rate of EV ICD freedom from major system- or procedure-related complications at six months to be in line with the rates observed in transvenous ICD studies<sup>7-12</sup> and the S-ICD IDE study.<sup>13</sup> The study's investigators observed 25 major complications in 23 patients at 6 months. Procedure-related complications included revision for lead dislodgement and treatment of postoperative wound infection.<sup>1</sup> No system changes were required for seven events.


Scan the QR code to read  
the published manuscript



## BENEFITS

# Projected longevity

Projected longevity similar to other Medtronic single-chamber ICDs

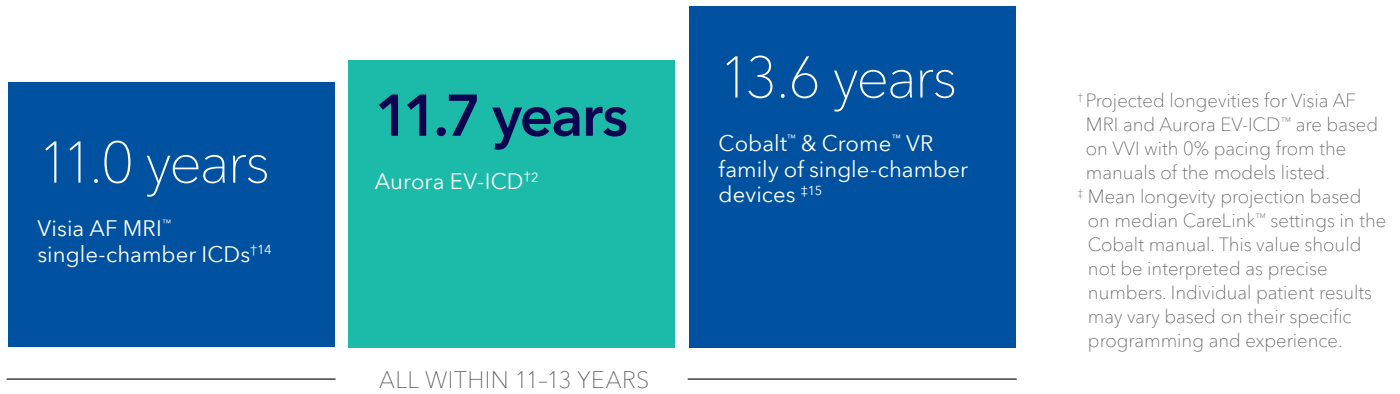

## Option for 40 J energy delivery on all shocks

(including the first shock)<sup>2</sup>




## Effective defibrillation

at implantation in the EV ICD Pivotal clinical study<sup>1</sup> was greater than observed in historical transvenous ICD studies<sup>16-19</sup> and similar to S-ICD.<sup>13</sup>

## Designed for patient comfort

With its PhysioCurve design, Aurora EV-ICD showed a 27% average reduction in tissue pressure relative to the subcutaneous ICD across the three implant locations modeled.<sup>20</sup>

- Tapered at the head and bottom of device to reduce tissue pressure and promote patient comfort
- Smaller footprint for a smaller incision

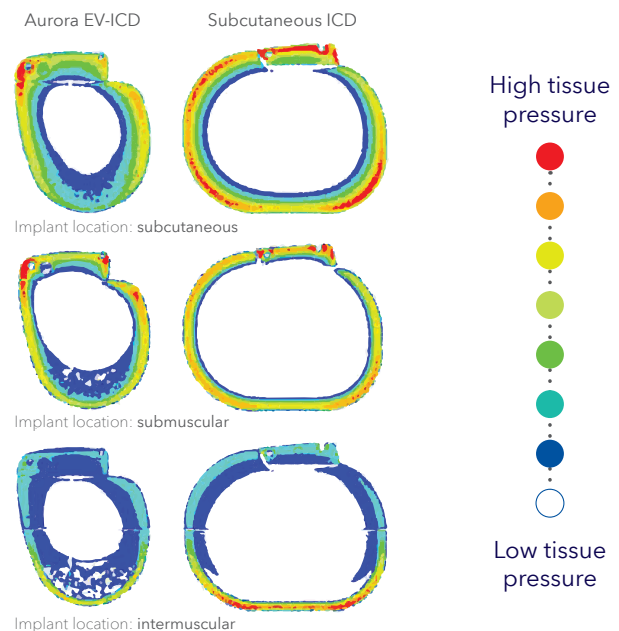

## PACING CAPABILITIES

### ATP


The only extravascular ICD to offer ATP

As with Medtronic transvenous ICDs, ATP therapy in Aurora EV-ICD includes Burst and Ramp pacing pulses, each with a programmable number of sequences.

In the EV ICD Pivotal clinical study, ATP successfully terminated 70% of episodes (32 of 46).<sup>1</sup> This is in the range of the ATP efficacy reported in transvenous ICD publications, 52% to 87%.<sup>21-24</sup>

Aurora EV-ICD has three pacing vector options for ATP

- 1 Coil 2 to coil 1
- 2 Ring 1 to coil 2
- 3 Ring 1 to ring 2

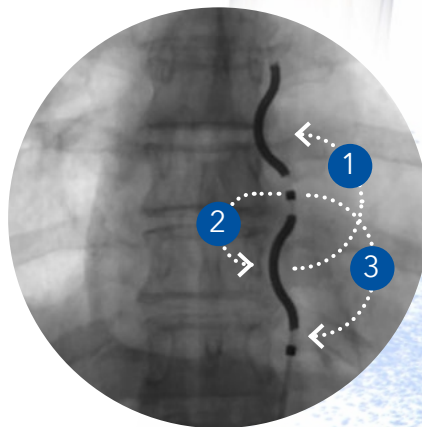



### Pause Prevention pacing

Pause Prevention is a pacing feature that monitors the heart for significant pauses and responds by providing temporary bradycardia pacing support.

The device records data about episodes that meet the programmed pause detection criteria when Pause Prevention is programmed to On or Monitor. This data is useful for analyzing Pause Prevention episodes and the events leading up to them.

### Post-shock pacing

Aurora EV-ICD can be programmed to deliver temporary post-shock pacing following a defibrillation or cardioversion therapy, as there may be a temporary bradycardia or asystole after the heart receives a high-voltage therapy.

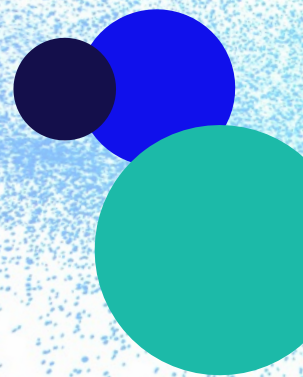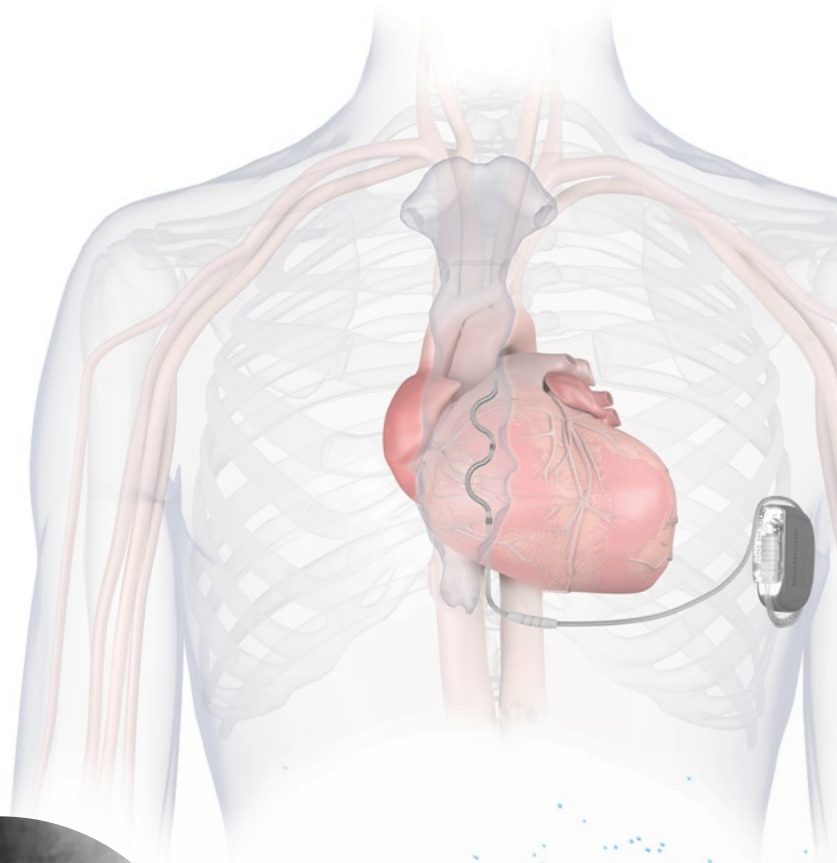

## Epsilon EV™ defibrillation lead key features

- Epsilon-shaped distal section is intended to optimize the electrodes' locations relative to the heart and the device

- Defibrillation coils positioned toward the patient's right side for a wider defibrillation vector between the coils and the device


- Pacing/sensing ring electrodes positioned toward the patient's left side so they are closer to the heart

- Curvature intended to help stabilize the lead in the mediastinal tissue

- Four electrodes, consisting of two coils and two rings, to support three different pacing vector options and three sensing vector options

- Isodiametric 8.7 Fr lead body and four conductor cables extending to the distal tip of the lead to provide high tensile strength for extractability

- Chronic lead extraction was prospectively studied in sheep. Chronic removal of 19 leads from the substernal space was successfully performed between one and three years post-implant using traction and simple tools.<sup>25</sup>

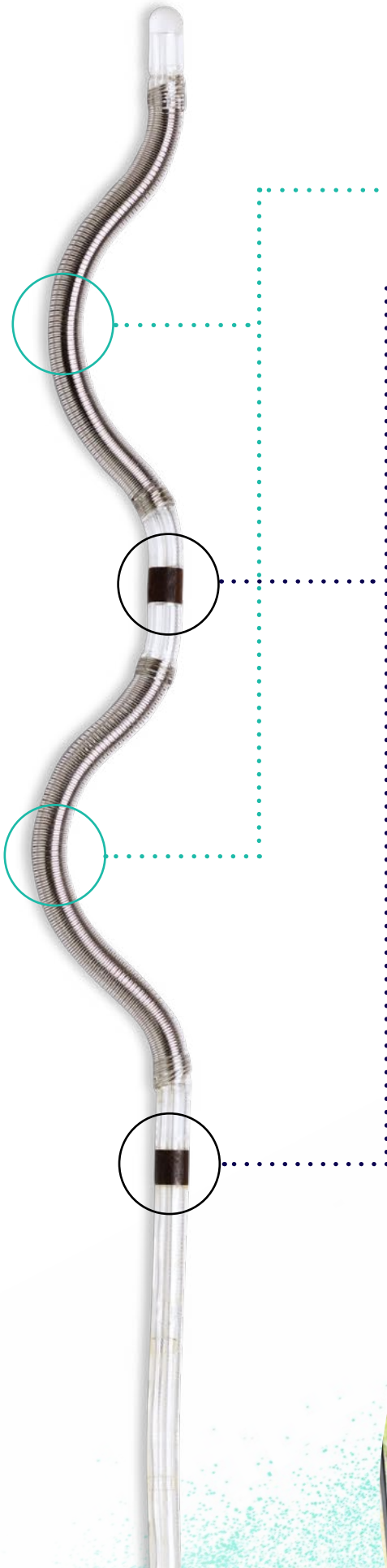

## Diagnostics and programmability

- Programmable VF detection intervals from 12/16 to 120/160 allow for tailoring the timing of therapy delivery
- Multiple therapy zones provide more programming flexibility
- Monitor zone allows for documentation of slow VTs, including non-sustained VTs
- EGM storage of VT/VF, VT monitor and SVT episodes, including slow VT episodes, provides additional clinical information

## MRI access at 1.5T and 3T

### How MRI SureScan™ technology works

- SureScan devices are specifically engineered for the MRI environment, with enhancements for patient safety during an MRI scan
- 1.5T and 3T MRI access when MR conditions for use are met
- MRI scanning conditions are straightforward: no anatomical exclusion zone, no patient height restriction, 1.5 and 3T compatibility<sup>2</sup>



### References

- <sup>1</sup> Friedman P, Murgatroyd F, Boersma LVA, et al. Efficacy and Safety of an Extravascular Implantable Cardioverter-Defibrillator. *N Engl J Med*. October 6, 2022;387(14):1292-1302.
- <sup>2</sup> Medtronic Aurora EV-ICD™ MRI SureScan™ DVEA3E4 device manual.
- <sup>3</sup> Crozier I, Haqqani H, Kotschet E, et al. First-in-Human Chronic Implant Experience of the Substernal Extravascular Implantable Cardioverter-Defibrillator. *JACC Clin Electrophysiol*. November 2020;6(12):1525-1536.
- <sup>4</sup> Chan JYS, Lelakowski J, Murgatroyd FD, et al. Novel Extravascular Defibrillation Configuration With a Coil in the Substernal Space: The ASD Clinical Study. *JACC Clin Electrophysiol*. August 2017;3(8):905-910.
- <sup>5</sup> Sholevar DP, Tung S, Kuriachan V, et al. Feasibility of extravascular pacing with a novel substernal electrode configuration: The Substernal Pacing Acute Clinical Evaluation Study. *Heart Rhythm*. April 2018;15(4):536-542.
- <sup>6</sup> Boersma LVA, Merkely B, Neuzil P, et al. Therapy From a Novel Substernal Lead: The ASD2 Study. *JACC Clin Electrophysiol*. February 2019;5(2):186-196.
- <sup>7</sup> Bardy GH, Lee KL, Mark DB, et al. Amiodarone or an Implantable Cardioverter-Defibrillator for Congestive Heart Failure. *N Engl J Med*. January 20, 2005;352(3):225-237.
- <sup>8</sup> Sweeney MO, Ellenbogen KA, Tang ASL, et al. Atrial pacing or ventricular backup-only pacing in implantable cardioverter-defibrillator patients. *Heart Rhythm*. November 2010;7(11):1552-1560.
- <sup>9</sup> Curtis AB, Worley SJ, Adamson PB, et al. Biventricular pacing for Atrioventricular Block and Systolic Dysfunction. *N Engl J Med*. April 25, 2013;368(17):1585-1593.
- <sup>10</sup> Linde C, Abraham WT, Gold MR, Sutton MSJ, Ghio S, Daubert C. Randomized Trial of Cardiac Resynchronization in Mildly Symptomatic Heart Failure Patients and in Asymptomatic Patients With Left Ventricular Dysfunction and Previous Heart Failure Symptoms. *J Am Coll Cardiol*. December 2, 2008;52(23):1834-1843.
- <sup>11</sup> Gold MR, Sommer T, Schwitzer J, et al. Full-Body MRI in Patients With an Implantable Cardioverter-Defibrillator: Primary Results of a Randomized Study. *J Am Coll Cardiol*. June 23, 2015;65(24):2581-2588.
- <sup>12</sup> Auricchio A, Schloss EJ, Kurita T, et al. Low inappropriate shock rates in patients with single- and dual/triple-chamber implantable cardioverter-defibrillators using a novel suite of detection algorithms: PainFree SST trial primary results. *Heart Rhythm*. May 2015;12(5):926-936.
- <sup>13</sup> Weiss R, Knight BP, Gold MR, et al. Safety and Efficacy of a Totally Subcutaneous Implantable-Cardioverter Defibrillator. *Circulation*. August 27, 2013;128(9):944-953.
- <sup>14</sup> Medtronic Visia AF MRI™ VR SureScan™ DVFB2D4 Device Manual M980310A001 A. Available at [https://manuals.medtronic.com/content/dam/emanuals/crdm/M980310A001A\\_view.pdf](https://manuals.medtronic.com/content/dam/emanuals/crdm/M980310A001A_view.pdf). Accessed March 21, 2023.
- <sup>15</sup> Medtronic Cobalt XT VR ICD MRI SureScan Model DVPA2D4 device manual. Available at: [https://manuals.medtronic.com/content/dam/emanuals/crdm/M975366A001C\\_view.pdf](https://manuals.medtronic.com/content/dam/emanuals/crdm/M975366A001C_view.pdf). Accessed March 21, 2023.
- <sup>16</sup> Michowitz Y, Lellouche N, Contractor T, et al. Defibrillation threshold testing fails to show clinical benefit during long-term follow-up of patients undergoing cardiac resynchronization therapy defibrillator implantation. *Europace*. May 2011;13(5):683-688.
- <sup>17</sup> Leong-Sit P, Gula LJ, Diamantouros P, et al. Effect of defibrillation testing on management during implantable cardioverter-defibrillator implantation. *Am Heart J*. December 2006;152(6):1104-1108.
- <sup>18</sup> Pires LA, Johnson KM. Intraoperative testing of the implantable cardioverter-defibrillator: how much is enough? *J Cardiovasc Electrophysiol*. February 2006;17(2):140-145.
- <sup>19</sup> Healey JS, Hohnloser SH, Glikson M, et al. Cardioverter defibrillator implantation without induction of ventricular fibrillation: a single-blind, non-inferiority, randomized controlled trial. *Lancet*. February 28, 2015;385(9970):785-791.
- <sup>20</sup> Mathiesen D, Bhatia V, Formosa E, et al. Prediction of Patient Comfort for Extravascular and Subcutaneous Implantable Cardioverter-Defibrillation Devices Using Modeling. AHA Abstract 13385. *Circulation*. November 8, 2021;144:A13385.
- <sup>21</sup> Arenal A, Proclemer A, Kloppe A, et al. Different impact of long-detection interval and anti-tachycardia pacing in reducing unnecessary shocks: data from the ADVANCE III trial. *Europace*. November 2016;18(11):1719-1725.
- <sup>22</sup> Schuger C, Daubert JP, Zareba W, et al. Reassessing the role of antitachycardia pacing in fast ventricular arrhythmias in primary prevention implantable cardioverter-defibrillator recipients: Results from MADIT-RIT. *Heart Rhythm*. March 2021;18(3):399-403.
- <sup>23</sup> Lee S, Stern R, Wathen M, et al. Anti-Tachycardia Pacing Therapy Effectively Terminates Fast Ventricular Tachycardia after Longer Detection Duration in Primary Prevention Patients: Results from the PREPARE Trial. *Heart Rhythm*. 2008;5(5):S334-S356.
- <sup>24</sup> Sterns LD, Auricchio A, Schloss EJ, et al. Anti-tachycardia pacing success in implantable cardioverter defibrillators by patient, device, and programming characteristics. *Heart Rhythm*. February 2023;20(2):190-197.
- <sup>25</sup> Thompson AE, Marshall M, Lentz L, Mazzetti H. Three-Year Extraction Experience of a Novel Substernal Extravascular Defibrillation Lead in Sheep. *Pacing Clin Electrophysiol*. March 2022;45(3):314-322.

## Brief Statement

### Aurora EV-ICD™ MRI SureScan™ System and Associated Tunneling Tools

#### Indications

**Device:** The Aurora EV-ICD™ MRI SureScan™ Model DVEA3E4 device is indicated for the automated treatment of patients who have experienced, or are at significant risk of developing, life-threatening ventricular tachyarrhythmias through the delivery of antitachycardia pacing, cardioversion, and defibrillation therapies. Medical conditions that may indicate a patient for an EV-ICD for primary or secondary prevention of sudden cardiac death due to life-threatening ventricular tachyarrhythmias include: previous ventricular tachyarrhythmias, coronary disease with left ventricular dysfunction, cardiomyopathy, inherited primary arrhythmia syndromes, and congenital heart disease.

Note: For patient-specific recommendations regarding indications for primary and secondary prevention of sudden cardiac death, refer to current clinical guidelines from the European Society of Cardiology (ESC), American Heart Association (AHA), American College of Cardiology (ACC), and Heart Rhythm Society (HRS).

**Lead:** The Epsila EV™ MRI SureScan™ Model EV2401 extravascular lead is indicated for use in the anterior mediastinum for pacing therapies, cardioversion, and defibrillation when an extravascular implantable cardioverter defibrillator is indicated to treat patients who have experienced, or are at significant risk of developing, life-threatening ventricular tachyarrhythmias.

**Tunneling Tools:** The Epsila EV™ Model EAZ101 sternal tunneling tool is indicated for use in the implant of a compatible anterior mediastinum defibrillation lead.

The Epsila EV™ Model EAZ201 transverse tunneling tool is indicated for use in the implant of a compatible anterior mediastinum defibrillation lead.

#### MR Conditions for Use

The Aurora EV-ICD MRI SureScan system is MR Conditional and, as such, is designed to allow patients to be safely scanned by an MRI machine when used according to the specified MR conditions for use. A complete SureScan system is required for use in the MR environment. Before performing an MR scan, refer to the MRI technical manual for MRI-specific warnings and precautions. When programmed to On, the MRI SureScan feature allows the patient to be safely scanned. A complete SureScan system includes a SureScan extravascular ICD device (Model DVEA3E4) with a SureScan extravascular lead (Model EV2401). To verify that components are part of a SureScan system, visit <http://www.mrisurescan.com>. Any other combination may result in a hazard to the patient during an MRI scan.

#### Contraindications

The Aurora EV-ICD MRI SureScan Model DVEA3E4 device is contraindicated for use in the following situations:

- If implanted with a unipolar pacemaker, a device delivering dual-chamber or triple-chamber pacing, and/or a device delivering antitachycardia therapies
- If incessant ventricular tachycardia (VT) or ventricular fibrillation (VF) exists
- If the patient's primary disorder is chronic atrial tachycardia with no concomitant VT or VF
- If symptomatic bradycardia exists
- If tachyarrhythmias with transient or reversible causes exist

The Epsila EV MRI SureScan Model EV2401 lead is contraindicated for any application that is not specified in the Indications.

The Epsila EV Model EAZ101 sternal tunneling tool is contraindicated for use in patients with a prior sternotomy.

The Epsila EV Model EAZ201 transverse tunneling tool is contraindicated for any application that is not specified in the Indications.

#### Warnings and Precautions

**Device and Lead:** It is important to read the Aurora EV-ICD MRI Technical Manual before conducting an MRI scan on a patient with an implanted SureScan system. The MRI SureScan feature permits a mode of operation that allows a patient with a SureScan system to be safely scanned by an MRI machine. When programmed to On, MRI SureScan operation disables arrhythmia detection and all user-defined diagnostics. Do not scan a patient without first programming the MRI SureScan mode to On. Scanning the patient without programming the MRI SureScan mode On may result in patient harm or damage to the SureScan system.

Patients and their implanted systems must be screened to meet the following requirements for MRI: no implanted lead extenders, lead adaptors or

abandoned leads present; no broken leads or leads with intermittent electrical contact as confirmed by lead impedance history; and the Sure Scan device must be operating within the projected service life; the device does not provide pacing therapy when SureScan mode is programmed On. Do not scan pacemaker-dependent patients. MRI scans during the lead maturation period have not been prospectively studied by Medtronic and are not recommended. If scanning a patient with multiple devices, ensure all devices meet the MRI labeling conditions.

Use only the Epsila EV MRI SureScan Model EV2401 extravascular lead with a Medtronic EV4 implantable cardioverter defibrillator system. The known potential adverse consequences of using any other combination may include undersensing of cardiac activity, failure to deliver necessary therapy, or an intermittent electrical connection. All can present serious risks for adverse events to the patient. The EV4 connector is a Medtronic proprietary design, not an industry standard. No claims of safety and efficacy can be made with regard to devices that are not labeled as EV4 by Medtronic.

Pre-implant consideration for concomitant implant with a neurostimulator and cardiac device implants: Some patients have medical conditions that require the implant of both a neurostimulator and a cardiac device (for example, a pacemaker, a defibrillator, or a monitor). In this case, physicians involved with either device should contact Medtronic Technical Services or their Medtronic representative before implanting the patient with the second device. Based on the particular devices that the physicians have prescribed, Medtronic can provide the necessary precautions and warnings related to the implant procedure.

Use of the DVEA3E4 device has not been evaluated in patients who have undergone a prior sternotomy.

The DVEA3E4 device has not been tested specifically for pediatric use.

Use of the EV2401 lead has not been evaluated in patients who have undergone a prior sternotomy. Performing a sternotomy on a patient with an implanted lead has not been evaluated.

Do not implant the EV2401 lead using any tools other than the Medtronic tunneling tools designed for implanting the extravascular ICD system.

**Tunneling Tools:** The tunneling tools have not been tested for use with non-Medtronic products or for pediatric use.

Use of the EAZ201 transverse tunneling tool have not been evaluated in patients who have undergone a prior sternotomy.

#### Potential Adverse Events

Implant and usage of this system may result in adverse events, which may lead to injury, death, or other serious adverse reactions. Potential adverse events include, but are not limited to acute tissue trauma, allergic reaction, bradyarrhythmia, cardiac arrest, cardiac inflammation, cardiac perforation, cardiac tamponade, death, device migration, discomfort, dizziness, dyspnea, erosion, extracardiac stimulation, fever, hematoma, hemorrhage, hemothorax, hiccups, hospitalization, inappropriate shock, infection, insulation failure, lead abrasion, lead fracture, lead migration or dislodgement, lethargy, mental anguish, organ damage (liver, mammary arteries, diaphragmatic arteries), pain, palpitations, pericardial effusion, pericarditis, pneumothorax, return of cardiac symptoms, seroma, syncope, tachyarrhythmia, toxic reaction, and wound dehiscence.

Potential MRI adverse events include the following: lead electrode heating resulting in tissue damage near the lead electrodes or patient discomfort or both; spontaneous tachyarrhythmia occurring during the scan that is not detected and treated because tachyarrhythmia detection is suspended while MRI SureScan mode is programmed to On; device heating resulting in tissue damage in the implant pocket or patient discomfort or both; MR-induced muscle stimulation resulting in patient discomfort; damage to the device or lead causing the system to fail to detect or treat irregular heartbeats or causing the system to treat the patient's condition incorrectly; damage to the functionality or mechanical integrity of the device resulting in the inability of the device to communicate with the programmer; and movement or vibration of the device or leads resulting in dislodgment.

*See the Aurora EV-ICD MRI SureScan technical manual before performing an MRI Scan, and the device, lead and tunneling tools manuals for detailed information regarding the implant procedure, indications, contraindications, warnings, precautions, and potential complications/adverse events. Refer to the Medtronic Manual Library website [www.medtronic.com/manuals](http://www.medtronic.com/manuals). For further information, please call Medtronic at 1-800-328-2518 and/or consult Medtronic's website at [www.medtronic.com](http://www.medtronic.com) or [www.mrisurescan.com](http://www.mrisurescan.com).*

**Caution:** Federal law (USA) restricts these devices to sale by or on the order of a physician.

# Medtronic

710 Medtronic Parkway  
Minneapolis, MN 55432-5604  
USA

Toll-free in USA: 800.633.8766  
Worldwide: +1.763.514.4000

[medtronic.com](http://medtronic.com)

Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

UC202115487 EN ©2023 Medtronic. All Rights Reserved. 09/2023