


**Medtronic**

Telescope™  
Guide Extension Catheter

Extended reach.  
Smooth delivery.



Your complex  
clinical practice  
requires a guide  
extension catheter  
that can take you  
**further.**


## Superior deliverability<sup>1</sup>

Strong pushwire and coil-reinforced distal segment enhance performance.<sup>1</sup>

## Softest tip<sup>1</sup>

Our extruded TruFlex™ soft polymer tip responsively deflects and provides flexibility.<sup>1</sup>

## Soft tip compression

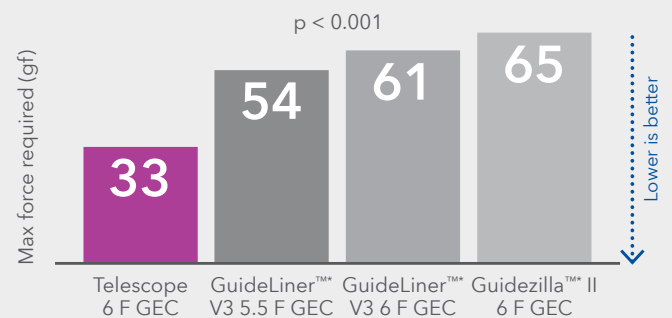

## SmoothPass technology

SmoothPass technology includes three components that work together to smoothly channel interventional devices.<sup>2</sup>

Entry port contributes to smooth device entry and delivery<sup>2</sup>



Adverse events including, but not limited to, dissection, air embolism, blood loss, and hypotension may occur when using this device.


™Third-party brands are trademarks of their respective owners.

<sup>1</sup>May not be indicative of clinical performance.


<sup>1</sup>Based on bench test data.† n = 8 of each GEC: Telescope 6 F, Guidezilla II 6 F GEC, GuideLiner V3 5.5 F/6 F GEC.

<sup>2</sup>Based on the Telescope GEC design freeze document (internal design document).


Deliverability

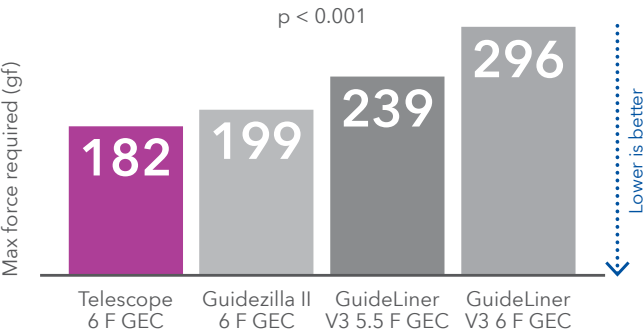

Pushability

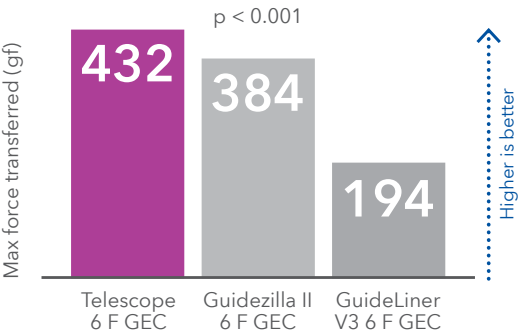


The TruFlex tip required at least **39% less force** to deflect.<sup>1</sup>

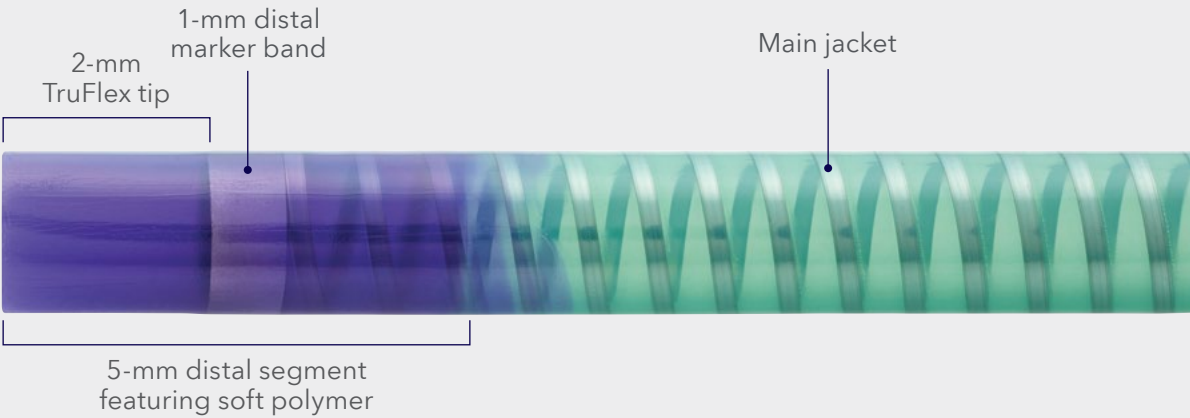

Short 4-cm polymer on-ramp maintains superior deliverability and channels interventional devices<sup>1</sup>

Tapered distal pushwire section transitions from solid, round proximal pushwire for flexibility<sup>2</sup>

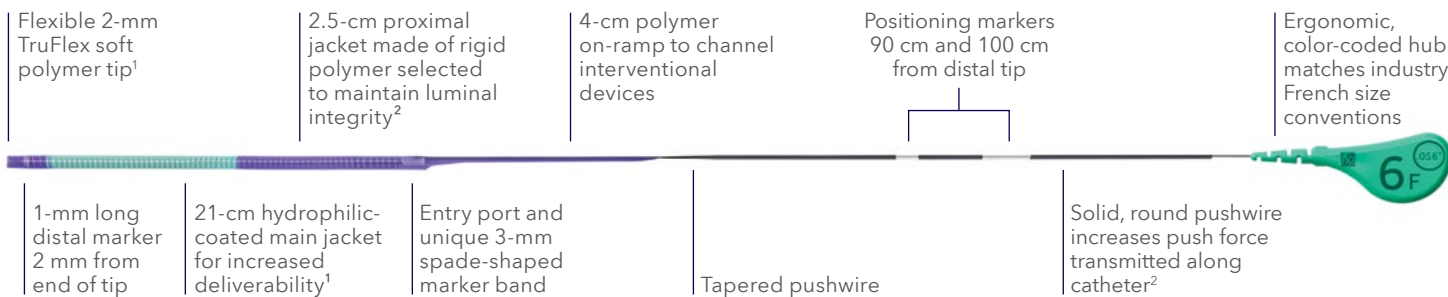

## Technical features

Technical features	Telescope GEC
Catheter length (cm)	150
Distal extension length (cm)	25
I.D. (in)	0.056 (6 F), 0.062 (7 F)
O.D. (in)	0.067 (6 F), 0.075 (7 F)
Required GC I.D. (in)	≥ 0.070 (6 F), ≥ 0.078 (7 F)
Marker band material	Platinum-iridium
Marker band lengths and locations	1 mm long, 2 mm from distal tip 3 mm long, spade-shaped at entry port
Coating	Hydrophilic, outer layer of distal 21 cm
Pushwire length (cm)	125
Tapered pushwire portion (cm)	10
On-ramp length (cm)	4
On-ramp material	Nylon-based polymer
TruFlex™ tip (mm)	2
Shelf life (yrs)	2

## Ordering information

French size (F)	CFN	French size (F)	CFN
6	TELE6F	7	TELE7F
	(240)TELE6F		(240)TELE7F

**CAUTION:** Federal (USA) law restricts these devices to sale by or on the order of a licensed healthcare practitioner. See package inserts for full product information.

# Medtronic

Medtronic  
Tel: 707.525.0111

Lifeline Customer Support  
Tel: 877.526.7890

Product Services  
Tel: 888.283.7868

Europe  
Medtronic Intl. Trading SARL  
Tel: 41 21 802 7000

Asia Pacific  
Medtronic Intl. Ltd.  
Tel: 65 6436 5000

Canada  
Medtronic of Canada Ltd.Tel:  
905 460 3800  
Toll-free: 800 268 5346

Latin America  
Medtronic USA, Inc.  
Tel: 786.709.4200

[medtronic.com](https://www.medtronic.com)

UC201909449a IE ©2023 Medtronic. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. For distribution only in markets where the Telescope guide extension catheter has been approved. Not for distribution in France.