

**Device Survival Summary** (95% Confidence Interval)  
 The following table shows CRT device survival estimates with 95% confidence intervals. Estimates are shown both with and without normal battery depletions included.

| Model Number | Family            | US Market Release | Registered US Implants | Estimated Active US Implants | Normal Battery Depletions (US) | Malfunctions (US)            |                                  | Device Survival Probability (%) |                   |                   |                               |                               |                               |       |  |  |  |  |
|--------------|-------------------|-------------------|------------------------|------------------------------|--------------------------------|------------------------------|----------------------------------|---------------------------------|-------------------|-------------------|-------------------------------|-------------------------------|-------------------------------|-------|--|--|--|--|
|              |                   |                   |                        |                              |                                | Therapy Function Compromised | Therapy Function Not Compromised | Years After Implant             |                   |                   |                               |                               |                               |       |  |  |  |  |
|              |                   |                   |                        |                              |                                | 1 yr                         | 2 yr                             | 3 yr                            | 4 yr              | 5 yr              | 6 yr                          | 7 yr                          | 8 yr                          | 10 yr |  |  |  |  |
| <b>7272</b>  | InSync ICD        | Jul-02            | 13,000                 | 1,000                        | 1,284                          | 15 + 223 = 238               | Total = 238                      | 99.9<br>+0.0/-0.1               | 99.8<br>+0.1/-0.1 | 99.1<br>+0.2/-0.2 | 97.8<br>+0.3/-0.4             | 96.8<br>+0.4/-0.5             | 95.5<br>+0.6/-0.7<br>at 71 mo |       |  |  |  |  |
| <b>7277</b>  | InSync Marquis    | Mar-03            | 7,000                  | 100                          | 588                            | 11 + 63 = 74                 | 74                               | 100.0<br>+0.0/-0.1              | 98.9<br>+0.3/-0.4 | 98.2<br>+0.4/-0.5 | 97.7<br>+0.5/-0.6             | 97.6<br>+0.5/-0.7             |                               |       |  |  |  |  |
| <b>7289</b>  | InSync II Marquis | Jul-03            | 28,000                 | 1,000                        | 5,001                          | 31 + 253 = 284               | 284                              | 99.9<br>+0.0/-0.0               | 99.7<br>+0.1/-0.1 | 98.7<br>+0.2/-0.2 | 98.0<br>+0.2/-0.3             | 97.9<br>+0.2/-0.3<br>at 49 mo |                               |       |  |  |  |  |
| <b>7297</b>  | InSync Sentry     | Nov-04            | 9,000                  | 4,000                        | 458                            | 1 + 29 = 30                  | 30                               | 100.0<br>+0.0/-0.1              | 99.9<br>+0.1/-0.1 | 99.6<br>+0.1/-0.2 | 99.5<br>+0.2/-0.2<br>at 47 mo |                               |                               |       |  |  |  |  |
| <b>7299</b>  | InSync Sentry     | Apr-05            | 31,000                 | 18,000                       | 708                            | 5 + 43 = 48                  | 48                               | 99.8<br>+0.1/-0.1               | 97.8<br>+0.3/-0.4 | 89.8<br>+0.7/-0.8 | 73.6<br>+3.1/-3.4<br>at 47 mo |                               |                               |       |  |  |  |  |
| <b>7303</b>  | InSync Maximo     | Jun-04            | 17,000                 | 6,000                        | 1,408                          | 5 + 57 = 62                  | 62                               | 100.0<br>+0.0/-0.0              | 99.8<br>+0.1/-0.1 | 99.6<br>+0.1/-0.1 | 99.4<br>+0.1/-0.2             | 99.4<br>+0.1/-0.2<br>at 52 mo |                               |       |  |  |  |  |
| <b>7304</b>  | InSync Maximo     | Apr-05            | 18,000                 | 11,000                       | 344                            | 2 + 22 = 24                  | 24                               | 100.0<br>+0.0/-0.0              | 99.9<br>+0.0/-0.1 | 99.7<br>+0.1/-0.1 | 99.7<br>+0.1/-0.1<br>at 42 mo |                               |                               |       |  |  |  |  |

continued

**Device Survival Summary** continued

| Model Number                    | Family          | US Market Release | Registered US Implants | Estimated Active US Implants | Normal Battery Depletions (US) | Malfunctions (US)            |                                  | Device Survival Probability (%) |                                |                    |                               |                    |                   |                   |                               |                               |                               |
|---------------------------------|-----------------|-------------------|------------------------|------------------------------|--------------------------------|------------------------------|----------------------------------|---------------------------------|--------------------------------|--------------------|-------------------------------|--------------------|-------------------|-------------------|-------------------------------|-------------------------------|-------------------------------|
|                                 |                 |                   |                        |                              |                                | Therapy Function Compromised | Therapy Function Not Compromised | Total                           | 1 yr                           | 2 yr               | 3 yr                          | 4 yr               | 5 yr              | 6 yr              | 7 yr                          | 8 yr                          | 10 yr                         |
| 8040                            | InSync          | Aug-01            | 15,000                 | 3,000                        | 533                            | 21                           | 7                                | 28                              | 100.0<br>+0.0/-0.0             | 100.0<br>+0.0/-0.1 | 99.9<br>+0.0/-0.1             | 99.8<br>+0.1/-0.1  | 99.7<br>+0.1/-0.2 | 99.6<br>+0.1/-0.2 | 99.6<br>+0.1/-0.2             | 99.6<br>+0.1/-0.2             | 99.6<br>+0.1/-0.2<br>at 86 mo |
|                                 |                 |                   |                        |                              |                                | 21                           | 7                                | 28                              | 99.9<br>+0.0/-0.1              | 99.7<br>+0.1/-0.1  | 98.3<br>+0.2/-0.3             | 96.2<br>+0.4/-0.4  | 91.3<br>+0.7/-0.7 | 84.6<br>+0.9/-1.0 | 73.5<br>+1.9/-2.0             | 69.5<br>+2.7/-2.9<br>at 86 mo |                               |
| 8042                            | InSync III      | Feb-03            | 29,000                 | 16,000                       | 243                            | 3                            | 3                                | 6                               | 100.0<br>+0.0/-0.0             | 100.0<br>+0.0/-0.0 | 100.0<br>+0.0/-0.0            | 100.0<br>+0.0/-0.1 | 99.9<br>+0.0/-0.1 | 99.9<br>+0.0/-0.1 | 99.9<br>+0.0/-0.1             | 99.9<br>+0.0/-0.1<br>at 73 mo |                               |
|                                 |                 |                   |                        |                              |                                | 3                            | 3                                | 6                               | 100.0<br>+0.0/-0.0             | 99.9<br>+0.0/-0.1  | 99.2<br>+0.1/-0.2             | 97.2<br>+0.3/-0.4  | 93.9<br>+0.6/-0.7 | 82.1<br>+4.1/-5.1 | 79.0<br>+4.8/-5.9<br>at 73 mo |                               |                               |
| C154DWK,<br>C164AWK,<br>C174AWK | Concerto        | May-06            | 68,000                 | 57,000                       | 168                            | 14                           | 36                               | 50                              | 99.9<br>+0.0/-0.0              | 99.8<br>+0.1/-0.1  | 99.4<br>+0.3/-0.5<br>at 29 mo |                    |                   |                   |                               |                               |                               |
|                                 |                 |                   |                        |                              |                                | 14                           | 36                               | 50                              | 99.8<br>+0.0/-0.1              | 97.5<br>+0.3/-0.3  | 93.8<br>+1.3/-1.7<br>at 29 mo |                    |                   |                   |                               |                               |                               |
| D224TRK                         | Consulta CRT-D  | Aug-08            | 2,000                  | 2,000                        | 0                              | 0                            | 0                                | 0                               | 100.0<br>+0.0/-0.0<br>at 3 mo  |                    |                               |                    |                   |                   |                               |                               |                               |
|                                 |                 |                   |                        |                              |                                | 0                            | 0                                | 0                               | 100.00<br>+0.0/-0.0<br>at 3 mo |                    |                               |                    |                   |                   |                               |                               |                               |
| D284TRK                         | Maximo II CRT-D | Mar-08            | 1,000                  | 1,000                        | 0                              | 0                            | 0                                | 0                               | 100.0<br>+0.0/-0.0<br>at 2 mo  |                    |                               |                    |                   |                   |                               |                               |                               |
|                                 |                 |                   |                        |                              |                                | 0                            | 0                                | 0                               | 100.00<br>+0.0/-0.0<br>at 2 mo |                    |                               |                    |                   |                   |                               |                               |                               |