

Epi/Myocardial Pacing Leads continued

Lead Survival Summary (95% Confidence Interval)

Model Number	Family	US Market Release	Leads Enrolled	Leads Active in Study	Qualifying Complications	Cumulative Months of Follow-Up in Study	Device Survival Probability (%)																
							Years After Implant																
							1 yr	2 yr	3 yr	4 yr	5 yr	6 yr	7 yr	8 yr	10 yr	12 yr	14 yr	16 yr					
4951, 4951M	Spectraflex	Oct-81	179	4	10	6,512	977 +1.6/-4.8	96.5 +2.2/-5.8	96.5 +2.2/-5.8	93.4 +3.7/-8.1	93.4 +3.7/-8.1 at 57 mo												
4965	CapSure Epi	Sep-96	188	28	8	5,451	98.8 +0.9/-3.6	97.7 +1.6/-5.0	96.5 +2.2/-6.1	91.9 +4.3/-8.9 at 45 mo													
4968	CapSure Epi	Sep-99	543	326	34	22,604	99.6 +0.3/-1.4	97.4 +1.2/-2.3	95.9 +1.7/-2.7	94.5 +2.1/-3.4	93.2 +2.5/-4.0	91.7 +3.0/-4.7	91.7 +3.0/-4.7	91.7 +3.0/-4.7									
5071	(No brand name)	Dec-92	229	30	11	7,092	96.3 +2.1/-4.4	94.5 +2.8/-5.5	91.7 +4.0/-7.3	91.7 +4.0/-7.3	91.7 +4.0/-7.3	91.7 +4.0/-7.3 at 66 mo											
6917, 6917A	Tenax	Jun-73	985	4	69	47,481	99.0 +0.5/-1.0	97.6 +0.9/-1.5	95.9 +1.3/-2.0	93.6 +1.8/-2.5	92.8 +2.0/-2.6	91.1 +2.3/-3.1	89.5 +2.7/-3.4	87.2 +3.1/-4.1	84.2 +3.9/-4.8	83.2 +4.2/-5.3	80.4 +5.2/-6.8 at 150 mo						