

# Epi/Myocardial Pacing Leads continued

## Lead Survival Summary (95% Confidence Interval)

Model Number	Family	US Market Release	Leads Enrolled	Leads Active in Study	Qualifying Complications	Cumulative Months of Follow-Up in Study	Device Survival Probability (%)																	
							Years After Implant																	
							1 yr	2 yr	3 yr	4 yr	5 yr	6 yr	7 yr	8 yr	10 yr	12 yr	14 yr	16 yr						
4951, 4951M	Spectraflex	Oct-81	179	4	14	4,494	97.7	96.5																
							+1.6/-4.8	+2.2/-5.8																
4965	CapSure Epi	Sep-96	191	27	8	4,903	98.8	97.6	95.9	95.9	93.8	92.4	92.4											
							+0.9/-3.5	+1.7/-5.3	+2.7/-7.4	+2.7/-7.4 at 39 mo	+2.3/-3.6	+2.7/-4.2	+2.7/-4.2											
4968	CapSure Epi	Sep-99	583	359	22	25,381	99.6	97.5	96.2	94.9	93.8	92.4	92.4											
							+0.3/-1.1	+1.1/-2.1	+1.5/-2.5	+1.9/-3.0	+2.3/-3.6	+2.7/-4.2	+2.7/-4.2											
5071	(No brand name)	Dec-92	235	35	12	5,315	96.6	94.8																
							+1.9/-4.1	+2.6/-5.1 at 21 mo																