


THE CONFIDENCE TO KNOW BLEEDING WILL STOP.


Veriset™ Haemostatic Patch

LAPAROSCOPIC SURGERY

Preparing Veriset™ Haemostatic Patch

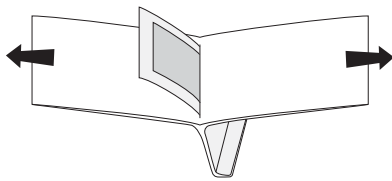


1. Non-sterile person opens the foil pouch.

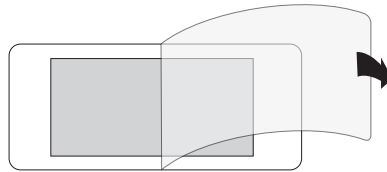


2. Sterile person opens the Tyvek™ pouch. Use the Veriset™ patch within 60 minutes after opening. Veriset™ patch should be placed on the bleeding site, extending ~1–2 cm beyond the site. Veriset™ patch may be cut as necessary.


Applying Veriset™ Haemostatic Patch

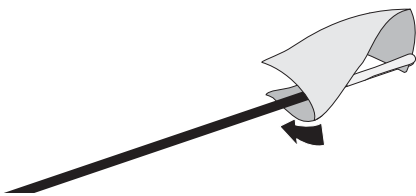


3. Grasp the Veriset™ patch on the edge with the grasper. Roll the patch around the grasper, with the darker side surface on the inside. For a rectangular patch, roll the patch along the long axis to facilitate entry into the trocar cannula.

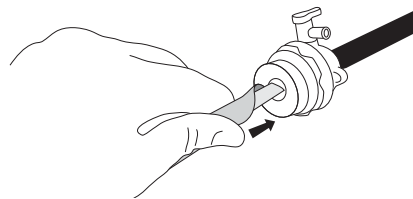

4. Support the rolled Veriset™ patch at the leading edge and insert through the trocar (10 mm to 12 mm).

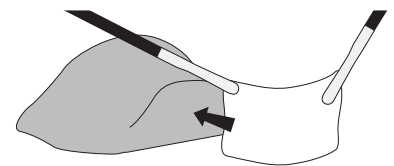

5. Identify the darker side of the Veriset™ patch. Place the darker side on the bleeding site.

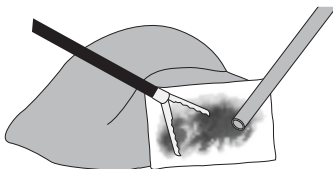

6. Hold the Veriset™ patch firmly in place by providing uniform pressure over the entire patch surface to ensure proper adherence. After 30 seconds, discontinue pressure and visually inspect the bleeding site for haemostasis. If bleeding has not stopped, reapply pressure and reassess after 30 seconds.



7. Nonhydrated portions of the Veriset™ patch should be adhered to the surface by hydrating with saline and applying pressure.


Do not remove Veriset™ patch at the end of surgery. Veriset™ patch absorbs after ~28 days.¹

THE CONFIDENCE TO KNOW BLEEDING WILL STOP.


Veriset™ Haemostatic Patch

OPEN SURGERY

Preparing Veriset™ Haemostatic Patch

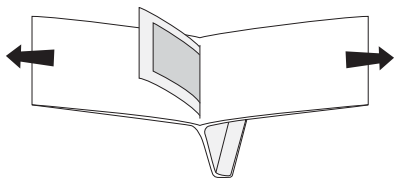


1. Non-sterile person opens the foil pouch.

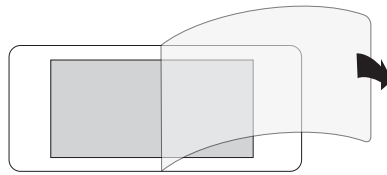

2. Sterile person opens the Tyvek™ pouch. Use the Veriset™ patch within 60 minutes after opening. Veriset™ patch should be placed on the bleeding site, extending ~1–2 cm beyond the site. Veriset™ patch may be cut as necessary.

Applying Veriset™ Haemostatic Patch

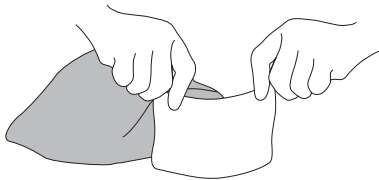

3. Place the darker side of Veriset™ patch on the bleeding site.



4. Hold Veriset™ patch firmly in place. Apply uniform pressure to ensure adherence to the bleeding site. A swab may be used. After 30 seconds, discontinue pressure and inspect the bleeding site.

Do not move or peel away Veriset™ patch; it may not re-adhere to the bleeding site. If bleeding has not stopped, reapply pressure and reassess after 30 seconds.



5. Using a syringe filled with saline, it is essential to hydrate all non-hydrated portions of Veriset™ patch. Apply pressure for 30 seconds to the newly hydrated portions.

Do not remove Veriset™ patch at the end of surgery. Veriset™ patch absorbs after ~28 days.¹

1. Veriset™ Haemostatic Patch Instructions for Use and Product Specifications [package insert]. Minneapolis, MN: Medtronic, 2014.

IMPORTANT: Please refer to the package insert for complete instructions, contraindications, warnings and precautions.

© 2016 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. 16-eu-veriset-or-guide-lap-open-combined-759527.

Medtronic



To contact us, please visit
[medtronic.com/covidien/support/emea-customer-service](https://www.medtronic.com/covidien/support/emea-customer-service)

Use scan app to read