

# APPOSE™ ULC SKIN STAPLERS



Single Use Skin Staplers  
with Stainless Steel Staples

**Medtronic**  
Further, Together

## FEATURE

- 2 staple width options
- 2 staple height options
- Clear plastic cartridge

## BENEFIT

- Wound closure flexibility
- Enhanced cosmesis
- Enhance staple count visibility

### Proper Positioning of Appose™ ULC Single Use Skin Stapler

The Appose™ ULC Skin Stapler is designed to be fired from multiple angles. It has an ejector spring that automatically disengages the staple to facilitate instrument removal in any direction.

### How to Avoid Incomplete Staple Formation

The flat, dimpled staple top span is designed to ensure proper staple formation. To use the skin stapler properly, squeeze the trigger completely and release. Do NOT pull or lift up the stapler when releasing the trigger. Failure to completely squeeze the handle can result in incomplete staple formation and insufficient wound approximation.





### Appose™ ULC Single Use Skin Stapler Sizes

The Appose™ ULC Single Use Skin Staplers have two different staple sizes: Regular and Wide. In the closed position, the regular staple has a width of 5.8 mm and leg length of 4.1 mm; the wide staple has a width of 6.5 mm and leg length of 4.1 mm.

## ORDER INFORMATION

Order Code	Description	Unit/Box
8886803512	Appose™ ULC Skin Stapler with 35 Regular staples	12
8886803712	Appose™ ULC Skin Stapler with 35 Wide staples	12
150462	Premium™ Single Use Skin Staple Remover	12

### APPOSE™ ULC STAPLE SPECIFICATIONS

	Size	Closed Position	Wire
35R	10.87 mm 	5.88 mm 	0.57 mm
35W	13 mm 	5.88 mm 	0.56 mm

**Fully squeeze and release trigger BEFORE lifting.**



**Do Not lift or rotate until staple is released.**



### Stainless Steel Composition of Staple

Carbon	0.03%
Chrome	16.0 -19.0%
Nickel	10.0 -15.5%
Manganese	2.0%
Silicon	1.0%
Sulphur	0.03%
Phosphorus	0.045%
Molybdenum	2.0 - 3.0%
Nitrogen	0.10%
Copper	0.50%
Iron	Balance

**IMPORTANT:** Please refer to the package insert for complete instructions, contraindications, warnings and precautions.

© 2016 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.  
16-eu-appease-sell-sheet-855165

**Medtronic**



To contact us, please visit  
[medtronic.com/covidien/support/emea-customer-service](https://www.medtronic.com/covidien/support/emea-customer-service)

Use scan app to read