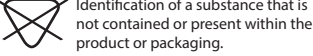
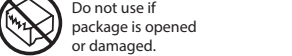
en
----

**Indications for Use**

The BIS Pediatric Sensor is applied directly to the patient’s skin to enable recordings of electrophysiological (such as EEG) signals in pediatric patients.

- |  |   |   |   |
|--|---|---|---|
|  | Identification of a substance that is not contained or present within the product or packaging. |  | Do not use if package is opened or damaged. |
|--|---|---|---|

**WARNING**

**TO REDUCE THE HAZARD OF BURNS DURING USE OF BRAIN-STIMULATING DEVICES (e.g., TRANSCRANIAL ELECTRICAL MOTOR EVOKED POTENTIAL), PLACE STIMULATING ELECTRODES AS FAR AS POSSIBLE FROM THE BIS SENSOR AND PLACE SENSOR ACCORDING TO INSTRUCTIONS.**

**CAUTIONS**

Patient position may increase the risk of skin irritation on the forehead. With patients in prone position, consider minimizing pressure on the sensor.

**NOTE:** Do not use if sensor is dry. To avoid dry out, do not open pack until ready for use. Due to intimate skin contact, reuse may pose risk of infection. If skin rash or other unusual symptoms develop, stop use and remove. Limited to short-term use (maximum 24 hours). Do not cut sensor, as this will result in improper operation. Upon removal, slight redness of skin may occur and typically resolves within a short period of time.

**INSTRUCTIONS:**

- Wipe skin with alcohol and dry
- Position sensor diagonally on the forehead as follows:
  - 1 at center of forehead, approximately 1.5 inches (4cm) above the bridge of nose
  - 4 directly above eyebrow
  - 3 on temple, between corner of eye and hairline
- Press edges of sensor to assure adhesion
- Press 1, 2, 3 and 4 firmly for 5 seconds
- Insert sensor tab into patient interface cable
  - To minimize twisting of the sensor, where it connects to the cable, consider securing with tape and/or relieving cable strain.

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