
Surestream™


ETCO₂ SAMPLING

Products for Intubated Patients Directions for Use

Sure VentLine™ Set Adult/Pediatric

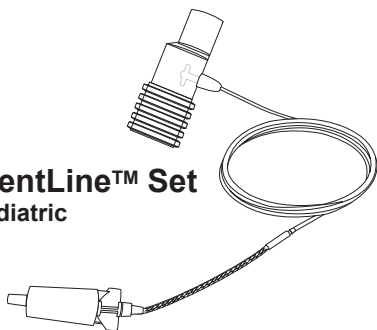Sure VentLine™ H Set Adult/Pediatric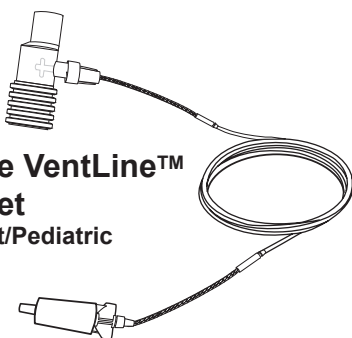



Surestream™ EtCO₂ Sampling for Intubated Patients

Instructions for use:

1. Firmly connect the small-end (male) of the Surestream™ Airway Adapter **1** to the female-end of the Wye piece (ventilation source).
2. Firmly connect the patient's endotracheal tube connector into the large-end (female) **2** of the Surestream™ Airway Adapter.
3. Ensure that the tubing is not twisted or crimped.
4. Insert the Surestream connector **3** securely but gently into the CO₂ input connector on the device **4**.

Instructions for use with a closed suction system:

1. Firmly connect the large-end (female) **2** of the Surestream™ airway adapter to the ventilator connector on the closed suction system.

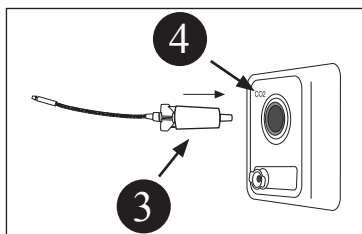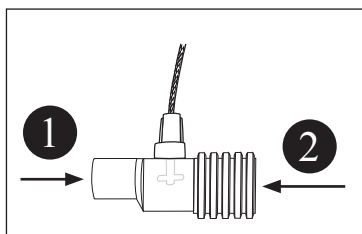Surestream™ EtCO₂ Sampling for Intubated Patients Adult and Pediatric Patients

**Sure VentLine® Sets are for short-term use.
Sure VentLine® H Sets are for humidified
ventilation and long-term use.**

Airway Adapters for Adult to Pediatric patients:

- Added dead space <6.6 cc.
- To be used with an endotracheal tube > 4.5mm bore.

Single patient use, not to be re-processed

Non-sterile / non-pyrogenic

Does not contain natural rubber latex

Warning: When using with a closed suction system, do not place the airway adapter (tee) between the suction catheter and endotracheal tube. This is to ensure that the airway adapter does not interfere with the functioning of the suction catheter.

Warning: Loose or damaged connections may compromise ventilation or cause an inaccurate measurement of respiratory gases. Securely connect all components and check connections for leaks according to standard clinical procedures.

Warning: Check CO₂ and O₂ tubing regularly during use to ensure that no kinks are present. Kinked tubing may cause inaccurate CO₂ sampling or affect O₂ delivery to patient.

Caution: Ensure that tubing is not stretched during use.

Caution: Do not attempt to clean, disinfect, sterilize or flush any part of the sampling line as this can cause damage to the monitor.

Caution: Dispose of sampling lines according to standard operating procedures or local regulations for the disposal of contaminated medical waste.

Caution: Use of the Sure VentLine® H Set (the H in its name indicates that it is for use in humidified environments) during MRI scanning may cause interference. The use of non H sampling lines is advised.

Note: During nebulization or suction, in order to avoid moisture buildup and sampling line occlusion, remove the sampling line luer connector from the monitor.

Note: Replace the sampling line according to hospital protocol or when a blockage is indicated by the device. Excessive patient secretions or a buildup of liquids in the airway tubing may occlude the sampling line, requiring more frequent replacement.

Federal law (U.S.) restricts this device to sale by or on the order of a physician.

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Oridion Medical 1987 Ltd.
Oridion Capnography Inc.
Obelis S.A.