
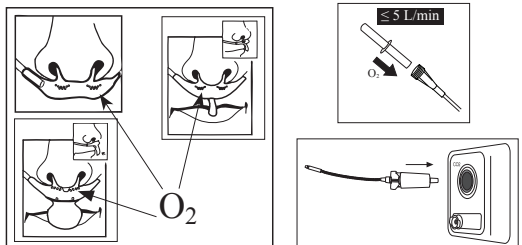
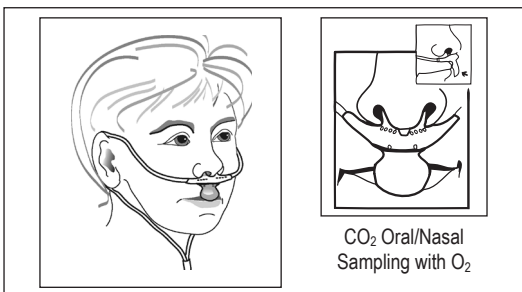
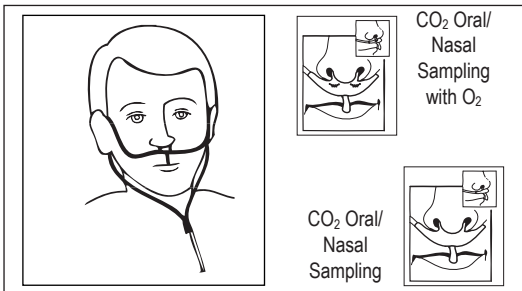
Surestream™

ETCO₂ SAMPLING

Products for Non-Intubated Patients

Directions for Use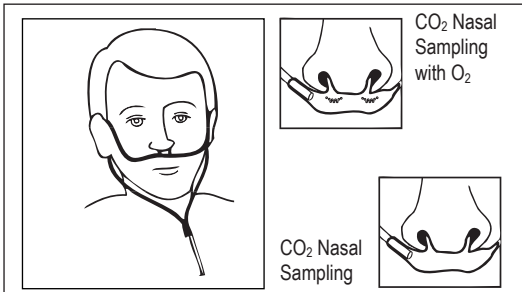

en

Surestream™ EtCO₂ Sampling for Non-Intubated Patients

Single patient use, not to be re-processed.

Non-sterile / non-pyrogenic.

Does not contain natural rubber latex

Warning: Loose or damaged connections may compromise ventilation or cause an inaccurate measurement of respiratory gases. Securely connect all components and check connections for leaks according to standard clinical procedures.

Warning: Do not cut or remove any part of the sample line. Cutting the sample line could lead to erroneous readings.

Warning: Check CO₂ and O₂ tubing regularly during use to ensure that no kinks are present. Kinked tubing may cause inaccurate CO₂ sampling or affect O₂ delivery to patient.

Caution: Ensure that tubing is not stretched during use.

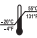
Caution: Do not attempt to clean, disinfect, sterilize or flush any part of the sampling line as this can cause damage to the monitor.

Caution: Dispose of sampling lines according to standard operating procedures or local regulations for the disposal of contaminated medical waste.

Caution: Use of a CO₂ sampling line with H in its name (indicating that it is for use in humidified environments) during MRI scanning may cause interference. These sampling lines include Smart SureLine Plus H O₂ and SureLine H O₂. The use of non H sampling lines is advised.

Note: Replace the sampling line according to hospital protocol or when a blockage is indicated by the device. Excessive patient secretions or a build-up of liquids in the airway tubing may occlude the sampling line, requiring more frequent replacement.

Federal law (U.S.) restricts this device to sale by or on the order of a physician.

	Do not reuse
	Does not contain natural rubber latex
	Fire Hazard
	Storage temperature
	Rx Only
	Caution, consult accompanying documents
	CE Mark

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Oridion Medical 1987 Ltd.
Oridion Capnography Inc.
Obelis S.A.