
Microstream[®]


ETCO₂ SAMPLING

**Products for Intubated
Patients
Adult and Pediatric Patients
Directions for Use**


FilterLine[®] Sets

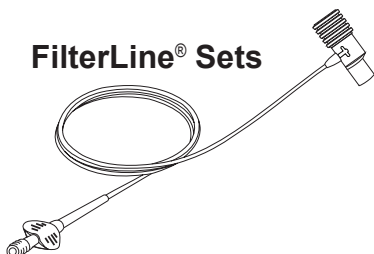FilterLine[®] H Sets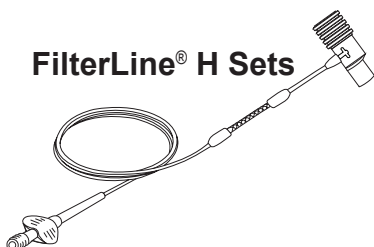

VitaLine[™] H Sets

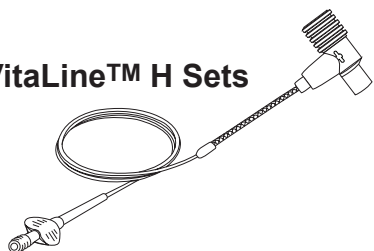US Patents: www.covidien.com/patents.

© 2015 Oridion Medical 1987 Ltd.
All rights reserved.

PN MQ04616N

Microstream® EtCO₂ Sampling for Intubated Patients Adult and Pediatric Patients

FilterLine® Sets for short-term use

FilterLine® H Sets for humidified ventilation
and long-term use

VitaLine™ H Sets for ambient humid environments up to 75% ambient relative humidity

Airway Adapters for Adult to Pediatric patients:

- Added dead space <6.6 cc.
- To be used with an endotracheal tube > 4.5mm bore.
- Suitable for use with 22mm and 15mm patient tubing.

For use with all Microstream-enabled monitors.

Single patient use, not to be re-processed.

Non-sterile / non-pyrogenic.

Does not contain natural rubber latex

Warning: When used with closed suction system, do not place the airway adapter between the suction catheter and endotracheal tube. This is to ensure that the airway adapter does not interfere with the functioning of the suction catheter.

Warning: Loose or damaged connections may compromise ventilation or cause an inaccurate measurement of respiratory gases. Securely connect all components and check connections for leaks according to standard clinical procedures.

Warning: Check CO₂ and O₂ tubing regularly during use to ensure that no kinks are present. Kinked tubing may cause inaccurate CO₂ sampling or affect O₂ delivery to patient.

Caution: Ensure that tubing is not stretched during use.

Caution: Do not attempt to clean, disinfect, sterilize or flush any part of the sampling line as this can cause damage to the monitor.

Caution: Dispose of sampling lines according to standard operating procedures or local regulations for the disposal of contaminated medical waste.

Caution: Use of a CO₂ sampling line with H in its name (indicating that it is for use in humidified environments) during MRI scanning may cause interference. These sampling lines include FilterLine H Set/Long and VitaLine H Set. The use of non H sampling lines is advised.

Note: During nebulization or suction, in order to avoid moisture buildup and sampling line occlusion, remove the sampling line luer connector from the monitor.

Note: Replace the sampling line according to hospital protocol or when a blockage is indicated by the device. Excessive patient secretions or a build-up of liquids in the airway tubing may occlude the sampling line, requiring more frequent replacement.

Note: When connecting a sampling line to the monitor, insert the sampling line connector clockwise into the monitor CO₂ port and turn the connector until it can no longer be turned, to ensure that it is connected securely to the monitor. This will assure that there is no leak of gases during measurement at the connection point and that measurement accuracy is not compromised.

Note: Following connection of the CO₂ sampling line, check that CO₂ values appear on the monitor display.



- Note:** During setup, ensure that airway adapter can be easily attached and detached from the tubing before proceeding.
- Note:** Expect delay times for the following products to be longer than the standard by the given times: Long (4m) FilterLines/CapnoLines - 3 seconds.
- Note:** Sampling lines with H in their names include a moisture reduction component (Nafion®* or its equivalent) for use in higher humidity environments where long duration use of CO₂ sampling is required.

Federal law (U.S.) restricts this device to sale by or on the order of a physician.

Microstream® EtCO₂ Sampling for Intubated Patients

Instructions for use:

1. Firmly connect the small-end (male) of the Microstream® Airway Adapter **1** to the female-end of the Wye piece (ventilation source).
2. Firmly connect the patient's endotracheal tube connector into the large-end (female) **2** of the Microstream® Airway Adapter.
3. Ensure that the tubing is not twisted or crimped.
4. Twist the female luer connector **3** securely but gently into the Microstream® input connector **4**.
5. Following connection of the CO₂ sampling line, check that CO₂ values appear on the monitor display.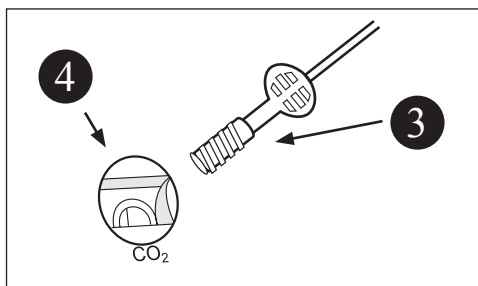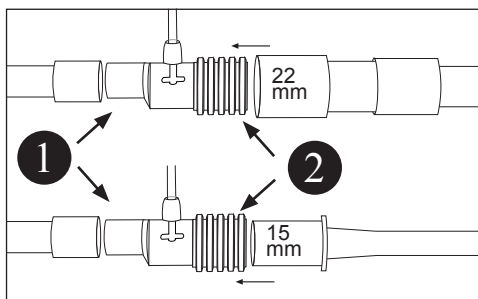


Microstream® EtCO₂ Sampling with a Closed Suction System

Instructions for use with a closed suction system:

1. Set up the parts of the system as seen in D, below. See legend.
2. Connect the system as seen in E, below, as follows:
3. Firmly connect the small-end (male) of the Microstream® Airway Adapter **6** to the female-end of the Wye piece (ventilation source) **7**.
4. Firmly connect the large-end (female) of the Microstream® airway adapter **6** to the ventilator connector on the closed suction system **8** so that the airway adapter does not interfere with the functioning of the suction tubing **9**.
5. The setup seen at bottom right is incorrect; if set up in this way, the airway adapter will interfere with the functioning of the suction tubing.


Setup of Microstream EtCO₂ Sampling with a closed suction system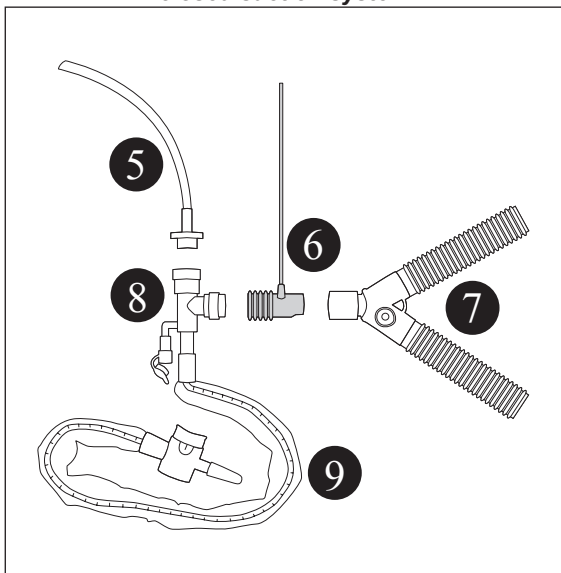


Legend

1. Male-end of airway adapter
2. Female end of airway adapter
3. Female luer connector
4. Microstream® input port
5. Endotracheal tube
6. FilterLine Set with airway adapter
7. Wye piece (ventilation source)
8. Ventilator connector on closed suction system, with lavage
9. Suction tubing

**Correctly Connected
Microstream EtCO₂ Sampling
with a closed suction system**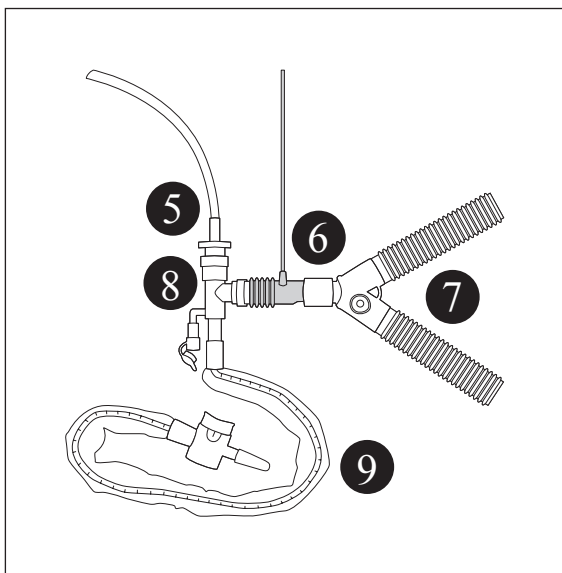

**Incorrectly Connected
Microstream EtCO₂ Sampling
with a closed suction system**

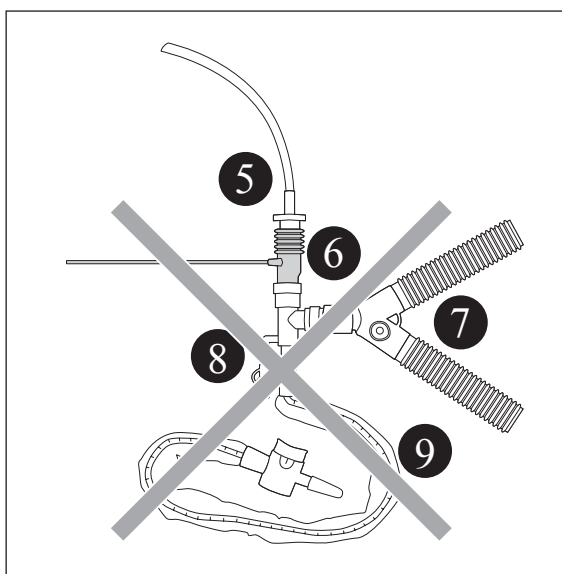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 Single patient use	For use with a single patient
Rx ONLY	Rx Only
 Consult instructions for use	Consult instructions for use
 Caution, consult accompanying documents	Caution, consult accompanying documents
 -45°C to 158°F / 70°C Temperature limitation	Storage temperature
 This product does not contain natural rubber latex	Does not contain natural rubber latex
 MR Unsafe	Not safe for use in MR environments
	CE Mark

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Oridion®, Microstream®, FilterLine®, VitaLine™, Smart CapnoLine®, CapnoLine®, and Smart CapnoLine Guardian™ are trademarks or registered trademarks of Oridion Medical 1987 Ltd.™* and ®* are trademarks of their respective owners.

Oridion Medical 1987 Ltd.
7 Hamarpe St.
P.O. Box 45025
Jerusalem 9777407, Israel
US Toll Free: 1-888-674-3466
1-888-ORIDION

Obelis S.A.
Bd Général Wahis 53
1030 Brussels
Belgium

www.oridion.com