
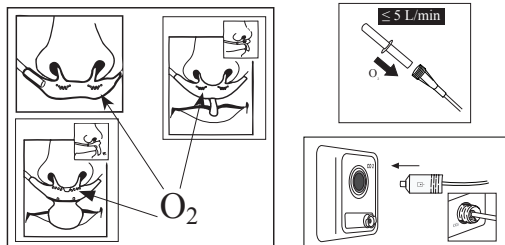
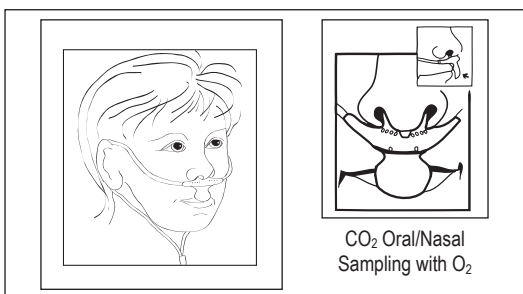
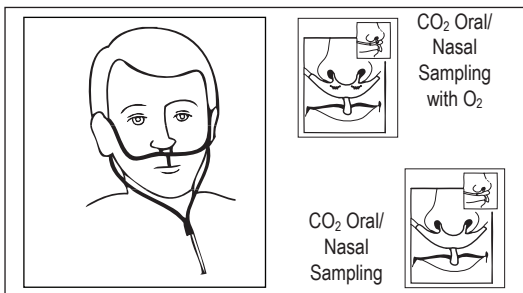
Surestream™

ETCO₂ SAMPLING

Products for Non-Intubated Patients

Directions for Use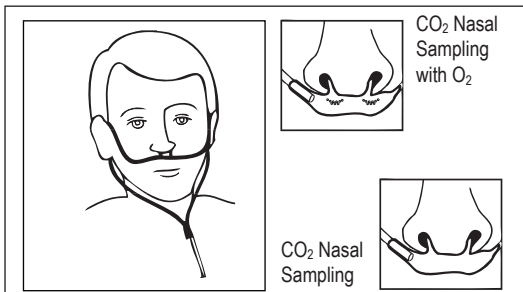

US Patents: www.covidien.com/patents.

© 2015 Oridion Medical 1987 Ltd.
All rights reserved.

PN 015147D

Oridion®

Surestream™ EtCO₂ Sampling for Non-Intubated Patients

Single patient use, not to be re-processed.

Non-sterile / non-pyrogenic.

Does not contain natural rubber latex

Warning: Loose or damaged connections may compromise ventilation or cause an inaccurate measurement of respiratory gases. Securely connect all components and check connections for leaks according to standard clinical procedures.

Warning: Do not cut or remove any part of the sample line. Cutting the sample line could lead to erroneous readings.

Warning: Check CO₂ and O₂ tubing regularly during use to ensure that no kinks are present. Kinked tubing may cause inaccurate CO₂ sampling or affect O₂ delivery to patient.

Caution: Ensure that tubing is not stretched during use.

Caution: Do not attempt to clean, disinfect, sterilize or flush any part of the sampling line as this can cause damage to the monitor.

Caution: Dispose of sampling lines according to standard operating procedures or local regulations for the disposal of contaminated medical waste.

Note: Replace the sampling line according to hospital protocol or when a blockage is indicated by the device. Excessive patient secretions or a build-up of liquids in the airway tubing may occlude the sampling line, requiring more frequent replacement.

Note: Sampling lines are recommended for use with oxygen provided at up to 5l/min. At higher levels of oxygen provision, dilution of CO₂ readings may occur, leading to lower CO₂ values.

Federal law (U.S.) restricts this device to sale by or on the order of a physician.

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 Single patient use	For use with a single patient
Rx ONLY	Rx Only
 Consult instructions for use	Consult instructions for use
 Caution, consult accompanying documents	Caution, consult accompanying documents
 -4°F -20°C 131°F 55°C Temperature limitation	Storage temperature
 PHT DEHP	DEHP
 This product does not contain natural rubber latex	Does not contain natural rubber latex
 Fire hazard	Fire Hazard
 0482	CE Mark

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Oridion®, Microstream®, FilterLine®, Smart CapnoLine®, CapnoLine®, Smart CapnoLine Guardian™, and Surestream™ are trademarks or registered trademarks of Oridion Medical 1987 Ltd. ™* and ®* are trademarks of their respective owners.

Oridion Medical 1987 Ltd.
7 Hamarpe St.
P.O. Box 45025
Jerusalem 9777407, Israel
US Toll Free: 1-888-674-3466
1-888-ORIDION

Obelis S.A.
Bd Général Wahis 53
1030 Brussels
Belgium

www.oridion.com