Nellcor™ SpO₂ Ear Sensor Clip Reusable

- en** Instructions For Use
- fr** Mode d'emploi
- de** Gebrauchsanweisung
- nl** Gebruiksaanwijzing
- es** Instrucciones de uso
- it** Istruzioni per l'uso
- pt** Instruções de utilização

This device is not made with natural rubber latex or DEHP.

Indications/Contraindications

The *Nellcor™ SpO₂ Ear Sensor Clip Reusable*, model D-YSE, is intended for use **only** with the *Nellcor SpO₂ Sensor Multisite Reusable*, model D-YS, when continuous, noninvasive arterial oxygen saturation and pulse rate monitoring are required. It is indicated for use for patients weighing 30 kg or more, using the ear lobe or pinna as a monitoring site. The device is for use in any environment where noninvasive oxygen saturation monitoring is desired and compatible pulse oximetry monitors are indicated for use.

Use of the ear clip with the D-YS sensor is contraindicated for patients whose ears are pierced at the indicated application sites.

Instructions for Use

- 1) Read the Directions for Use for the D-YS sensor prior to use of this product.
 - 2) Clean the ear clip and the D-YS sensor before and after each patient use (refer to Cleaning section).
 - 3) Press open the ear clip with the thumb and forefinger, as shown (A).
 - 4) Slide one of the D-YS sensor alignment buttons along the ear clip slot until the sensor pad is *fully engaged* in the clip (B).
 - 5) Slide the second D-YS sensor button along the other ear clip slot until the second sensor pad is *fully engaged* in its side of the clip (C).
- Note: Be sure that the D-YS sensor pads are oriented as shown in the illustrations. The optical components must *face each other*.
- 6) The sensor is now ready to be applied to the patient. The appropriate sensor sites are either the ear lobe (a), or the ear pinna (b), shown in Figure (D).
 - 7) Clip the sensor onto the ear so that the sensor cable runs down the side of the patient's face and body (E). If problems arise in fitting the sensor to the ear with the ear clip, consider using the D-YS sensor (with an adhesive wrap instead of the ear clip) on another body site, as described in the Directions for Use for the D-YS sensor.

Removal

Caution: For the comfort of the patient and to avoid damaging the sensor, do not pull on the D-YS sensor cable to remove the sensor and ear clip from the patient.

To remove the ear clip and sensor from the patient's ear, press open the ear clip and remove.

To remove the D-YS sensor from the ear clip, grasp each sensor pad and gently slide it out of the ear clip. **Do not pull on the cable.**

Cleaning

Caution: To avoid damage to the D-YS sensor, remove the sensor from the ear clip before cleaning either piece. To clean the sensor, refer to the instructions in the D-YS sensor Directions for Use.

Caution: Do not sterilize the ear clip by irradiation, steam, or ethylene oxide.

The ear clip may be cleaned by wiping or soaking it (for 10 minutes) in isopropyl alcohol (70%). If the ear clip is soaked, be sure to rinse it with water and air dry it prior to use on the next patient.

After each cleaning and prior to use again, the ear clip should be inspected for cracking or breakage, and discarded if any defects are noted.

Discard if loss of spring tension allows slippage or movement of the ear clip from its proper position on the ear lobe or pinna. Slippage caused by loss of spring tension may result in inaccurate sensor readings.

Warnings

- 1) Keep the ear clip out of the reach of children under the age of 3. It is small enough to be swallowed and may block the trachea.
- 2) The D-YS sensor and ear clip may be used on the same site for up to 4 hours. However, the sensor site must be inspected routinely to ensure skin integrity and correct positioning. If skin integrity changes, move the sensor to another site.
- 3) Do not use supplemental tape to adhere the ear clip and sensor to the site; this can restrict blood flow and cause inaccurate measurements.
- 4) Do not expose the sensor and ear clip to strong ambient light during clinical use; this may cause inaccurate measurements.
- 5) Do not use the sensor with the ear clip on any site other than those on the ear. This may result in inaccurate readings due to improper positioning of the sensor's optical components.
- 6) Failure to apply the D-YS sensor and ear clip properly may cause incorrect measurements. If variations in application site configuration or size prevent proper application, remove the ear clip from the sensor and apply the D-YS sensor to another site, according to the D-YS sensor Directions for Use.
- 7) Performance of the D-YS sensor and ear clip may be compromised by motion, particularly of a repetitive nature. If readings cannot be obtained, try another application site on the same ear or a site on the other ear. If readings still cannot be obtained, it may be necessary to remove the ear clip from the sensor and apply a *Nellcor* sensor to another body site, according to the Directions for Use included with the sensor.
- 8) As with all medical equipment, carefully route sensor wires and patient cabling to reduce the possibility of patient entanglement or strangulation.
- 9) If the sensor is misapplied with excessive pressure for prolonged periods, a pressure injury can occur.

Note: Refer to the D-YS sensor Directions for Use, and the oximeter operator's manual for complete instructions for use of the sensor with the monitor.

Warranty

To obtain information about a warranty, if any, for this product, contact Covidien Technical Services or your local Covidien representative.

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© 2011 Covidien.
COVIDIEN, COVIDIEN with logo and Covidien logo are U.S. and internationally registered trademarks of Covidien AG. Other brands are trademarks of a Covidien company.
Covidien LLC, 15 Hampshire Street, Mansfield, MA 02048 USA.
Covidien Ireland Limited, IDA Business & Technology Park, Tullamore.