

# Nellcor™

## Pulse Oximetry Interface Cable


<b>en</b>	Instructions For Use
<b>fr</b>	Mode d'emploi
<b>de</b>	Gebrauchsanweisung
<b>nl</b>	Gebruiksaanwijzing
<b>it</b>	Istruzioni per l'uso
<b>es</b>	Instrucciones de uso
<b>sv</b>	Bruksanvisning
<b>da</b>	Brugsanvisning
<b>no</b>	Bruksanvisning
<b>fi</b>	Käyttöohjeet
<b>pt</b>	Instruções de uso
<b>ru</b>	Инструкция по применению
<b>pl</b>	Instrukcja użytkowania
<b>cs</b>	Návod k použití
<b>sl</b>	Navodila za uporabo
<b>hu</b>	Használati útmutató
<b>el</b>	Οδηγίες χρήσης
<b>tr</b>	Kullanma Talimatı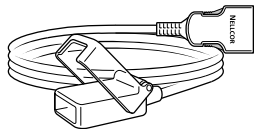


**1**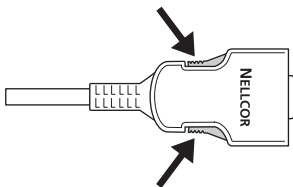

**2**

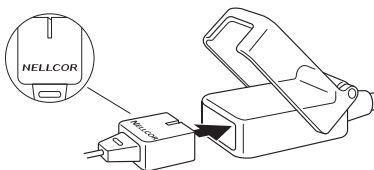**3**
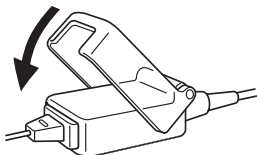
# MC10

## Pulse Oximetry Interface Cable

Identification of a substance that is contained or present within the product or packaging.

Identification of a substance that is not contained or present within the product or packaging.

**IP22**

Protected against access to hazardous parts with a finger and protected against vertically falling water drops when sensor tilted up to 15°.

## Directions for Use

---

This device is not made with natural rubber latex or DEHP.

### Description/Intended Use

The Nellcor™ Pulse Oximetry Interface Cable, model MC10, is a 10 foot (3 meter) cable that links a Nellcor oxygen sensor to a Nellcor or Nellcor-compatible monitor. The MC10 cable is for use with Nellcor oxygen sensors, except for Nellcor oxygen sensors with white plugs.

### Instructions for Use (Illustrations on page 2)

Use ONLY Nellcor oxygen sensors that have one side of the connecting plug slotted. When connecting the sensor to the cable, always orient the sensor plug so that NELLCOR is visible.

1. To connect the MC10 cable to the monitor, depress the buttons on both sides of the cable connector as it is inserted into the monitor socket. Release the buttons after the cable connector is firmly seated in the socket. **(1)**
2. Plug the sensor into the other end of the MC10 cable with the clear plastic sensor latch. Ensure that NELLCOR is visible on the sensor plug as it is connected to the cable. Plug the sensor straight in; do not twist. **(2)**
3. Snap the clear plastic sensor latch in place over the sensor plug. **(3)**
4. To disconnect the sensor from the MC10 cable, open the clear plastic sensor latch. Grasp the sensor plug, and pull it straight out from the cable connector; do not twist. Do not disconnect the sensor by pulling it out by the sensor cable.
5. To disconnect the MC10 from the monitor, depress the buttons on both sides of the cable connector. Pull the connector straight out from the monitor socket and release the buttons; do not twist. **(1)**

## Cleaning

Clean the cable by wiping it with isopropyl alcohol.

## WARNINGS

1. **Examine cable before use. Do not use a damaged cable or one with exposed wires.**
2. **Do not connect the MC10 to any monitor for which the MC10 is not intended for use, as this could result in electromagnetic interference or susceptibility to or from other devices within the patient environment.**
3. **As with all medical equipment, carefully route cables to reduce the possibility of patient entanglement or strangulation.**

## CAUTIONS

1. Do not sterilize. This cable is not designed for sterilization, including irradiation, or exposure to steam or ethylene oxide. Do not immerse the cable in cleaning solutions, disinfecting solutions, or water. Damage to the cable can result.
2. Electrosurgical interference may compromise performance.
3. When disconnecting the cable from the monitor, always depress the buttons on both sides of the connector as it is removed from the monitor socket. Failure to do so may result in damage to the cable connector or to the monitor socket.
4. To avoid damaging the MC10, grasp the connector rather than the cable when disconnecting it from the sensor or from the monitor.
5. Check cable connections. Failure to properly connect the plugs will result in intermittent readings or no readings.

## Warranty

To obtain information about a warranty for this product, contact Covidien Technical Services or your local Covidien representative.

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© 2011 Covidien.
registered trademarks of Covidien AG.
Other brands are trademarks of a Covidien company.
Covidien Inc, 15 Hampshire Street,
**EC REP** Covidien Ireland Limited,