

# BARRX™ RADIOFREQUENCY ABLATION PROCEDURAL QUICK GUIDE

	CATHETER	ABLATION/ CLEANING PROTOCOL	ENERGY SETTING
<b>BARRETT'S ESOPHAGUS</b>	Barrx™ 360 express RFA balloon catheter	1 - clean - 1	10 J/cm <sup>2</sup>
	Barrx™ channel RFA endoscopic catheter	2 - clean - 2	12 J/cm <sup>2</sup> (NDBE & Dysplasia)
	Barrx™ 60 RFA focal catheter	2 - clean - 2	
	Barrx™ 90 RFA focal catheter	2 - clean - 2	
Barrx™ ultra long RFA focal catheter	1 - clean - 1		
<b>GAVE</b>	Barrx™ channel RFA endoscopic catheter	1 - 4 (no cleaning)	12 J/cm <sup>2</sup>
	Barrx™ 60 RFA focal catheter		
	Barrx™ 90 RFA focal catheter		
	Barrx™ ultra long RFA focal catheter		
<b>RADIATION PROCTITIS</b>	Barrx™ channel RFA endoscopic catheter	1 - 2 (no cleaning)	12 J/cm <sup>2</sup>
	Barrx™ 60 RFA focal catheter		
	Barrx™ 90 RFA focal catheter		

## NO SALINE OR LUBRICATION

### ITEMS NEEDED:




- 4x4 gauze pads
- Mucomyst
- Bite block
- Sterile water
- Alcohol pad (ultra long)

Treatment area footprint (left to right):  
Barrx™ ultra long RFA focal catheter,  
Barrx™ 90 RFA focal catheter, Barrx™  
60 RFA focal catheter, and the Barrx™  
channel RFA endoscopic catheter.

Questions? Call or email your  
Medtronic representative.  
For full details, please refer to the  
product's Instructions For Use (IFU).

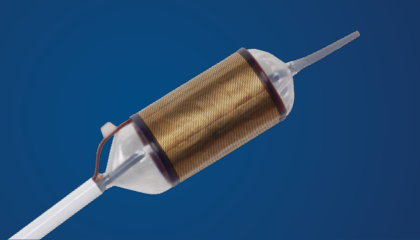

(Not shown in actual size)

CATHETER	SPECIFICATIONS	PRODUCT CODES
<p>Barrx™ 360 express RFA balloon catheter</p> 	<p>Balloon length: 8 cm            Electrode length: 4 cm            Catheter shaft diameter: 7 mm            Catheter shaft working length: 85 cm            Maximum no. of ablations: 16</p>	<p>64082</p>
<p>Barrx™ channel RFA endoscopic catheter</p> 	<p><b>RECOMMENDED SCOPE SIZES:</b>            endoscopes with working channel diameter of 2.8 mm or larger  <b>ELECTRODE:</b>            15.7 mm length – 7.5 mm width (when “wings” are unfolded)            Catheter shaft diameter: 2.5 mm            Catheter shaft length: 135 cm            Maximum no. of ablations: 120</p>	<p>TTS-1100</p>
<p>Barrx™ 60 RFA focal catheter</p> 	<p><b>RECOMMENDED SCOPE SIZES:</b>            8.6 mm to 9.8 mm  <b>ELECTRODE:</b>            15 mm length – 10 mm width            Catheter shaft diameter: 4 mm            Catheter shaft length: 160 cm            Maximum no. of ablations: 80</p>	<p>90-9300</p>
<p>Barrx™ 90 RFA focal catheter</p> 	<p><b>RECOMMENDED SCOPE SIZES:</b>            8.6 mm to 12.8 mm  <b>ELECTRODE:</b>            20 mm length – 13 mm width            Catheter shaft diameter: 4 mm            Catheter shaft length: 160 cm            Maximum no. of ablations: 80</p>	<p>90-9100</p>
<p>Barrx™ ultra long RFA focal catheter</p> 	<p><b>RECOMMENDED SCOPE SIZES:</b>            8.6 mm to 9.8 mm  <b>ELECTRODE:</b>            40 mm length – 13 mm width            Catheter shaft diameter: 4 mm            Catheter shaft length: 160 cm            Maximum no. of ablations: 80</p>	<p>90-9200</p>

[medtronic.com/gi](https://www.medtronic.com/gi)

**Caution:** Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner. Rx only.

**Risk Information:** The following are transient side effects that may be expected after treatment: chest pain, difficulty swallowing, painful swallowing, throat pain, and/or fever. Complications include mucosal laceration, minor or major bleeding, endoscopic clipping to manage mucosal laceration or bleeding, perforation of the stomach, esophagus, or pharynx, surgery to manage perforation, esophageal stricture, endoscopic dilation to manage stricture, pleural effusion, transfusion secondary to major bleeding, cardiac arrhythmia, aspiration and infection. Potential complication that has not been observed is death.

Please refer to the product user manual for additional detailed information.

US130189(4) © 2020 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

**Medtronic**  
 Further, Together