
# Re/Flex™

## Reusable Inspiratory Filter

**REF** 4-074600-00  
4-074602-00  
4-074604-00  
4-074643-00  
4-074644-00

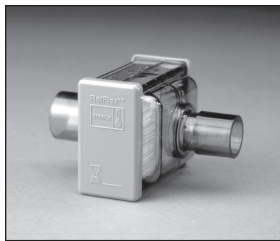**PURITAN  
BENNETT**

English	2
Français	5
Deutsch	8
Nederlands	11
Italiano	14
Español	17
Svenska	20
Dansk	23
Norsk	26
Suomi	29
Português	32
Русский	35
Polski	38
Česky	41
Slovensky	44
Magyar	47
Ελληνικά	50
Türkçe	53

## Re/Flex™

Reusable Inspiratory Filter

**REF** 4-074600-00, 4-074602-00, 4-074604-00, 4-074643-00, 4-074644-00

 Indicates flow direction. The Re/Flex filter is designed to be used only in this orientation.

## Directions for Use

---

### Description

The *Re/Flex* Reusable Inspiratory Filter minimizes contaminants delivered to the patient, and reduces the possibility of contaminants reentering the ventilator.

### Installation

#### WARNING

1. To ensure that all patient connections are leak-tight, perform a patient circuit leak check every time the *Re/Flex* Reusable inspiratory Filter is used.

#### CAUTION

1. To ensure proper installation, observe the flow direction markings on the filter.

### Use

New *Re/Flex* filters are provided non-sterile, but are ready to use with no further cleaning or disinfection. Handle filters with care; abusive handling can damage filter medium.

### Cleaning and Disinfection

If necessary, wipe filter housing clean with damp cloth. Any cleaning solution used for external wipe-down must be compatible with polysulfone plastic (filter housing and end cap material). The reusable *Re/Flex* filter can be steam autoclaved at temperatures not to exceed 135 °C (275 °F). Effective disinfection normally occurs at a minimum cycle of 132 °C (270 °F) for 20 minutes for gravity-displacement cycles, or 132 °C (270 °F) for 4 minutes for pre-vacuum cycles (requires minimum 20 minute drying

cycle after sterilization). If filter is provided with a coupling, separate coupling from filter before autoclaving. Wrap filter in muslin or equivalent paper before autoclaving. Follow the steam autoclave manufacturer's instructions.

## **WARNING**

- 1. To avoid damage to the filter, do not immerse in liquid or attempt to scrub or touch filter media. Soaking or rinsing filter may result in retention of harmful residues and/or an increase in air flow resistance. Damage to the filter media can reduce filtration efficiency.**
- 2. Do not use ethylene oxide (ETO) to disinfect filter. When handling the filter, always follow your hospital's infection control guidelines for handling infectious material.**

**Note:** We recognize that cleaning, sterilization, sanitation, and disinfection practices vary widely among health care institutions. It is not possible for Puritan Bennett to specify or require specific practices that will meet all needs or to be responsible for the effectiveness of cleaning, disinfection and other practices carried out in patient care settings. We do recommend that users of this product requiring cleaning and disinfection consider the "Standards and Recommended Practices for Sterilization" published by Association for the Advancement of Medical Instruments (AAMI) as well as the Centers for Disease Prevention and Control (CDC) publication, "Guideline for Prevention of Nosocomial Pneumonia."

## **Maintenance**

The *Re/Flex* filter can withstand repeated autoclave cycles at temperatures not exceeding 135 °C (275 °F). Inspect filter before each reuse and discard any filter housing or media showing visible damage, or if filter flow resistance is too high for your application. For Puritan Bennett ventilator applications, we recommend discontinued use of the filter if flow resistance is greater than 4 cmH<sub>2</sub>O at 100 Lpm flow.

Replace filter after one year of service (maximum). Space is provided on filter end cap to note anticipated retirement date. Calculate filter retirement date based on typical use rates.

## Specifications

**Viral/bacterial filtration efficiency:** Greater than 99.999%

**Filter medium:** Minimum filter efficiency of 99.97% retention of particles 0.3  $\mu\text{m}$  nominal at 100 Lpm flow

**Resistance to air flow:** Less than 2  $\text{cmH}_2\text{O}$  at 60 Lpm when new

**Size:** 10.16 cm long x 7.62 cm wide x 6.09 cm deep (4.0 in. x 3.0 in. x 2.4 in.)

**Internal volume:** Approximately 136 ml

**Leakage:** Less than 0.01 Lpm at 3 psi (211  $\text{cmH}_2\text{O}$ ) internal pressure

**Connectors:** The 22 mm connectors meet requirements of the International Standards Organizations (ISO) 5356-1 (Anesthetic and respiratory equipment - conical connectors)

**Note:** Each *Re/Flex* filter is assembled in accordance with Class 100,000 clean room specifications. It is provided non-sterile.

## Ordering Information

REF	Description
4-074600-00	<i>Re/Flex</i> filter, 22mm ISO conical connectors
4-074602-00	<i>Re/Flex</i> filter, Bennett barb connectors
4-074604-00	<i>Re/Flex</i> filter, 22mm ISO male outlet/Bennett barb inlet
4-074644-00	<i>Re/Flex</i> filter, Bennett barb connector with coupling
4-074643-00	<i>Re/Flex</i> filter, 22mm ISO/barb with coupling

*Re/Flex* is a trademark of Puritan-Bennett Corporation.

© Puritan-Bennett Corporation. 2006. All rights reserved.

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*Re/Flex*, Puritan-Bennett Corporation'ın bir ticari markasıdır.
Tyco Healthcare UK LTD