



COVIDIEN


positive results for life™

Puritan Bennett™ 560 Ventilator

Setup Guide


POWERING ON THE VENTILATOR

To turn the ventilator on, set the **I/O** switch (a covered, rocker-type switch located at the rear of the ventilator) to the **I** position, as shown in the figure below.

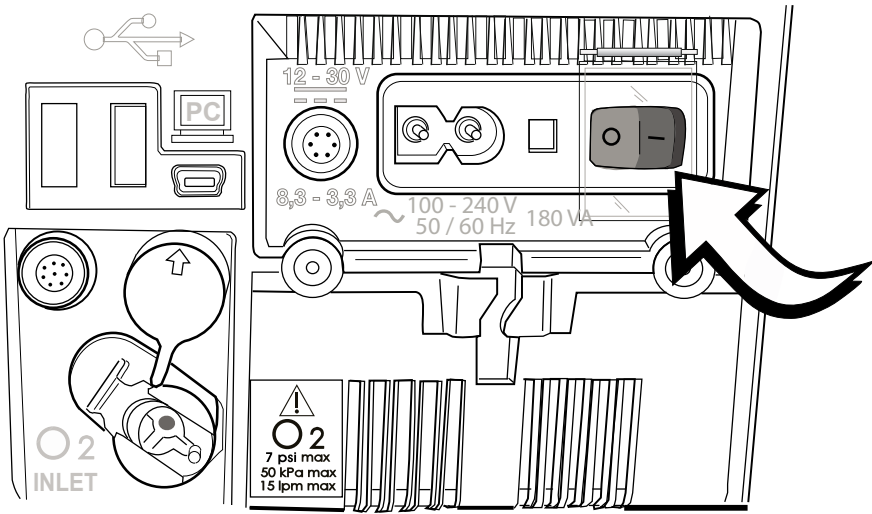


The following events occur:

The front panel indicators flash (except for the indicator showing the type of power supply in use, which remains lit).

- The audible alarms briefly sound.
- The display's backlight turns on.
- The Puritan Bennett™ brand name is displayed momentarily.
- The blue **VENT STDBY** indicator to the right of the **VENTILATION ON/OFF**  key illuminates, indicating the device is in standby mode.

INITIATING VENTILATION

Press **VENTILATION ON/OFF**  button on the front panel.

The blue light indicator, at the upper right of the **VENTILATION ON/OFF**  key, turns off.


- A “beep” sounds.
- The ventilation starts.
- The values of the monitored parameters are displayed in the right-hand window.

STOPPING VENTILATION








1. Hold the **VENTILATION ON/OFF** button for 3 seconds.
2. A double “beep” sounds.
3. Release the **VENTILATION ON/OFF** button. Upon release, ventilation stops, and the blue LED illuminates, indicating that the ventilator is in standby mode.

POWERING OFF THE VENTILATOR

Set the **I/O** switch to the **O** position to power off the ventilator

- The blue **LED** (to the right of the **VENTILATION ON/OFF**  button) turns off.
- The ventilator screen switches off.

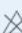SYMBOLS AND DEFINITIONS



	Software lock enabled
ABS/REL	Peak inspiratory pressure monitoring convention <ul style="list-style-type: none">• ABS setting indicates the inspiratory pressure setting will determine the peak inspiratory pressure (monitoring) regardless of the PEEP setting.• REL setting indicates the PEEP is added to the inspiratory pressure setting to determine the peak inspiratory pressure (monitoring).
	Internal battery indicator. Indicates remaining internal battery life in hours and minutes.
	Exhalation valve detected
	No exhalation valve detected
	Apnea alarm has been deactivated
	Audio paused symbol. The sounding of audible alarms is currently disabled.
	Alarm paused symbol. One or more alarms have been paused, or reset/cancelled.



ALARMS

The alarm conditions are indicated by priority level.

VERY HIGH PRIORITY (VHP)	HIGH PRIORITY (HP)	MEDIUM PRIORITY (MP)
IMMEDIATE CRITICAL SITUATION: VENTILATION NOT POSSIBLE	CRITICAL SITUATION IN THE SHORT TERM	CRITICAL SITUATION IN THE LONG TERM
CONTINUOUS RED LED ILLUMINATION	FLASHING RED LED ILLUMINATION	FLASHING YELLOW LED ILLUMINATION
CONTINUOUS SOUND SIGNALING	INTERMITTENT HIGH SPEED SOUND SIGNALING	INTERMITTENT MEDIUM SPEED SOUND SIGNALING

To silence the audible portion of activated alarms for 60 seconds:

Press the **ALARM CONTROL**  key once. The **AUDIO PAUSED** symbol  displays on the screen.

To manually pause the alarm, press the **ALARM CONTROL**  key twice. The **ALARM PAUSED** symbol  displays on the screen. The audible portion, light indicator, and message are all halted.

WHAT TO DO IF A VENTILATOR FAILURE OCCURS

If a problem with the ventilator is suspected, **FIRST CHECK THAT THE PATIENT IS NOT IN DANGER**. If necessary, remove the patient from the ventilator and provide an alternate means of ventilation. Keep in mind that troubleshooting information is available in the manual to assist you in the event of a problem. Refer to the chapter, “Alarms and Troubleshooting”. If you cannot determine the cause of a problem, contact your equipment supplier or Covidien.

POWER SUPPLY OPTIONS

The “mains” AC electrical supply (115/230 V nominal) should be used as the primary source of power whenever available.

The ventilator is equipped with a 12-30 VDC power connector, for usage with an external battery or car adaptor cable. The ventilator should only be operated on the internal battery when no other power source is available.


INTERNAL BATTERY CAPACITY

With a fully charged battery at a normal room temperature of 25 °C (± 5 °C), the ventilator can be expected to operate on internal battery power for 4 to 11 hours.¹

1. On fully charged battery at a setting of PIP = 10 cmH₂O, V_T = 200 ml and rate = 20 bpm. Level adjustments, environmental conditions and physiological characteristics of the patient affect battery operating time.


FRONT

- 1 LCD Display
- 2 Control Panel
- 3 FiO₂ Sensor Connection
- 4 Patient Connection Port
- 5 Patient Pressure Monitoring Port
- 6 Exhalation Valve Port
- 7 Lateral and Front Openings
- 8 From Patient Port
- 9 Exhaled Gas Outlet

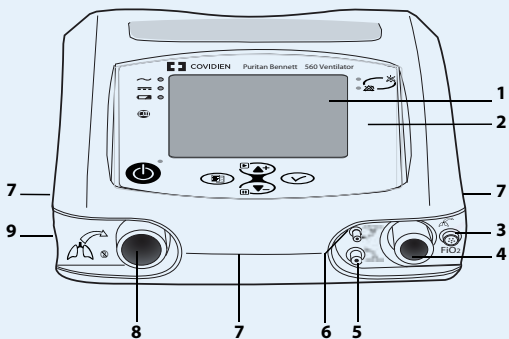


BACK

- 1 Ergonomic carrying handle
- 2 On/Off (I/O) switch with protective cover
- 3 AC power ("Mains") cable connector
- 4 AC power ("Mains") cable holding system
- 5 Access cover for the internal battery
- 6 DC power cable connector with key
- 7 PC Cable Connector
- 8 O₂ Inlet Port
- 9 Nurse Call Output Connector
- 10 USB Memory Device connection
- 11 Air Inlet Filter

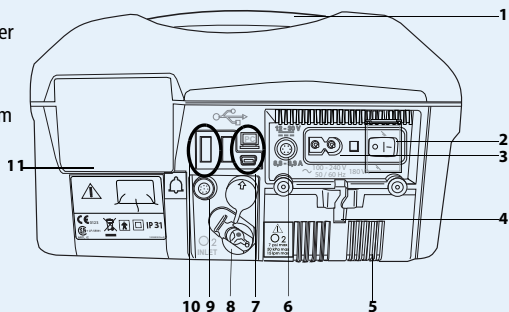


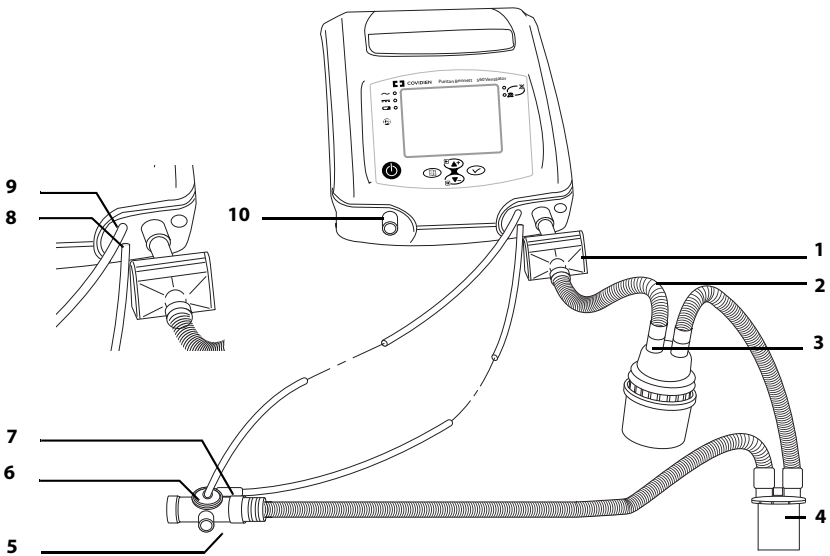
KEYBOARD

- 1 Alarm indicators
- 2 ALARM CONTROL key
- 3 Display screen
- 4 UP/UNFREEZE key
- 5 ENTER key
- 6 DOWN/FREEZE key
- 7 MENU key
- 8 VENTILATION ON/OFF button
- 9 Ventilation status indicator
- 10 Electrical power source indicators

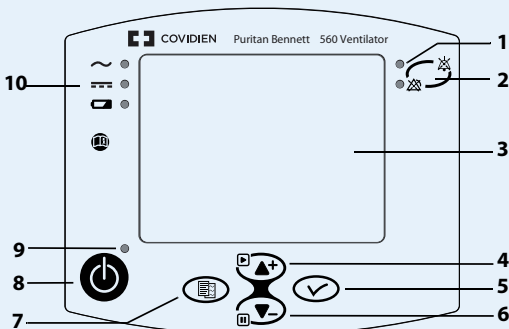

TO CONNECT A SINGLE LIMB CIRCUIT WITH AN EXHALATION VALVE



1. Inspect the components of the patient circuit for any signs of damage, such as cracks (which might cause leakage). Do not use damaged components to assemble the patient circuit.
2. Install the bacteria filter (item 1) on the **TO PATIENT**  outlet port, as shown.
3. Attach one end of the short circuit tubing (item 2) to the bacteria filter (item 1).
4. Attach the other end of the circuit tubing (item 2) to the inlet port of the humidifier (item 3).
5. Place a water trap (item 4) between the outlet port of the humidifier and the inlet of the exhalation valve (item 5).
6. Ensure the exhalation valve (item 5) as close as possible to the patient.
7. Connect one end of the proximal pressure tubing (item 7) to the proximal pressure port on the exhalation valve (item 5) and the other end onto the ventilator patient pressure port (item 8).
8. Connect one end of the exhalation valve tubing (item 6) to the exhalation valve port on the exhalation valve (item 5) and the other end onto the ventilator exhalation valve port (item 9).
9. To protect the exhalation port (as it will not be used in this configuration), place the cap (if provided with the breathing circuit) over the exhalation port opening (item 10).




TO CONNECT A SINGLE LIMB CIRCUIT WITHOUT AN EXHALATION VALVE

- 1-4. Follow steps 1-4 above.
5. Place a water trap (item 4) between the outlet port of the humidifier and the patient end.
6. Place a vented interface to the end of the patient circuit.
7. Connect one end of the proximal pressure tubing (item 7) to the proximal pressure port on a T connector, and the other end onto the ventilator patient pressure port

TO CONNECT A DOUBLE LIMB CIRCUIT:

1. Inspect the components of the patient circuit for any signs of damage, such as cracks (which might cause leakage). Do not use damaged components to assemble the patient circuit.
2. Install the bacteria filter (item 1) on the **TO PATIENT**  outlet port.
3. Attach one end of the short circuit tubing (item 4) to the filter (item 1).
4. Attach the other end of the circuit tubing to the inlet port of the humidifier (item 2).
5. Place a water trap (item 3) between the outlet port of the humidifier and the patient wye (item 5) on the double limb circuit.
6. Place a water trap (item 3) between the patient wye (item 5) and the inlet port of exhalation bacteria filter (item 12).
7. Connect the exhalation bacterial filter to the **FROM PATIENT**  inlet port.
8. Connect one end of the small proximal pressure tubing (item 6) to the double limb patient wye circuit connection (item 5) and the other end on the ventilator patient pressure port.
9. Place the exhalation valve assembly (item 9) on the exhaust port. Place a exhalation bacteria filter (item 14) between the exhalation block (item 10) and the exhalation limb of the patient circuit. (item 11). Connect the tubing (item 8) from the exhalation valve assembly to the exhalation valve port (item 15)

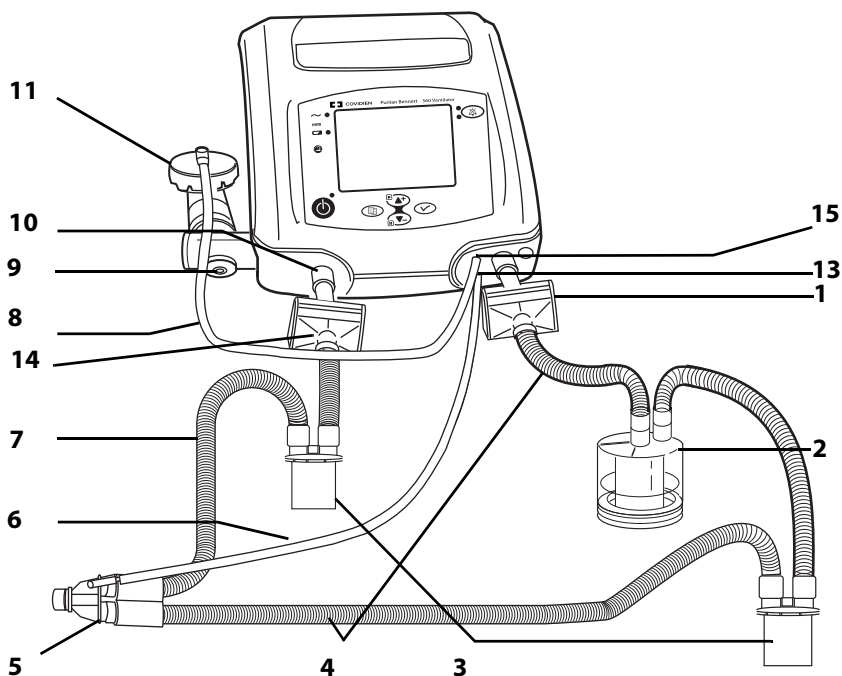

COVIDIEN TECHNICAL SERVICES

For additional ordering information, please contact your Covidien sales representative.

IMPORTANT : Please refer to the package insert for complete instructions, contraindications, warnings and precautions



COVIDIEN, COVIDIEN with logo, Covidien logo and "positive results for life" are U.S. and internationally registered trademarks of Covidien AG. Other brands are trademarks of a Covidien company.
© 2010 Covidien. - C-VE-PB560SetUpGuide/GB - 11/2011

COVIDIEN (UK)
COMMERCIAL LTD

4500 PARKWAY
WHITELEY
FAREHAM
HAMPSHIRE
PO15 7NY

+44 (0) 1329 224000 [T]
+44 (0) 1329 220213 [F]

COVIDIEN IRELAND
COMMERCIAL LTD

BLOCK G, GROUND FLOOR
CHERRYWOOD BUSINESS PARK,
LOUGHLINSTOWN,
CO. DUBLIN

+353 (0) 1.4073173 [T]
+353 (0) 1.4073174 [F]

WWW.COVIDIEN.COM