BIS™ Pediatric Sensor

This product cannot be adequately cleaned and/or sterilized by the user in order to facilitate safe reuse, and is therefore intended for single use. Attempts to clean or sterilize these devices may result in a bio-incompatibility, infection or product failure risks to the patient.

	Identification of a substance that is not contained or present within the product or packaging.		Do not use if package is opened or damaged
---	---	---	--

WARNING:
TO REDUCE THE HAZARD OF BURNS DURING USE OF BRAIN-STIMULATING DEVICES (e.g., TRANSCRANIAL ELECTRICAL MOTOR EVOKED POTENTIAL), PLACE STIMULATING ELECTRODES AS FAR AS POSSIBLE FROM THE BIS SENSOR AND MAKE CERTAIN THAT SENSOR IS PLACED ACCORDING TO PACKAGE INSTRUCTIONS.

CAUTIONS:

Do not use if sensor is dry. To avoid dry out, do not open pack until ready for use. Due to intimate skin contact, reuse may pose risk of infection. If skin rash or other unusual symptom develops, stop use and remove. Limited to short-term use (maximum of 24 hours). Do not cut sensor components, as this can result in improper operation.

NOTE: Upon removal, slight redness of skin may be seen and typically resolves within a short period of time.

INSTRUCTIONS:

- Wipe skin with alcohol and dry
- Position sensor diagonally on the forehead as follows:
 - 1 at center of forehead, approximately 1.5 inches (4cm) above the bridge of nose
 - 4 directly above eyebrow
 - 3 on temple, between corner of eye and hairline
- Press edges of sensor to assure adhesion
- Press 1, 2, 3 and 4 firmly for 5 seconds
- Insert sensor tab into patient interface cable
 - To minimize twisting of the sensor, where it connects to the cable, consider securing with tape and/or relieving cable strain.

	Identification of a substance that is not contained or present within the product or packaging.		Do not use if package is opened or damaged
---	---	---	--

it	fr	de	es	ro	ru	uk	pl	pt	sv	nl	fi	da	no	cs	sk	hu
-----------	-----------	-----------	-----------	-----------	-----------	-----------	-----------	-----------	-----------	-----------	-----------	-----------	-----------	-----------	-----------	-----------

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© 2011 Covidien. Covidien Inc, 15 Hampshire Street, Mansfield, MA 02048 USA. COVIDIEN Covidien Ireland Limited, IDA Business & Technology Park, Tullamore.
COVIDIEN, COVIDIEN with logo and Covidien logo are U.S. and/or internationally registered trademarks of Covidien AG. Other brands are trademarks of a Covidien company.