



# Medtronic

## Connecting the Medtronic CareLink® Network to Paceart®

### A SEAMLESS FLOW OF INFORMATION

Superb on their own. Better together. Paceart and the Medtronic CareLink Network each offer a unique way to provide optimal follow-up for arrhythmia patients with implanted cardiac devices. Connecting these technologies ensures a seamless flow of data from device to clinic and beyond.

### What does this mean for you?

Integrating the Medtronic CareLink Network with Paceart streamlines clinic workflow, improves efficiency, and enhances data quality.

### Automatic data transfer

Medtronic CareLink transmissions for ICDs, CRT-Ds, and pacemakers are automatically downloaded into the Paceart System nightly. Clinicians can also transfer single- or multiple-patient device data on demand for instant access. Automatic downloads may help eliminate errors related to manual data entry.

### Documentation and analysis for insightful care

Paceart serves as a central hub for organizing and storing patient and device data - putting baseline ECGs, past thresholds, and trend analysis at your fingertips. Clinicians can review Medtronic CareLink transmissions along with data from programmers for more than 1,000 devices from all major manufacturers, as well as transtelephonic monitoring systems such as cardiac event recorders and pacemaker transmitters. Paceart also provides professional reporting capabilities to document both remote transmissions and in-clinic follow-up sessions.

### Easy remote transmission scheduling

Use Paceart to schedule Medtronic CareLink transmissions, manage missed transmissions, and generate letters to patients.

### Comprehensive correspondence

Paceart creates letters to other physicians involved in the care of Medtronic CareLink patients, easily incorporating remote device interrogation data. Paceart also can produce patient scheduling letters and appointment reminders.


### Scheduling Medtronic CareLink patients


Paceart's drop-down appointment menu allows you to schedule remote device follow-ups for Medtronic CareLink patients and manage missed transmissions.



"A great challenge we have in medicine is that information comes to us from many different areas, such as Medtronic CareLink transmissions and in-office patient evaluations. Paceart allows us to integrate that information so that I always know where to look for it. Paceart is a timesaver, an incredible data management tool, and a valuable patient management tool."

- Charles Love, MD, Director  
Arrhythmia Device Services  
The Ohio State University Medical Center  
Columbus, Ohio

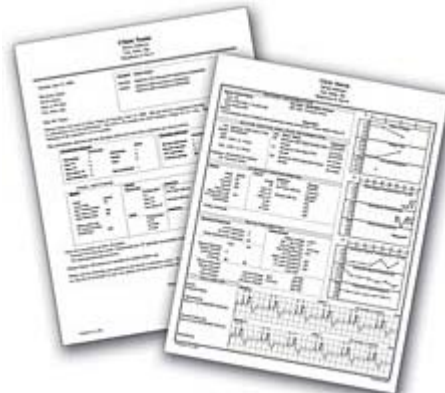

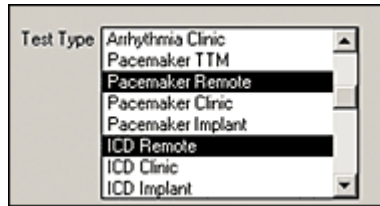

### Requirements

To connect the Medtronic CareLink Network to Paceart, clinics should have:

- Paceart System 2006, First Edition or later. (Available to Paceart customers on version 3.0 or higher with a current service contract.)
- Paceart programmer data-import capabilities.
- An active account with the Medtronic CareLink Network.
- Internet access from a server-connected workstation (needed to retrieve remotely acquired data from the Medtronic CareLink Clinician Website).

**To learn more about how your clinic can benefit from Paceart's integration with the Medtronic CareLink Network, contact your Medtronic Paceart representative or call 1 (800) Paceart.**

### Brief Statement

For further information regarding Paceart, please call Medtronic at 1 (800) 722-3278 and/or consult Medtronic's website at [www.Paceart.com](http://www.Paceart.com).

### Medtronic CareLink® Monitor/Medtronic CareLink® Network

#### Intended Use

The Medtronic CareLink Monitor and the Medtronic CareLink Network are indicated for use in the transfer of patient data from some Medtronic implantable cardiac devices based on physician instructions and as described in the product manual. These products are not a substitute for appropriate medical attention in the event of an emergency and should only be used as directed by a physician.

#### Contraindications

There are no contraindications for the Medtronic CareLink Monitor.

#### Warnings and Precautions

The Medtronic CareLink Monitor must only be used for interrogating compatible Medtronic implantable devices. The Medtronic CareLink Monitor is intended for use within the prescribing country.

See the device manual for detailed information regarding the instructions for use, indications, contraindications, warnings, precautions, and potential complications/adverse events. For further information, please call Medtronic at 1 (800) 328-2518 and/or consult Medtronic's website at [www.medtronic.com](http://www.medtronic.com).

**Caution:** Federal law (USA) restricts this device to sale by or on the order of a physician.