

Endurant™ II/IIs

AAA Stent Graft System

FOR USE BY PHYSICIAN ONLY

Date of CT Study: _____

Patient ID: _____

Patient DOB: / /


CT Slice Thickness: _____

Implanting Physician: _____

Hospital Name: _____

Evaluation Date: _____

Procedure Date: _____

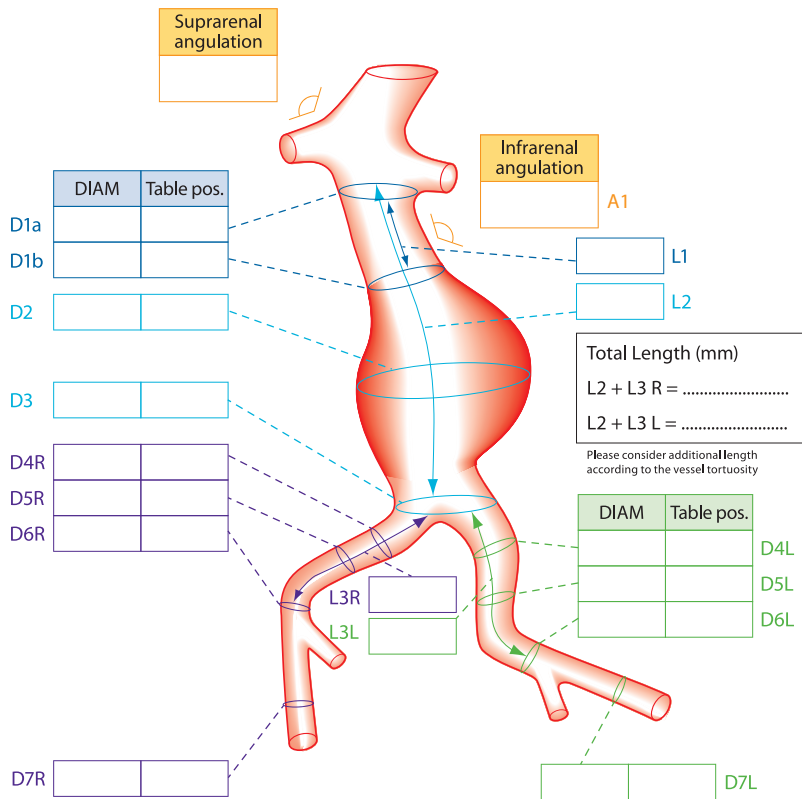

SMA patent?

Yes No

Lowest renal artery

Right Left

Disease Progression Risk

Proximal neck:

- Short (L1)
- Wide (D1a)
- Angled (A1)
- Conical (% change D1/L1)

Proximal neck thrombus

- Nil Mild
- Moderate Severe

Proximal neck calcification

- Nil Mild
- Moderate Severe

Consider EndoAnchors?

- Yes No

Lumbar patent?

- Yes No

IMA patent?

- Yes No

Right iliac calcification

- Nil Mild
- Moderate Severe

Left iliac calcification

- Nil Mild
- Moderate Severe

Coil hypo

- Right Left
- No

Proposed bifur side

- Right Left

Please reference appropriate product *Instructions for Use* for a more detailed list of indications, warnings, precautions and potential adverse events and sizing guidelines.

Drawing

Comments

QTY	Product Code	QTY	Product Code

Physician Signature: _____

Endurant™ II/IIs

AAA Stent Graft System

PRODUCT CODES

ENDURANT II BIFURCATIONS

Product Code						
	Proximal Graft Diameter (mm)	Distal Graft Diameter (mm)	Distal Design	Total Covered Length (mm)	Delivery System	Catheter Outer Diameter (F)
ETBF	23	13	C	124	EE	18
ETBF	23	13	C	145	EE	18
ETBF	23	13	C	166	EE	18
ETBF	23	16	C	124	EE	18
ETBF	23	16	C	145	EE	18
ETBF	23	16	C	166	EE	18
ETBF	25	13	C	124	EE	18
ETBF	25	13	C	145	EE	18
ETBF	25	13	C	166	EE	18
ETBF	25	16	C	124	EE	18
ETBF	25	16	C	145	EE	18
ETBF	25	16	C	166	EE	18
ETBF	28	13	C	124	EE	18
ETBF	28	13	C	145	EE	18
ETBF	28	13	C	166	EE	18
ETBF	28	16	C	124	EE	18
ETBF	28	16	C	145	EE	18
ETBF	28	16	C	166	EE	18
ETBF	28	20	C	124	EE	18
ETBF	28	20	C	145	EE	18
ETBF	28	20	C	166	EE	18
ETBF	32	16	C	124	EE	20
ETBF	32	16	C	145	EE	20
ETBF	32	16	C	166	EE	20
ETBF	32	20	C	124	EE	20
ETBF	32	20	C	145	EE	20
ETBF	32	20	C	166	EE	20
ETBF	36	16	C	145	EE	20
ETBF	36	16	C	166	EE	20
ETBF	36	20	C	145	EE	20
ETBF	36	20	C	166	EE	20

ENDURANT IIs BIFURCATIONS

Product Code						
	Proximal Graft Diameter (mm)	Distal Graft Diameter (mm)	Distal Design	Total Covered Length (mm)	Delivery System	Catheter Outer Diameter (F)
ESBF	23	14	C	103	EE	18
ESBF	25	14	C	103	EE	18
ESBF	28	14	C	103	EE	18
ESBF	32	14	C	103	EE	20
ESBF	36	14	C	103	EE	20

LIMBS*

Product Code								
	Proximal Graft Diameter (mm)	Distal Graft Diameter (mm)	Distal Design	Total Covered Length (mm)	Delivery System	Catheter Outer Diameter (F)	Total Contralateral Covered Length with Ell/EIIs Bifurcated [†]	Total Ipsilateral Covered Length with Ells Bifurcated [†]
ETLW	16	10	C	82	EE	14	136	155
ETLW	16	10	C	93	EE	14	147	166
ETLW	16	10	C	124	EE	14	178	177-197
ETLW	16	10	C	156	EE	16	210	209-229
ETLW	16	10	C	199	EE	16	253	252-272
ETLW	16	13	C	82	EE	14	136	155
ETLW	16	13	C	93	EE	14	147	166
ETLW	16	13	C	124	EE	14	178	177-197
ETLW	16	13	C	156	EE	16	210	209-229
ETLW	16	13	C	199	EE	16	253	252-272
ETLW	16	16	C	82	EE	14	136	135-155
ETLW	16	16	C	93	EE	14	147	146-166
ETLW	16	16	C	124	EE	14	178	177-197
ETLW	16	16	C	156	EE	16	210	209-229
ETLW	16	16	C	199	EE	16	253	252-272
ETLW	16	20	C	82	EE	16	136	155
ETLW	16	20	C	93	EE	16	147	166
ETLW	16	20	C	124	EE	16	178	177-197
ETLW	16	20	C	156	EE	16	210	209-229
ETLW	16	20	C	199	EE	16	253	252-272
ETLW	16	24	C	82	EE	16	136	155
ETLW	16	24	C	93	EE	16	147	166
ETLW	16	24	C	124	EE	16	178	177-197
ETLW	16	24	C	156	EE	16	210	209-229
ETLW	16	24	C	199	EE	16	253	252-272
ETLW	16	28	C	82	EE	16	136	155
ETLW	16	28	C	93	EE	16	147	166
ETLW	16	28	C	124	EE	16	178	177-197
ETLW	16	28	C	156	EE	16	210	209-229
ETLW	16	28	C	199	EE	16	253	252-272

*The limb mates with the AUI stent graft on the ipsilateral side.

[†]These calculations assume the minimum 30 mm overlap between the bifurcated stent graft and the contralateral iliac limb per the Endurant II Stent Graft System Instructions For Use. When using the 124 mm length bifurcated stent graft, subtract 10 mm from Total Contralateral Covered Length with Bifurcated.

[†]The 3-5 stent overlap is available only with select limbs. Please refer to the Instructions For Use for more information.

AORTIC EXTENSIONS

Product Code						
	Proximal Graft Diameter (mm)	Distal Graft Diameter (mm)	Distal Design	Total Covered Length (mm)	Delivery System	Catheter Outer Diameter (F)
ETCF	23	23	C	49	EE	18
ETCF	25	25	C	49	EE	18
ETCF	28	28	C	49	EE	18
ETCF	32	32	C	49	EE	20
ETCF	36	36	C	49	EE	20

ABDOMINAL TUBES

Product Code						
	Proximal Graft Diameter (mm)	Distal Graft Diameter (mm)	Distal Design	Total Covered Length (mm)	Delivery System	Catheter Outer Diameter (F)
ETTF	23	23	C	70	EE	18
ETTF	25	25	C	70	EE	18
ETTF	28	28	C	70	EE	18
ETTF	32	32	C	70	EE	20
ETTF	36	36	C	70	EE	20

AUI

Product Code						
	Proximal Graft Diameter (mm)	Distal Graft Diameter (mm)	Distal Design	Total Covered Length (mm)	Delivery System	Catheter Outer Diameter (F)
ETUF	23	14	C	102	EE	18
ETUF	25	14	C	102	EE	18
ETUF	28	14	C	102	EE	18
ETUF	32	14	C	102	EE	20
ETUF	36	14	C	102	EE	20

ILIAC EXTENSIONS

Product Code						
	Proximal Graft Diameter (mm)	Distal Graft Diameter (mm)	Distal Design	Total Covered Length (mm)	Delivery System	Catheter Outer Diameter (F)
ETEW	10	10	C	82	EE	14
ETEW	13	13	C	82	EE	14
ETEW	20	20	C	82	EE	16
ETEW	24	24	C	82	EE	16
ETEW	28	28	C	82	EE	18

Medtronic

Invatec S.p.A.

Via Martiri della Libertà 7 25030
Roncadelle (BS)
Italy
Tel: +39.030.2589311

Medtronic

710 Medtronic Parkway NE
Minneapolis, MN 55432
USA
Tel: +1.763.514.4000

**Medtronic International Trading
Sàrl**

Route du Molliau 31
CH-1131 Tolochenaz
Switzerland
Tel: +41.21.802.7000

Medtronic of Canada Ltd.

99 Hereford Street
Brampton, Ontario L6Y 0R3
Canada
Tel: +1.905.460.3800

Medtronic Latin America

3750 NW 87 Avenue, Suite 700
Miami, FL 33178
USA
Tel: +1.305.500.9328

Medtronic International Ltd.

49 Changi South
Avenue 2
Singapore 486056
Tel: +65.6436.5000

Medtronic Australasia Pty Ltd.

97 Waterloo Road
North Ryde, NSW 2113
Australia
Tel: +61.29857.9000

Medtronic New Zealand

Unit N16, Mezzanine Level 5
Gloucester Park Road
Onehunga
Auckland
New Zealand

Medtronic Korea Co., Ltd.

5F, Sajo Building
1001 Daechi-dong, Kangnam-ku
Seoul, 135-280
Korea
Tel: +82.2.3404.3600

Medtronic META FZ-LLC

Office Park, Block D, 2nd Floor
PO Box 500638
Dubai Internet City | Dubai
United Arab Emirates
Tel: +971.4.818.2666

medtronic.com/aortic