

Medtronic

DBS therapy patient programmer

**Model TH91D for Percept™
neurostimulators**

Quick guide



DBS therapy patient programmer Model TH91 for Percept™ neurostimulators

Quick guide

To view your therapy




Samsung Handset
Power Button



TM91 Communicator
Power Button

- 1 Turn on the handset and communicator. Check that they are charged.

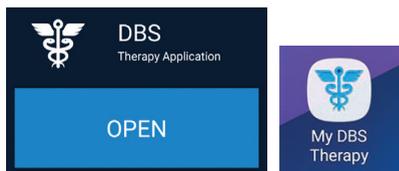

- 2 If the app is not already open, tap the DBS Therapy Application **OPEN** button or the **My DBS Therapy** app icon on the handset.




- 3 Ensure the communicator is within 1 meter of your neurostimulator and tap the **CONNECT** button on the handset.

4

When the DBS Therapy app is open and connected to the neurostimulator, the HOME screen will be visible.[†]

[†]Depending on your settings, your HOME screen may not look exactly like this.

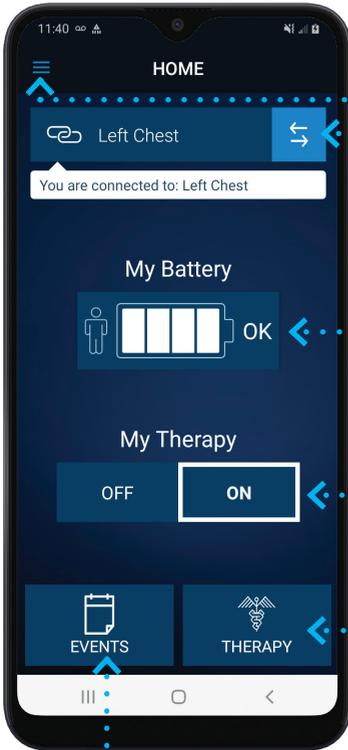

Menu button

Tap to access additional app features.

The neurostimulator

Tap to switch neurostimulators (only shows if you have two neurostimulators).

My Battery

Tap to view the status of all batteries.

My Therapy ON/OFF

Tap to turn your therapy on and off.

Effects of turning therapy off

Warning: Avoid turning your therapy off without first talking with your clinician, because your symptoms might return. For some, this can cause a serious medical emergency. Refer to your therapy-specific patient booklet for more information.

THERAPY button

Tap to view or adjust your therapy (only shows if configured by your clinician).

EVENTS button

To record events (only shows if configured by your clinician).



For video tutorials, tap the menu button on your patient programmer to view.

Tap the tutorial button to view videos about how to use your programmer.

Turn Therapy Off/On

Note: You may need your communicator held over your neurostimulator to turn stimulation off.


- 1 To Turn Therapy Off: On the **HOME** screen, tap the **OFF** button. Tap to confirm you want to turn off therapy in the pop-up message. **Fig. A**

Effects of turning therapy off

Warning: Avoid turning your therapy off without first talking with your clinician, because your symptoms might return. For some, this can cause a serious medical emergency. Refer to your therapy-specific patient booklet for more information.



A

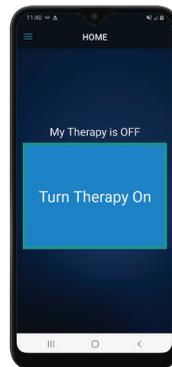

B

- 2 To Turn Therapy On: On the **HOME** screen, tap the **Turn Therapy On** button. **Fig. B**

How to check your battery status

- 1 On the **HOME** screen, tap the **My Battery** icon.
- 2 The **BATTERY** screen **Fig. C** shows the status of three different batteries[†]:
 - My Battery - the battery level of the neurostimulator, the device implanted in your body. If you have two neurostimulators implanted, you can view the battery level of the other device by switching to the other device.
 - Communicator Battery - the battery level of the communicator, the device that communicates with your neurostimulator.
 - Handset Battery - the battery level of the handset, the device that runs the My DBS Therapy Application.

[†]Depending on your settings and your neurostimulator, your BATTERY screen may not look exactly like this.



C

Note: You can also tap the Menu button (≡) and tap **Battery**.

Changing groups

Note: Your clinician may have set up groups, which are preset therapy options that you can choose from. If available, changing groups is a way for you to quickly adjust your therapy.

- 1 On the **HOME** screen, tap **THERAPY**.
- 2 Tap the **GROUP** button (☑) on the **THERAPY** screen. **Fig. D**[†]
- 3 Tap the group that you want to change to **Fig. E**[†]

After choosing a group, the **THERAPY** screen for that group will appear. This means you have successfully changed groups.

[†]Depending on your settings, your screen may not look exactly like this.



D

E

Adjusting stimulation

Note: If it is available to you, you can raise and lower the level of your therapy. You may not be able to adjust therapy. Your DBS clinician will decide whether to provide you with this option.

- 1 On the **HOME** screen, tap **THERAPY**.
- 2 Adjust your therapy up and down using the arrow buttons. **Fig. D**
 - If an arrow turns gray, you cannot adjust therapy any further in that direction.
 - Between the arrow buttons, Left is the left side of your body and Right is the right side.
 - To revert to clinician defined settings for the group, hit revert button. **Fig. D**

Events

Note: Your clinician may set up events for you to record in the app (e.g. Took Medication). When you record an event, the app will automatically record other information that may be helpful to your DBS clinician, such as your current therapy settings. Your clinician may also program event recording to adjust the stimulation cycle.

To record an event:

- 1 On the **HOME** screen, tap the **EVENTS** button **Fig. F** (see next page)
- 2 On the **Events** screen, tap an event button that you want to record (e.g., Took Medication), **Fig. G** (see next page) then tap **OK** on the **EVENT RECORDED** confirmation screen.

Note: The events set up by your DBS clinician may differ from those in **Fig. G**.

- If your DBS clinician set up more than four events, you can see additional events by tapping the **Forward** or **Back** buttons
- If you see an **EVENT NOT RECORDED** alert, this indicates your DBS clinician removed the selected event from the app.

MRI workflow and MRI mode

Refer to your patient therapy guide and therapy-specific patient booklet for warnings, cautions, and important safety information about MRI scans.


Note:

- Bring your handset, communicator and patient ID card to the MRI appointment. **Do not take the handset or communicator into the MRI (magnet) room.**
- Your DBS clinician may activate MRI Mode for you, prior to your MRI scan appointment.
- MRI Mode should only be used for an MRI scan.

- 1 On the **HOME** screen, tap the menu button (**≡**) **Fig. H** and then tap **MRI Mode** on the menu.
- 2 If you don't have an **MRI Eligibility Report**, ask the MRI clinician if they received a report. Your DBS clinician may have already completed an **MRI Eligibility Report** and sent it directly to the MRI facility. If you or the MRI clinician have a report, tap **YES** on the **MRI Report** screen **Fig. I**. Otherwise, tap **NO** on the **MRI Report** screen.
 - a. If you tapped **YES** on the **MRI Report** screen, tap the scan eligibility type selected on your report (Full Body or Head Only), then tap **CONTINUE** on the **Scan Eligibility** screen **Fig. J** and proceed to step 3.
 - b. If you tapped **NO** on the **MRI Report** screen, the app will prompt you to test your system to determine your MRI eligibility. Tap **START TEST** on the **Test System** screen to continue with the test. To cancel and return to the previous screen, tap **BACK**. **Fig. K**




F

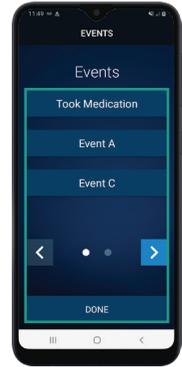

G




H

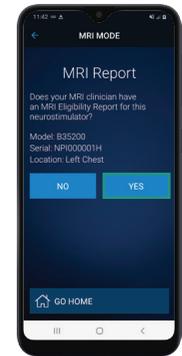

I




J

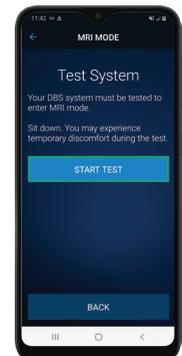

K

Note:

- Sit down. You may experience temporary discomfort during the test because the settings used may differ from your normal therapy settings. Once the test starts, you can stop the test at any time by tapping **STOP TEST** on the **Testing System** screen. **Fig. L**
- Upon completion of the test, if you see the **MRI Mode Not Available** screen, contact your DBS clinician. This screen indicates your DBS system cannot enter **MRI Mode** as currently configured. Your DBS clinician may be able to make adjustments to allow **MRI Mode**.
- If you have multiple neurostimulators and cannot get both neurostimulators prepared for an MRI, exit MRI Mode on the neurostimulator that is in MRI Mode before contacting your DBS clinician. Otherwise, proceed to step 3.

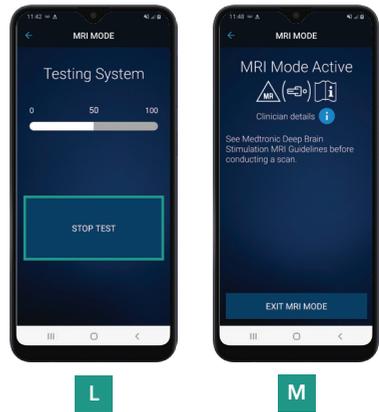

3 You will be asked to choose a therapy setting that is compatible with an MRI. This will always include **Therapy Off** but may also include an option for a therapy group if your clinician configured one that is MRI compatible. **Effects of turning therapy off. Warning:** Avoid turning your therapy off without first talking with your clinician, because your symptoms might return. For some, this can cause a serious medical emergency. Refer to your therapy-specific patient booklet for more information.

4 If this app only controls one neurostimulator, you will see the **MRI Mode Active** screen **Fig. M**, confirming your neurostimulator is in MRI Mode. Depending on your settings, your **MRI Mode Active** screen may not look exactly like **Fig. M**. This DBS system is now ready for an MRI scan.

5 When you are finished with the MRI scan, tap the Exit MRI Mode button go back to Home screen. Your neurostimulator will return to its original setting. **Fig. M**

Note:

- If you have two neurostimulators, tap the **CONTINUE** button to check whether the second neurostimulator is out of MRI Mode. If it is still in MRI Mode, take it out of MRI Mode. Ensure that your therapy returns to the original settings
- See your clinician if you have questions about your therapy after your MRI scan.



This material does not replace the Instructions for Use. Please refer to the product insert or Instructions for Use for a complete list of indications, contraindications, warnings, precautions, and other important medical information. Always follow the Instructions for Use. For further information, contact your local Medtronic representative and/or consult the Medtronic website at [medtronic.com.au](https://www.medtronic.com.au).

Medtronic

Medtronic Australasia Pty Ltd
2 Alma Road
Macquarie Park NSW 2113
Australia
Tel: +61 2 9857 9000
Fax: +61 2 9889 5167
Toll Free: 1800 668 670

Medtronic New Zealand Ltd
Level 3 - Building 5, Central Park
Corporate Centre
666 Great South Road
Penrose, Auckland 1051
New Zealand
Toll Free: 0800 377 807

[medtronic.com.au](https://www.medtronic.com.au)

©2024 Medtronic. All Rights Reserved.
13139493 | AU-2400004