

# Getting started with the MiniMed™ Mobile app

Compatible phone? Check your device at [www.medtronicdiabetes.ca/compatibility](http://www.medtronicdiabetes.ca/compatibility)



1. Open the app and follow the on-screen instructions.
2. Log into your CareLink™ Personal account. [Sign up](http://www.carelink.minimed.eu) if you do not already have an account. [www.carelink.minimed.eu](http://www.carelink.minimed.eu)
3. Follow the on-screen instructions to [pair the app](#) with your MiniMed™ 700 series pump.  
  
Remember to watch your phone for pairing requests.
4. Read the Warning screen and accept by tapping [OK](#) and then [Agree](#).




Automatic updates to mobile operating systems should be **deactivated** - if one becomes available, check your local website to see if it is supported before installing.

## Navigating the app\*

Tap the three bars to access the menu.

Current sensor reading.

Insulin delivery information.

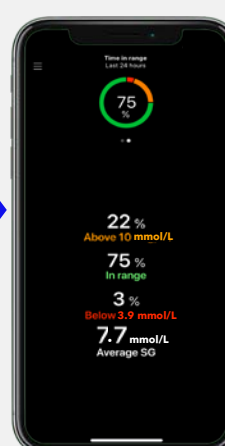


Sensor tracing. Events on the graph can be tapped on for details.

Swipe left to access Time in Range.

**Sync to CareLink™:** Allows you to manage data uploads to CareLink™ Personal. Switch on for automatic uploading or tap [Upload now](#) to upload all data in your MiniMed™ pump. You can also manage your care partners in this menu.

\*Images shown when paired with a MiniMed™ 780G system, representative patient profile.

# Notifications

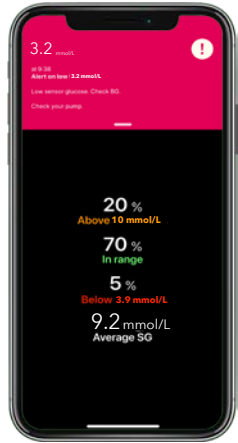


Notifications are colour coordinated:  
**Alarms** are red  
**Alerts** are yellow  
 Reminders are grey  
**Messages** are blue



## Important Information:

- If a care partner requests to follow you, go to the menu, select **Sync to CareLink™** and then **Manage Care Partners** to accept.
- For data to be sent to linked care partners, the device with the MiniMed™ Mobile app must be:
  - Within 6 meters (20 feet) of the pump
  - Have an internet connection
  - Be logged into CareLink™ Personal
  - Have the **Sync to CareLink™** turned on in the menu

Sync to CareLink™


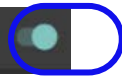


# Apple® Watch

MiniMed™ Mobile app is also compatible with Apple® Watch\*. See your values on your watch face or tap to enter the app.



The phone app also has a menu option for the watch app. It can, for example, tell you if your watch or pump is not compatible or if the app on the watch is missing.\*\*

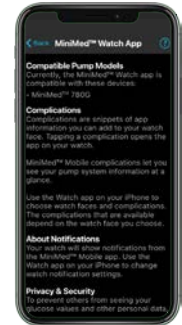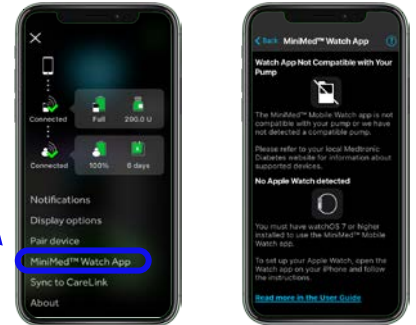

\* Check your local website for full Apple® Watch and MiniMed™ pump supported devices  
 \*\*Default setting on phone has been set to not download companion apps

**Note:** Notifications must be cleared on your mobile device/watch **and** pump.