
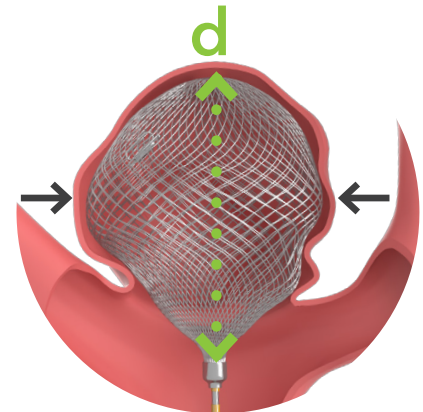
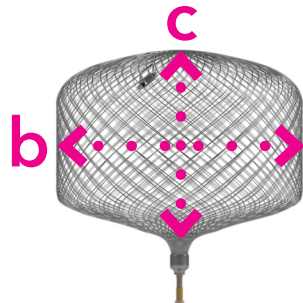


## Artisse™ Intrasaccular Device

# Sizing guide

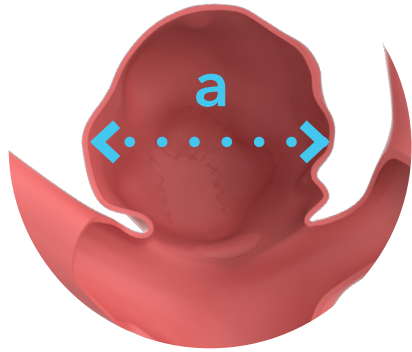

Maximum average aneurysm width (mm) <b>[a]</b>	Artisse™ device width and height (mm) <b>[b x c]</b>	Compressed Device height when sized at max. avg. aneurysm width (mm) <b>[d]</b>
3.4	4.5 x 3.0	4.5
3.4	4.5 x 4.0	5.2
3.8	5.0 x 3.0	4.9
3.8	5.0 x 4.0	5.5
4.1	5.5 x 3.0	5.2
4.1	5.5 x 4.0	5.9
4.1	5.5 x 5.0	6.9
4.5	6.0 x 3.0	5.6
4.5	6.0 x 4.0	6.2
4.5	6.0 x 5.0	7.2
4.9	6.5 x 3.0	5.9
4.9	6.5 x 4.0	6.6
4.9	6.5 x 5.0	7.6
5.3	7.0 x 3.0	6.3
5.3	7.0 x 4.0	7.0
5.3	7.0 x 5.0	7.9
5.6	7.5 x 4.0	7.4
5.6	7.5 x 5.0	8.3
6.0	8.0 x 4.0	7.8
6.0	8.0 x 5.0	8.7

**Note:** If the measured average aneurysm width is between widths [a], target a larger group of devices while accounting for aneurysm height. Aneurysm height should be less than the compressed device height [d] for proper neck coverage.

This material should not be considered the exclusive source of information, it does not replace or supersede information contained in the device manual(s). For a listing of indications, contraindications, precautions, warnings, and potential adverse events, please refer to the Instructions for Use.

©2026 Medtronic. All rights reserved. Medtronic, Medtronic logo and Engineering the extraordinary are trademarks of Medtronic. ™\*Third party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company. CA-CTL-0677-E Rev. 03/2026

Estimated width oversizing factor

