

# Medtronic

## Accuracy matters

# Introducing Interactive ECG

### Available for LINQ family of ICMs on the CareLink™ network

Navigate to Interactive ECG for LINQ™ II or Reveal LINQ™ devices by opening the **Quick Look** tab and clicking on the **Interactive ECG** button.

Interactive ECG



#### A High-resolution ECG waveforms

Improved charting technology displays ECG using vector graphics, allowing for high-quality and high-resolution ECG waveforms.

#### B Sweep speed control

Sweep speed control allows for different horizontal zooms of the ECG.

#### C Navigation control and multiple views

Navigator control bar below the ECG provides a view of the entire HR plot and allows for quick navigation to the relevant portion of the ECG. Multiple views (focused, HR plot, presenting, full) provides the users the ability to choose the best view for their needs.



#### D Amplitude zoom improved

Amplitude zoom works on data immediately without needing to refresh the page.

Click the + or - buttons to zoom in and out.

#### E Adjustable calipers and annotations

Interactive interface provides tools such as adjustable calipers and annotations.

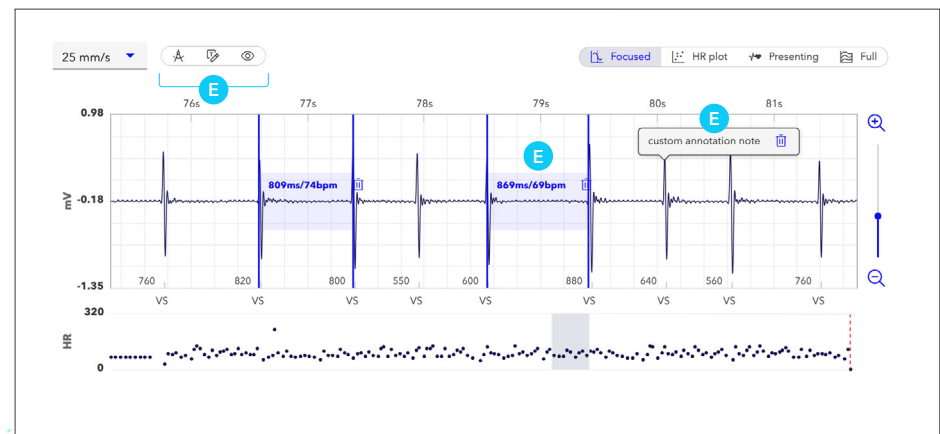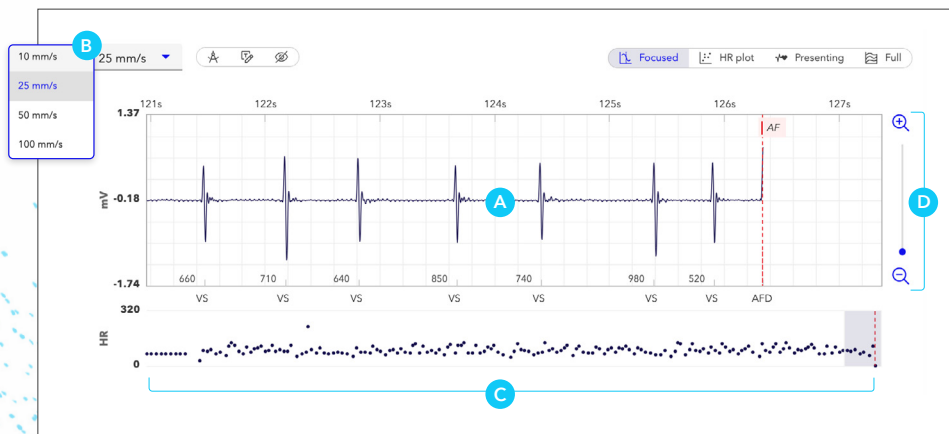

**Note:** The annotations and caliper measurements used during a CareLink session do not export to reports.

## Indications, Safety, and Warnings

If you are located in the United States, please refer to the brief statement(s) below to review applicable indications, safety, and warning information. See the device manual for information regarding the instructions for use, indications, contraindications, warnings, precautions, and potential complications/adverse events. For further information, please call Medtronic at 1.763.514.4000 and/or consult the Medtronic website at [www.medtronic.com](http://www.medtronic.com).

If you are located outside the United States, see the device manual for information regarding instructions for use, indications, contraindications, warnings, precautions, and potential adverse events. If using an MRI SureScan™ device, see the MRI SureScan™ technical manual before performing an MRI. For further information, contact your local Medtronic representative and/or consult the Medtronic website at [www.medtronic.eu](http://www.medtronic.eu).



[medtronic.com/manuals](http://medtronic.com/manuals)

For applicable products, consult instructions for use on [manuals.medtronic.com](http://manuals.medtronic.com). Manuals can be viewed using a current version of any major internet browser. For best results, use Adobe Acrobat® Reader with the browser.

## Brief Statement for LINQ Family of Insertable Cardiac Monitors (ICMs) Systems and Accessories

### Indications

The Reveal LINQ™ ICM is an insertable automatically-activated and patient-activated monitoring system that records subcutaneous ECG and is indicated in the following cases:

- patients with clinical syndromes or situations at increased risk of cardiac arrhythmias
- patients who experience transient symptoms such as dizziness, palpitation, syncope, and chest pain that may suggest a cardiac arrhythmia

This device has not been tested specifically for pediatric use.

The LINQ II™ ICM is an insertable automatically-activated and patient-activated monitoring system that records subcutaneous ECG and is indicated in adult patients, and in pediatric patients who are at least 2 years old, in the following cases:

- patients with clinical syndromes or situations at increased risk of cardiac arrhythmias
- patients who experience transient symptoms such as dizziness, palpitation, syncope, and chest pain that may suggest a cardiac arrhythmia

## Contraindications

There are no known contraindications for the insertion of a LINQ Family ICM or their accessories. However, the patient's particular medical condition may dictate whether or not a subcutaneous, chronically inserted device can be tolerated.

## Warnings and Precautions

Patients with a LINQ Family ICM should avoid sources of diathermy, high sources of radiation, electrosurgical cautery, external defibrillation, lithotripsy, therapeutic ultrasound and radiofrequency ablation to avoid electrical reset of the device, and/or inappropriate sensing as described in the Medical procedure and EMI Warnings, Precautions and Guidance Manual. MRI scans should be performed only in a specified MR environment under specified conditions as described in the Reveal LINQ ICM or LINQ II ICM MRI Technical Manual.

Accessories available for use with the LINQ Family of ICMs may experience connectivity or performance issues. See product manuals for details and troubleshooting instructions.

## Potential Adverse Events or Potential Complications

Potential adverse events from the LINQ Family of ICMs include, but are not limited to, device rejection phenomena (including local tissue reaction), device migration, infection, and erosion through the skin.

There are no known adverse events associated with the use of any LINQ Family of ICMs accessories.

See the device manuals for detailed information regarding the implant procedure, indications / intended use, contraindications, warnings, precautions, and potential complications/adverse events. For further information, please call Medtronic at (800) 328-2518 (Technical Services), (800) 551-5544 (Patient Services), and/or consult Medtronic's website at [www.medtronic.com](http://www.medtronic.com).

**Caution:** Federal law (USA) restricts prescription devices to sale by or on the order of a physician.

# Medtronic

710 Medtronic Parkway  
Minneapolis, MN 55432-5604  
USA

Toll-free in USA: 800.633.8766  
Worldwide: +1.763.514.4000

[medtronic.com](http://medtronic.com)

## Europe

Medtronic International Trading  
Sàrl.  
Route du Molliau 31  
Case postale  
CH-1131 Tolochenaz  
[medtronic.eu](http://medtronic.eu)  
Tel: +41 0 21 802 70 00  
Fax: +41 0 21 802 79 00

## United Kingdom/Ireland

Medtronic Limited  
Building 9  
Croxley Green Business Park  
Hatters Lane  
Watford  
Herts WD18 8WW  
[medtronic.co.uk](http://medtronic.co.uk)  
Tel: +44 0 1923 212213  
Fax: +44 0 1923 241004

©2025 Medtronic. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

UC202504638 IE 01/2025