

**Medtronic**

Made for  
**more.**

**Concerto Versa™ Detachable Coil**



# More compatible

Built with unparalleled catheter compatibility, Concerto Versa coil lets you address complex anatomies without sacrificing speed or precision.<sup>†</sup>

34%

shorter  
catheterization  
time<sup>†,1</sup>


35%

less  
radiation  
exposure<sup>†,1</sup>

## Dual catheter compatibility

Each coil has the flexibility to be used with diagnostic catheter or microcatheter.

4F Diagnostic

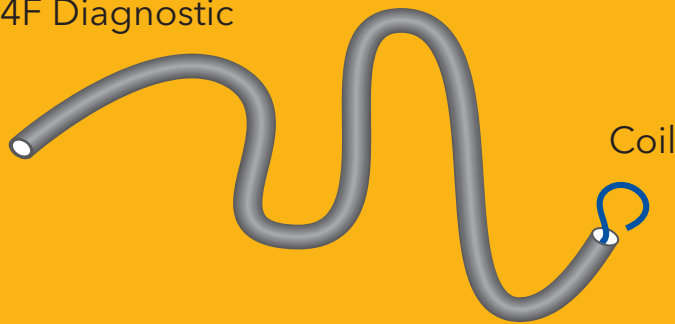

Diagnostic



<sup>†</sup> Compared to other microcatheter compatible coils when a 4F diagnostic catheter (ID ≤0.042") can be used to reach the embolization target.



# More volume

Designed with thick diameters and longer lengths to reduce the number of coils needed per procedure.<sup>‡,1</sup>


**SHAPE 1**

## Multi-sphere

Diameters 3-14 mm

**50%**

less inventory  
needed  
per diameter.<sup>§,1</sup>

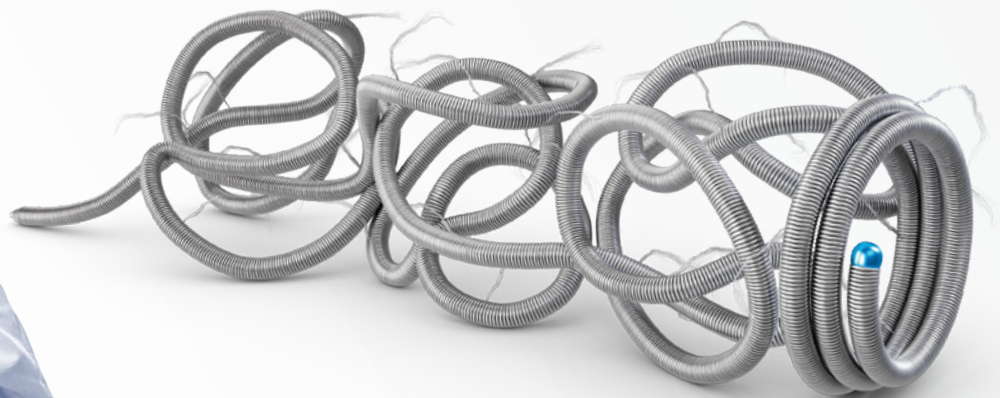

### Designed to position and pack

Oversized first spiral for positioning.

Oversized first sphere for framing.

Following spheres can:

- Pack into the first sphere
- Stack behind the first sphere

<sup>‡</sup> Compared to any other microcatheter compatible coil commercially available.  
<sup>§</sup> Compared to AZUR CX 35 and Interlock-35 for equivalent treatment options.

## SHAPE 2

# Sphere

Diameters 16-32 mm

Designed to frame and fill

Undersized first loop for easy formation.

Complex shape for framing or filling.



## More control

Combining high-volume capacity with small-coil control, Concerto Versa coil backs you up throughout your procedures:

Softness makes it trackable through the most tortuous anatomy.<sup>1</sup>

Fibers increase thrombogenicity compared to bare metal coils.<sup>2</sup>

99.7%

detachment system reliability<sup>1</sup>

**Medtronic**

Concerto Versa coil provides more volume.<sup>3</sup>

On average:

31%

more volume per coil than Ruby™\* Standard

33%

more volume per coil than Ruby Soft

204%

more volume per coil than Embold™\* Fibered

95%


more volume per coil than Interlock™\*-35

25%

more volume per coil than AZUR™\* CX 35

Fewer Concerto Versa coils are needed to fill a given volume.<sup>3</sup>

On average:

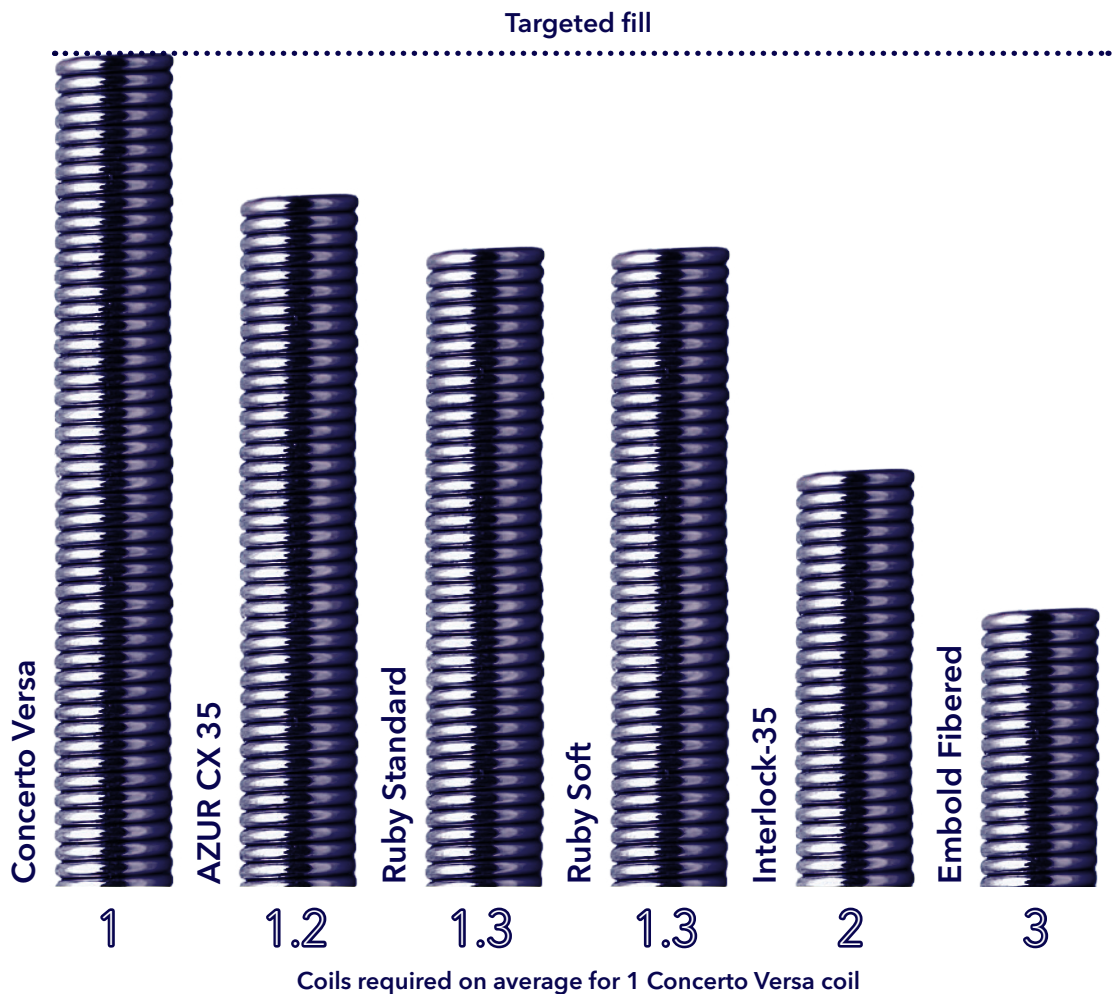

TM\* Third-party brands are trademarks of their respective owners.

## Product ordering information

CFN	Diameter (mm)	Length (cm)	Shape
CV-3-5	3	5	Multi-sphere
CV-3-15	3	15	Multi-sphere
CV-4-5	4	5	Multi-sphere
CV-4-15	4	15	Multi-sphere
CV-5-20	5	20	Multi-sphere
CV-5-30	5	30	Multi-sphere
CV-6-20	6	20	Multi-sphere
CV-6-40	6	40	Multi-sphere
CV-7-20	7	20	Multi-sphere
CV-7-40	7	40	Multi-sphere
CV-8-20	8	20	Multi-sphere
CV-8-40	8	40	Multi-sphere
CV-9-20	9	20	Multi-sphere
CV-9-40	9	40	Multi-sphere
CV-10-20	10	20	Multi-sphere
CV-10-40	10	40	Multi-sphere
CV-12-30	12	30	Multi-sphere
CV-12-50	12	50	Multi-sphere
CV-14-40	14	40	Multi-sphere
CV-14-65	14	65	Multi-sphere
CV-16-40	16	40	Sphere
CV-16-65	16	65	Sphere
CV-18-40	18	40	Sphere
CV-18-65	18	65	Sphere
CV-20-40	20	40	Sphere
CV-20-65	20	65	Sphere
CV-24-40	24	40	Sphere
CV-24-65	24	65	Sphere
CV-28-40	28	40	Sphere
CV-28-65	28	65	Sphere
CV-32-40	32	40	Sphere
CV-32-65	32	65	Sphere

### References:

1. Data on file at Medtronic D01346573\_A Design characterization report Concerto Versa 2.0. Bench and animal testing may not be indicative of clinical performance.
2. Girdhar G, Read M, Sohn J, Shah C, Shrivastava S. In-vitro thrombogenicity assessment of polymer filament modified and native platinum embolic coils. *J Neurol Sci.* April 15, 2014;339(1-2):97-101.
3. Data on file at Medtronic D01448665\_A Design characterization report Concerto Versa Coil Volume and Shelf Inventory Report.

**Indications for Use:** The Concerto Versa™ detachable coil is indicated for arterial and venous embolization in the peripheral vasculature.

**Contraindications:** There are no known contraindications.

**Potential Adverse Effects of the Device on Health:** The potential complications include, but are not limited to the following: access site complications such as hematoma, hemorrhage, pain; death; device complications such as coil stretching, friction, fracture, breakage, migration, premature detachment, non-detachment; reaction to device materials such as hypersensitivity, fever, shock, foreign body reaction; systemic complications such as infection, pain, shock, fever, inflammation, toxicity, organ failure, nerve damage, necrosis, hypotension, hematoma, hemorrhagic complications, edema; thromboembolic complications such as ischemia, infarction, occlusion, embolism; incomplete or subtherapeutic response; vascular complications such as vasoconstriction, stenosis, dissection, perforation, fistula formation, occlusion.

Indications, contraindications, warnings and instructions for use can be found in the product labeling supplied with each device, or contact a Medtronic representative.

**CAUTION:** Federal law (USA) restricts this device to sale by or on the order of a physician.

[medtronic.com/concertoversa](http://medtronic.com/concertoversa)

US-CV-2601188 v1.0 ©2026 Medtronic. All rights reserved. Medtronic and the Medtronic logo are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. For distribution in the USA only. 05/2026

