

Medtronic

PillCam™ Genius SB capsule endoscopy kit

Quick reference guide for PillCam™ Genius link device setup and capsule ingestion

1. Open the PillCam™ Genius Sync Agent





2. Input patient information (mandatory fields are marked with red asterisks) and click CONTINUE

3. Input relevant procedure information, selecting from drop down menus and click CONTINUE

4. Input the PillCam™ Genius SB kit information using the QR code, found under the front flap of the box



Scan the QR code to see the video

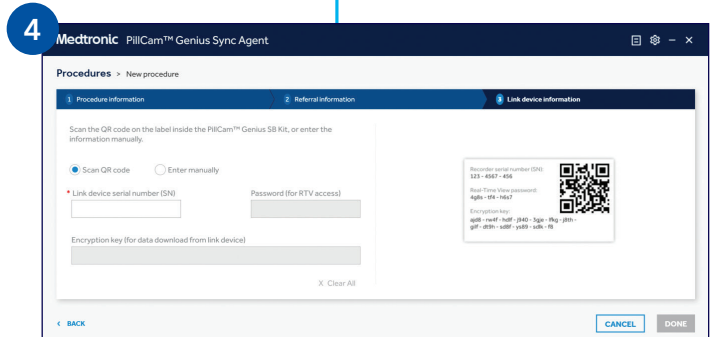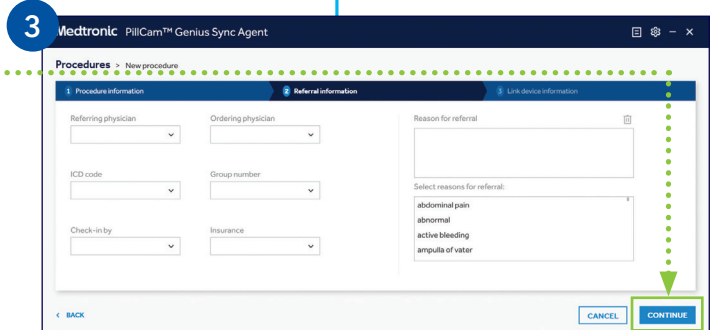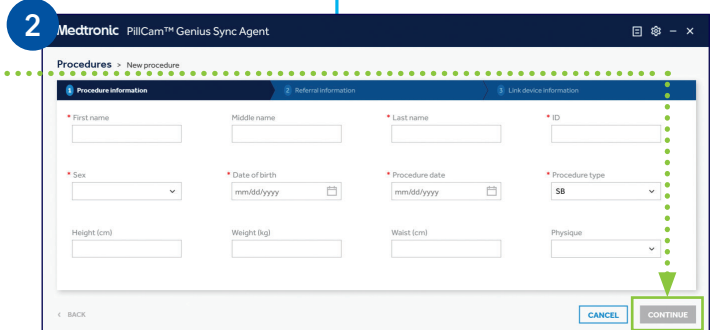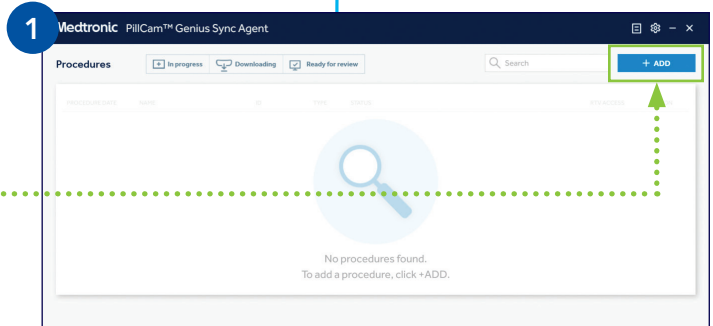

PillCam™ Genius SB capsule endoscopy kit

Quick reference guide for PillCam™ Genius link device setup and capsule ingestion


1. With the adhesive side facing you, remove tab 1 by peeling diagonally. When the link device is removed from the package, it will beep and vibrate and the LED will turn a solid green
2. While the patient is standing, ask them to expose their abdomen. Turn the link device to face the patient, align approximately 1 inch below the navel and apply to the skin
3. Hold the link device in place with the right hand while removing tab 2 by pulling at a slight diagonal. Press the adhesive against the skin as the tab is removed. Repeat on the opposite side to remove tab 3
4. Remove tab 4 on the top of the link device by pulling from the center to your left, smoothing as you remove it. Repeat on the opposite side for tab 5
5. Remove tab 6 on the bottom of the link device by pulling from the center to your left, smoothing as you remove it. Repeat on the opposite side for tab 7




PillCam™ Genius SB capsule endoscopy kit

Quick reference guide for PillCam™ Genius link device setup and capsule ingestion

6. Open the jewel case to activate the PillCam™ Genius SB capsule. The capsule LEDs will start blinking

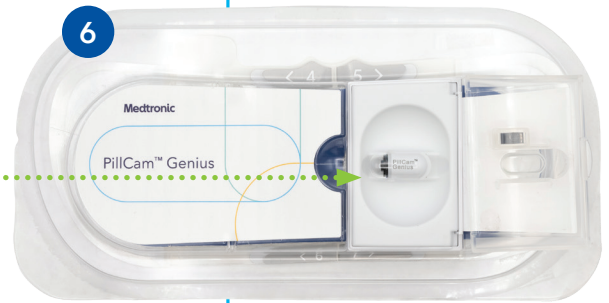

7. Once the capsule is activated, it will begin communicating with the link device. The link device will beep and vibrate and the LED will begin blinking green

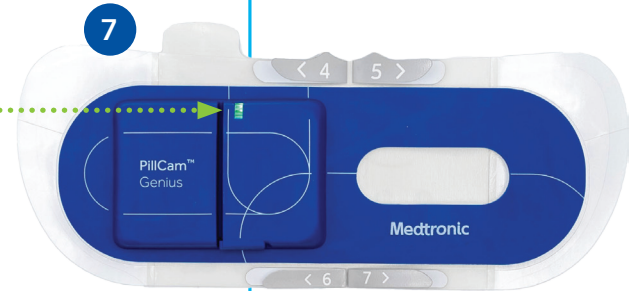

†Note that if the link device does not begin communicating with the capsule within 15 minutes from turning on, the link device will enter a non-recoverable error state

8. Ask the patient to remove the capsule from the jewel case, taking care to avoid touching the clear dome and swallow the capsule with a few sips of water



Medtronic

PillCam™ Genius SB capsule endoscopy kit

Quick reference guide for Real-time view setup

While at the workstation, open the PillCam™ Genius Sync agent on the workstation and the Real-time view (RTV) application on your iPad

1. Open the PillCam™ Genius Sync Agent (GSA) and find the row with the patient for which you will use RTV
2. Click the RTV icon, which will open a window with a QR code specific to that patient's kit





†Note that this QR code is different from the one on the box

3. From your iPad, open the RTV app and select + ADD LINK DEVICE
4. Using the camera of the iPad, scan the QR code in the GSA on the workstation monitor
5. The patient's name and ID will be added to the list in the RTV app
6. To view real-time images for a patient, open the RTV app on your iPad, find the patient's name and select CONNECT

†Note that you must be within 3 meters of the patient to view images

Names shown in images are not actual patients and are for demonstration purposes only.

US-DG-2400132 ©2024 Medtronic. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

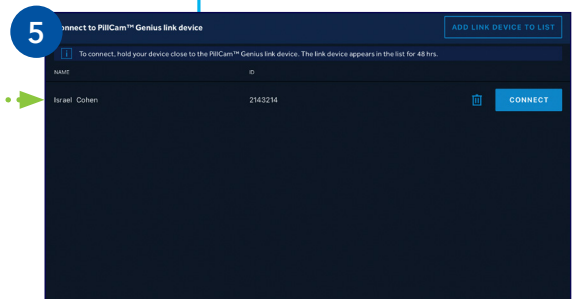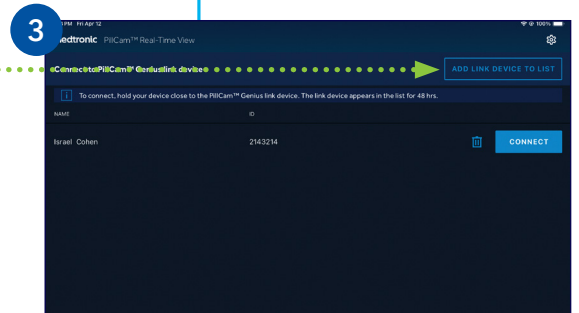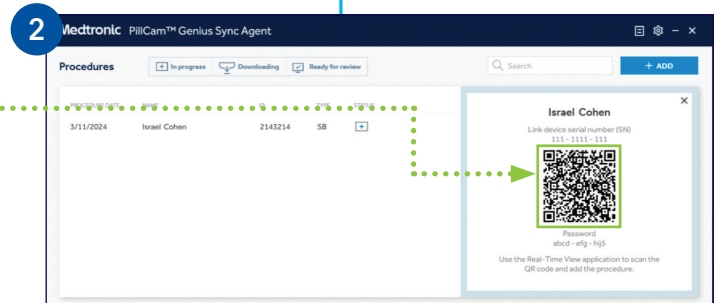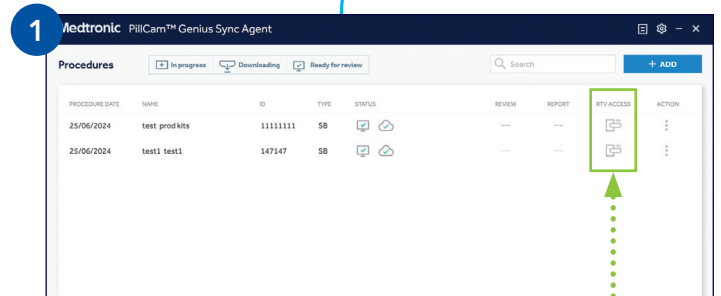

Medtronic

PillCam™ Genius SB capsule endoscopy kit




Quick reference guide for procedure data download

After the PillCam™ Genius link device is returned to the clinic, data is downloaded for compilation into a video for review

1. Remove the PillCam™ Genius link device from the envelope and anti-static bag. The device can remain folded to avoid exposing the adhesive
2. Expose the data download port by tearing the cover from left to right
3. Using a USB-C cord, plug the link device into your workstation
4. Data download and video creation will begin automatically. Progress can be seen in the PillCam™ Genius Sync Agent screen

5. When the data download is complete and the link device is ready to be disconnected, the blue LED will remain a **constant blue**
6. Once the procedure status is "Ready for Review" in the GSA, the files can be viewed locally using PillCam™ desktop software v9.7
7. If PillCam™ Cloud Reader is available, the procedure is automatically synchronized to the Cloud and can be read via that interface as well

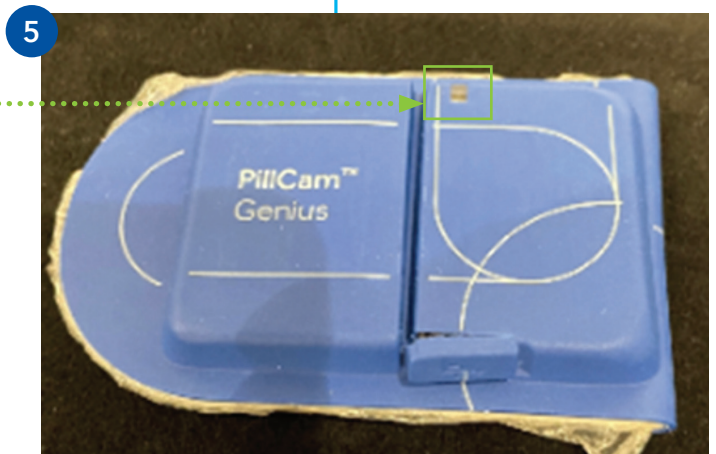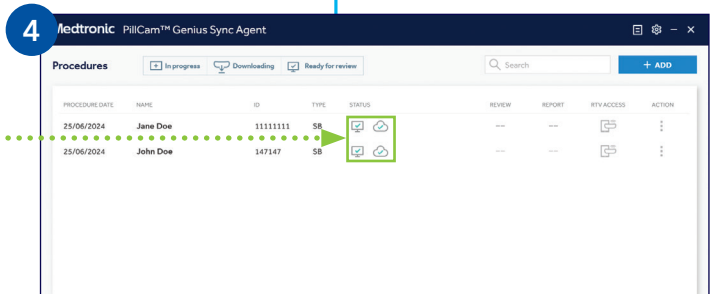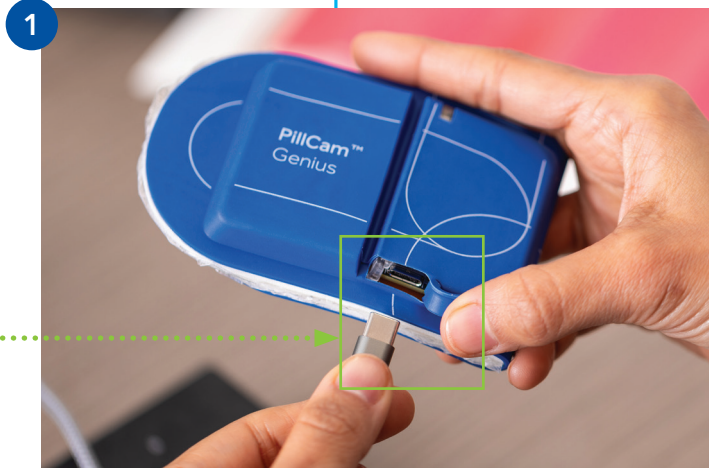


Names shown in images are not actual patients and are for demonstration purposes only.










PillCam™ Genius Sync Agent

Quick reference guide for software icons

The icon in the Windows notification area indicates the status of the PillCam™ Genius Sync Agent:

GSA tray icon	Description
	System functioning
	Disconnected from cloud (for PillCam™ Cloud Reader software users)
	Error in one or more procedures

The **Status** column displays the status of each procedure, as indicated by the following:

Status icon	Description
	In progress: from creating a new procedure in the PillCam™ Genius Sync Agent and during the procedure, until connecting the PillCam™ Genius link device to the workstation
	Downloading: during data transfer from the PillCam™ Genius link device to the workstation and during video creation
	Ready for local review: video is ready for review in PillCam™ desktop software
	Cloud sync: during video upload from the local archive to the cloud (PillCam™ Cloud Reader software users only)
	Ready for cloud review: procedure data synced successfully to the cloud from the local archive; procedure data can be reviewed in the cloud (PillCam™ Cloud Reader software users only)
	Downloading error: there is a problem with either downloading the data from the PillCam™ Genius link device or creating a video
	Cloud sync error: there is a problem uploading the video to the cloud (PillCam™ Cloud Reader software users only)