


# STEP-BY-STEP GUIDE TO HERNIA REPAIR


## Spacemaker™ Pro Access and Dissector System

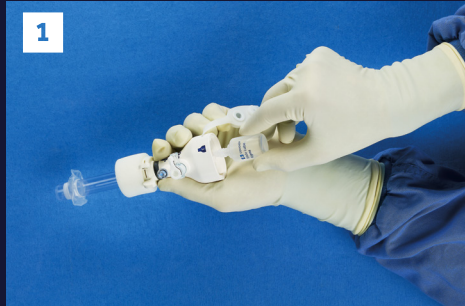


1 Apply Endo-Lube™ Solution



2 Insert Dissector†

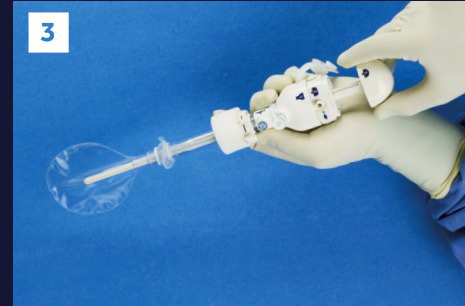

3 Remove Obturator

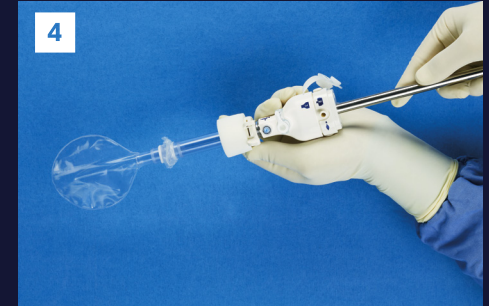


4 Insert Endoscope



5 Inflate Balloon Dissector



6 Remove Endoscope

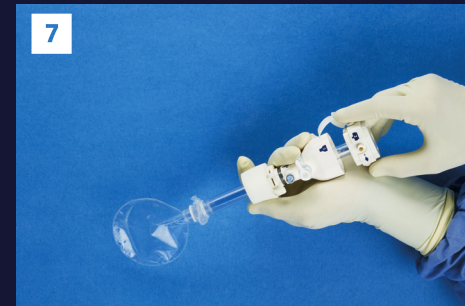

7 Release Access Cannula



8 Inflate Access Balloon



9 Advance Foam Collar



10 Remove Balloon Dissector



11 Connect CO<sub>2</sub> Hose



12 Re-Insert Endoscope

Always refer to Instructions for Use for detailed step-by-step instructions, indications, contraindications, intended use, warnings and precautions

† When using the accessory access cannula obturator, do not assemble the Spacemaker™ Pro system.

© 2021 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic.  
All other brands are trademarks of a Medtronic company. 01/2021 – US-HR-2000110 – [WF # 1277904]

**Medtronic**  
Further, Together