



THE CONFIDENCE TO KNOW BLEEDING WILL STOP.

Veriset™ Hemostatic Patch

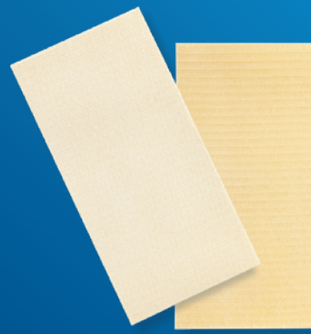


LAPAROSCOPIC SURGERY

Preparing Veriset™ Hemostatic Patch

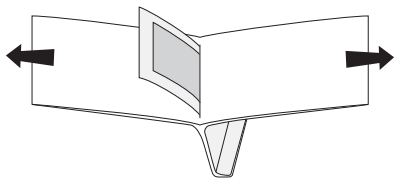


1. Open the foil pouch outside the sterile field.

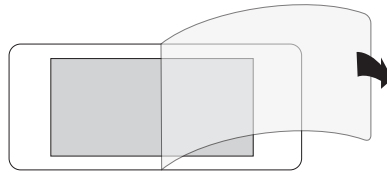


2. Open the Tyvek™ pouch within the sterile field. Use the Veriset™ patch within 60 minutes after opening the foil pouch. Veriset™ patch should be placed on the bleeding site, extending ~1–2 cm beyond the site. Veriset™ patch may be cut as necessary.


Applying Veriset™ Hemostatic Patch

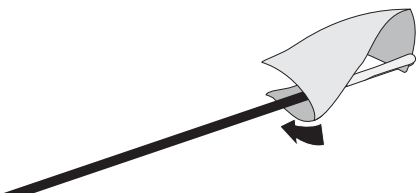

3. Grasp the Veriset™ patch on the edge with the grasper. Roll the patch around the grasper, with the darker side surface on the inside. For a rectangular patch, roll the patch along the long axis to facilitate entry into the trocar cannula.

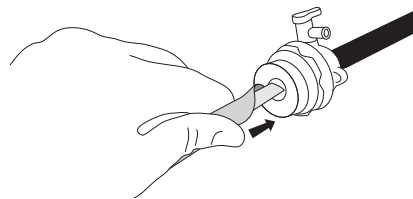


4. Support the rolled Veriset™ patch at the leading edge and insert through the trocar of appropriate size.

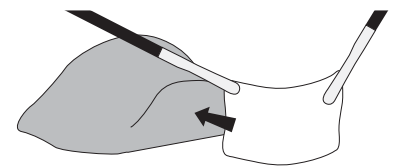


5. Identify the darker side of the Veriset™ patch. Place the darker side on the bleeding site.

Nonhydrated portions of the Veriset™ patch should be adhered to the surface by hydrating with saline and applying pressure.

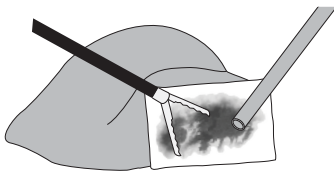


6. Hold the Veriset™ patch firmly in place by providing uniform pressure over the entire patch surface to ensure proper adherence. After 30 seconds, discontinue pressure and visually inspect the bleeding site for haemostasis. If bleeding has not stopped, reapply pressure and reassess after 30 seconds.

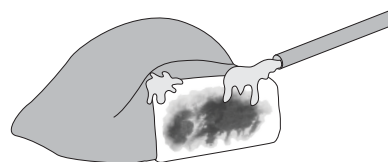

7. Do not remove Veriset™ patch at the end of surgery. Veriset™ patch absorbs after ~28 days.¹


THE CONFIDENCE TO KNOW BLEEDING WILL STOP.

Veriset™ Hemostatic Patch

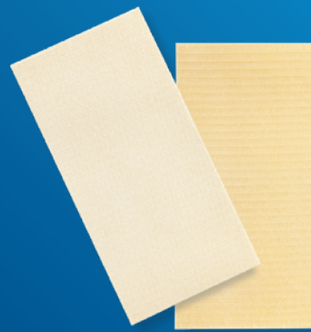


OPEN SURGERY

Preparing Veriset™ Hemostatic Patch

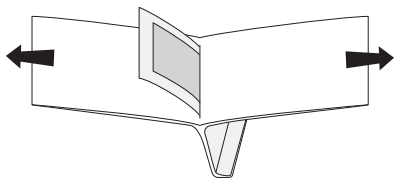


1. Open the foil pouch outside the sterile field.

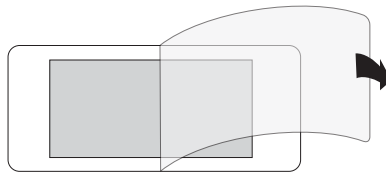

2. Open the Tyvek™ pouch within the sterile field. Use the Veriset™ patch within 60 minutes after opening the foil pouch. Veriset™ patch should be placed on the bleeding site, extending ~1–2 cm beyond the site. Veriset™ patch may be cut as necessary.

Applying Veriset™ Hemostatic Patch

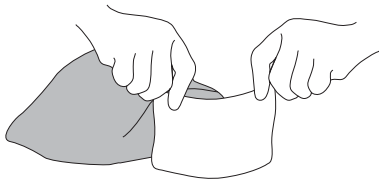

3. Place the darker side of Veriset™ patch on the bleeding site.

Using a syringe filled with saline, it is essential to hydrate all non-hydrated portions of Veriset™ patch. Apply pressure for 30 seconds to the newly hydrated portions.



4. Hold Veriset™ patch firmly in place. Apply uniform pressure to ensure adherence to the bleeding site. A swab may be used. After 30 seconds, discontinue pressure and inspect the bleeding site.

Do not move or peel away Veriset™ patch; it may not re-adhere to the bleeding site. If bleeding has not stopped, reapply pressure and reassess after 30 seconds.



5. Do not remove Veriset™ patch at the end of surgery. Veriset™ patch absorbs after ~28 days.¹

1. Clinical Study Report COVEULV0032, A Prospective, Multi-Center, Randomized, Single-Blind Study to Compare the Hemostatic Patch to Fibrin Sealant (TachoSil(r)) in Subjects undergoing Hepatic Surgery, appendix B

Medtronic

IMPORTANT: Please refer to the package insert for complete instructions, contraindications, warnings and precautions.

© 2021 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic.

™* Third party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company.

20-weu-veriset-or-guide-lap-open-combined-4831990

[medtronic.com/covidien/uk](https://www.medtronic.com/covidien/uk)