

# Medtronic

Engineering the extraordinary

See your Model WR9200 for Activa™ RC devices recharger user manual and your Model TH90 for Activa™ devices patient programmer manual for complete instructions and information on warnings, precautions, risks, and benefits. Rx only.



*Devices shown for illustrative purposes.*

## DBS Therapy Wireless Recharger


Model WR9200 for Activa™ RC devices

### Quick Guide

### Charge your recharger



- 1 Connect the USB cable to the dock.
- 2 Plug the USB cable into the power adapter and plug the adapter into a wall outlet. The dock must rest on a flat surface while plugged in.
- 3 Place the recharger, button-side up, into the dock. Leave your recharger on the dock between recharging sessions to maintain full charge.  
**PLEASE NOTE:** If you are charging your recharger for the first time, it is important to complete the charge without removing the charger from the dock to properly take it out of shipping mode.

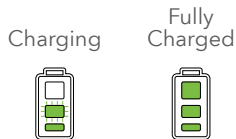

- 4 When your recharger is fully charged the battery icon will display as shown in the figure.

### Pairing the recharger to the app\*

\*recharger app is optional and not required to charge implant

- 1 Turn on and unlock your handset
- 2 Open the recharger app by tapping the app icon (📱)
- 3 Turn on your recharger.
- 4 In the recharger app, tap **Connect**.




- 5 Tap **Scan Code**.  
**Note:** Tap "Allow" on the pop-up requesting access to the camera, if it occurs.

- 6 Aim the handset camera at the scannable code on the recharger, keeping the entire code visible on the handset screen, until you receive confirmation on the handset that pairing is complete.

[See other side for additional instructions >](#)

# Charging your implant

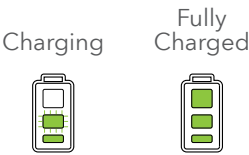


- 1 Check the battery level of the recharger, and charge your recharger, if necessary.  
See “Charging your recharger” on the other side for instructions.
- 2 Place the recharger into your drape or belt and position the two concentric circles on the drape and recharger over your neurostimulator site.  
**Note:** Use the counterweight when using the drape to balance the recharger
- 3 Turn on the recharger. The recharger will beep until it connects to your neurostimulator.



## Your Recharger Indicators

Tone	Repeating beeping tone	2 tones, rising in pitch	3 tones at the same pitch	None	A series of tones, rising in pitch	2 tones, falling in pitch and repeating	2 tones, falling in pitch and repeating
Light	Spinning	Solid	Solid	Slow Pulsing	Solid	Flashing	Flashing
Meaning	Recharger is searching for the neurostimulator.	Neurostimulator has been found. <b>Note:</b> If you hear 3 additional tones, see Elective Replacement Indicator in the following row.	Elective Replacement Indicator See your patient programmer or patient therapy app manual for more information.	Recharging neurostimulator.	Recharging session is complete.	Neurostimulator not found <b>or</b> Other alert/error Check the app for more information.	Recharger battery is low.

## Using the optional Recharger App

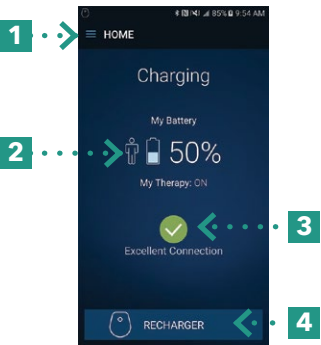

- 1 **Menu** button (≡) – Tap to access charging speed settings and app information. The recharger defaults to fastest speed.
- 2 **Neurostimulator Battery** icon – Your neurostimulator battery status. The number next to this image indicates how full the neurostimulator battery is.
- 3 **Connection** icon – Your connection status between your recharger and your neurostimulator.
- 4 **Recharger** button – Tap to view the recharger battery status and change the volume of your recharger.

Medtronic

710 Medtronic Parkway  
Minneapolis, MN 55432-5604  
USA  
Tel: 763-514-4000  
medtronic.com

©2022 Medtronic. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. UC202010093c EN