

Downloading session reports with the CT900 programming tablet




For the patient's first and last names to appear on in the name of the PDF file which will be exported, the following information must be provided in the **PATIENT** tab during the programming of the pump: patient's **FIRST NAME**, patient's **LAST NAME** and **DATE OF BIRTH**.

Step 1 - on the tablet:

- 1 Enter the **Patient Data Service** application.




- 2 From the **PATIENT** tab, select the last name of the patient whose data you want to export.

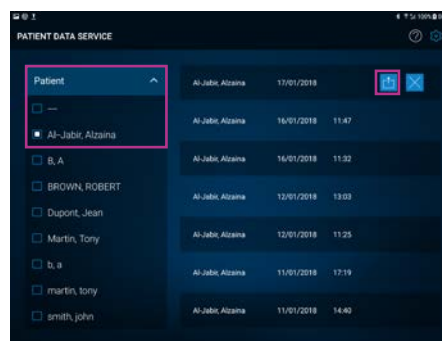

Note: If the tablet is used for different therapies (PUMPS, DBS, INTELLIS), click on **Therapy** to select the desired therapy.


All the pump test reports created with this tablet will appear.

- 3 Select the desired report (date and time) and click on
- 4 Select the type of report you want to export:

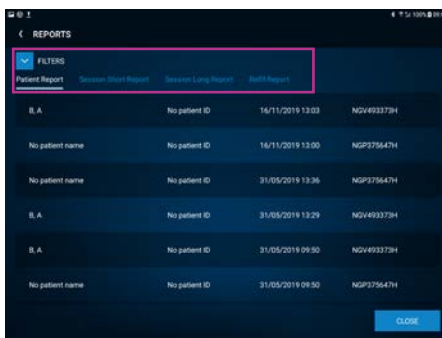


- **Patient report:** indicate the current pump settings (reservoir settings, medication and type of infusion), as well as the deadline for the next filling.
- **Session report long:** contains more detailed information from the short session report, and indicates the history of the diaries if they have been consulted during the session.
- **Refill report:** provides a list of all patients who have a patient record on the programme associated with an implantable pump.


- 5 Click on .

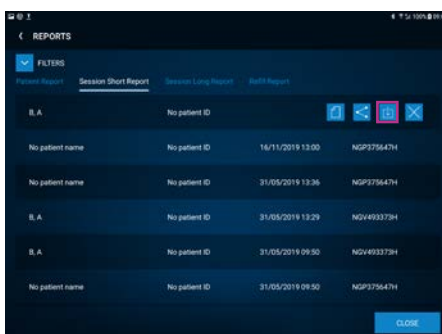

- 6 It is possible to delete patient identifying information. **CONFIRM**.

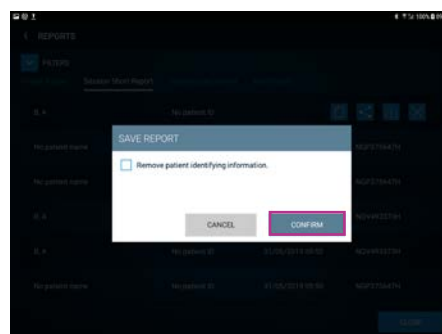

- 7 The screen below will appear and indicate the location where the PDF file has been saved.

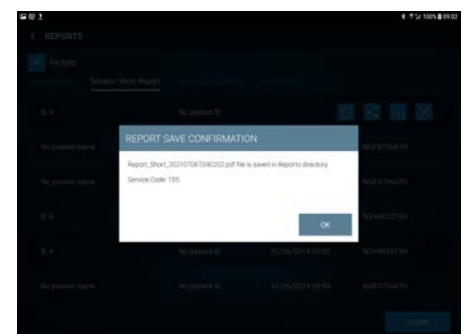

Step 2 - on the computer:




Use the tablet recharging cable to connect the tablet to a computer.


- 1 Select the tablet connected to the computer.
- 2 Click on the **REPORTS** file, gathering all downloaded reports.
- 3 Copy/cut and paste the reports in the desired location on your computer.




The PDF reports downloaded on the tablet are automatically stored.

It is possible to parameterize the automatic deletion of the reports in the Reports file after 1 to 180 days. To do so:

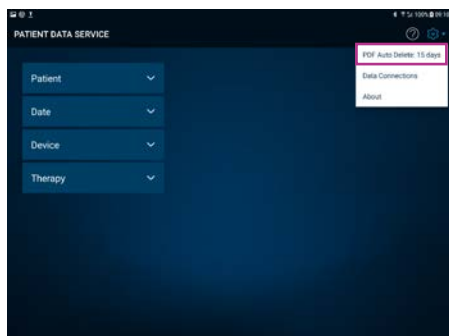

- Go to 
- Click on **PDF AUTO DELETE**
- Select **ACTIVATE AUTO DELETE** and select the **NUMBER OF DAYS**
- Click on **UPDATE**

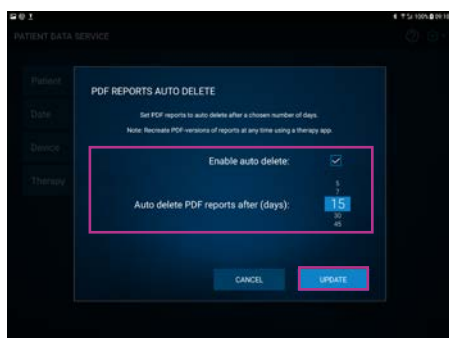

The PDF auto delete feature does not delete patient data records from the Patient Data Service app.

Medtronic

Europe
Medtronic International Trading Sàrl.
Route du Molliau 31
Case postale
CH-1131 Tolochenaz
www.medtronic.eu
Tel: +41 (0)21 802 70 00
Fax: +41 (0)21 802 79 00

United Kingdom/Ireland
Medtronic Limited
Building 9
Croxley Park
Hatters Lane
Watford
Herts WD18 8WW
www.medtronic.co.uk
Tel: +44 (0)1923 212213
Fax: +44 (0)1923 241004

See the device manual for detailed information regarding the instructions for use, the implant procedure, indications, contraindications, warnings, precautions, and potential adverse events. For further information, contact your local Medtronic representative and/or consult the Medtronic website at www.medtronic.eu. For applicable products, consult instructions for use on www.medtronic.com/manuals. Manuals can be viewed using a current version of any major internet browser. For best results, use Adobe Acrobat® Reader with the browser.

UC202014541aEE © Medtronic 2023.
All rights reserved.