# new paradigm in electrophysiology

Affera™ Mapping and Ablation System

Medtronic Engineering the extraordinary

# All-in-one – map | ablate | validate

## Sphere-9<sup>™</sup> Catheter

#### Unparalleled versatility

- HD mapping leveraging closeunipolar EGMs<sup>1</sup>
- One, single catheter with dualenergy<sup>2</sup> capabilities, RF and PF

#### Absolute control

- Tissue contact and stability through innovative 9 mm, conformable lattice tip
- 9 mini electrodes providing accurate, real-time tissue temperature monitoring<sup>1</sup>

mapping minutes


Average time
to complete preablation map<sup>2</sup>



0.6%
device-related
primary adverse
events
Using RF or
PF energy<sup>4</sup>

#### **Optimized workflow**

- Safe<sup>1,2</sup> and effective<sup>1,2</sup> procedures in less time<sup>2</sup> using fewer wide-area focal lesions<sup>1,3</sup>
- Single transseptal access and zero exchanges for optimized workflow

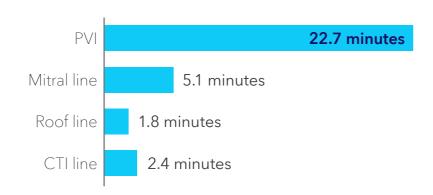

Transpired ablation time (n = 76, PF/RF)<sup>2</sup>

### Sphere-9 Catheter at a glance

- 1. Nitinol lattice ablation electrode
- 2. Nine platinum-iridium mini electrodes
- 3. Central reference electrode
- 4. Central irrigation
- 5. 8Fr Bi-directional steerability (D-F)
- 6. Handle controller for mapping management
- 7. Magnetic location sensors for curve visualization

# Elevating the norm

## Affera™ Prism-1 Mapping Software

Built with user experience in mind, the Affera Prism-1 mapping software addresses the most common needs of today's electrophysiologists with elegant features and an innovative suite of software functionalities to enhance clinical workflows, now and in the future.

#### Intuitive interface


A modern, sleek interface designed to enhance and streamline the interpretation of complex information

#### Integrated platform

Purposefully built for an optimized Sphere-9 user experience and forthcoming developments and catheter technologies

#### Insightful algorithms


Innovative clinical tools to enhance performance and remove complexity

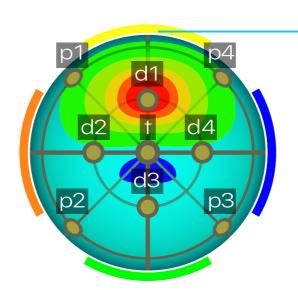


**Sensor globe** aggregates real time clinical information

- Tissue temperature
- Local impedance for contact verification

Precise delineation of complex anatomical regions and ridges with **vein mode** 

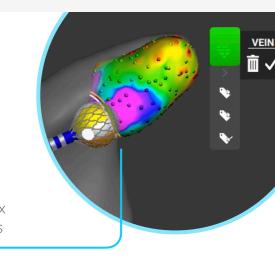


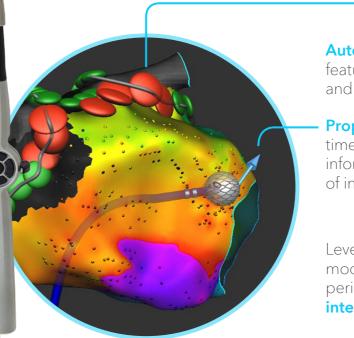

AutoTags and TagConnect™ features highlight visual contiguity and orientation of ablation tags

**Propagation arrow** provides real time spatiotemporal activation information regarding the area of interest

Leverage custom pacing modes and configure pacing period and amplitude using integrated stimulator

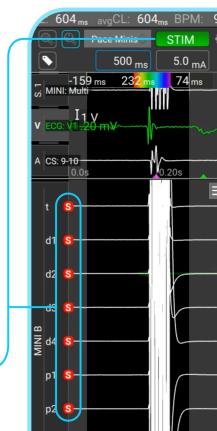

#### Affera Mapping and Ablation System ordering details

Capital Components		
Product Description	Details	
Affera Prism-1 Mapping System	Includes Software, CIU, Window Field Generator, Workstation (PC mouse, keyboard), Bard and Cardiolab™ interface accessories, cables/accessories	
HexaGen™ RF Generator	Includes RFG, remote control, foot pedal, cables/accessories	
HexaFlow™ Irrigation Pump	Includes pump, cables/accessories	
HexaPulse™ PF Generator	Includes PFG, cables/accessories	

**Note:** Capital systems are customized to support the individual needs and country requirements of each customer. Please work directly with your local sales representative to identify the correct order code.

Disposable Components		
Ordering Number	Product Description	Details
AFR-00001	Sphere-9 Catheter	Single use, HD mapping and ablation catheter
AFR-00007	Location Reference Patch Kit	Includes 4 procedure kits
AFR-00006	Catheter Extension Cable	Single use cable for use with Sphere-9 catheter
AFR-00002	Tubing Kit	Includes 4 procedure kits, compatible with HexaFlow irrigation pump

#### References:

- <sup>1</sup> Anter E, Neužil P, Rackauskas G, et al. A Lattice-Tip Temperature-Controlled Radiofrequency Ablation Catheter for Wide Thermal Lesions: First-in-Human Experience With Atrial Fibrillation. *JACC Clin Electrophysiol*. May 2020;6(5):507-519.
- <sup>2</sup> Reddy VY, Anter E, Rackauskas G, et al. Lattice-Tip Focal Ablation Catheter That Toggles Between Radiofrequency and Pulsed Field Energy to Treat Atrial Fibrillation: A First-in-Human Trial. *Circ Arrhythm Electrophysiol*. June 2020;13(6):e008718.
- <sup>3</sup> Reddy VY, Peichl P, Anter E, et al. B-PO01-055 ATRIAL FIBRILLATION ABLATION USING A FOCAL LATTICE-TIP CATHETER THAT TOGGLES BETWEEN PULSED FIELD AND RADIOFREQUENCY ENERGY: EFFECT ON THE ESOPHAGUS. Heart Rhythm. August 2021;18(8):S73.
- <sup>4</sup> CR-00014 Rev C. Medtronic data on file.

#### Note:

Formal product names in IFUs and user manuals may differ: Sphere- $9^{\text{TM}}$  Catheter = Sphere- $9^{\text{TM}}$  Mapping and Ablation Catheter Tubing Kit = HexaFlow<sup>TM</sup> Tubing Set

#### **Brief statement**

See the device manual for information regarding the instructions for use, indications, contraindications, warnings, precautions, and potential adverse events. For further information, contact your local Medtronic representative and/or consult the Medtronic website at medtronic.eu.

©2023 Medtronic. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. ™Third-party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company.

#### **Medtronic**

Europe

Medtronic International Trading Sàrl. Route du Molliau 31 Case postale CH-1131 Tolochenaz medtronic.eu Tel: +41 0 21 802 70 00

Fax: +41 0 21 802 79 00

United Kingdom/Ireland

Medtronic Limited Building 9 Croxley Green Business Park Hatters Lane Watford Herts WD18 8WW

medtronic.co.uk Tel: +44 0 1923 212213 Fax: +44 0 1923 241004 **Technical Support** 

Toll-free: 00800-22556338 Toll (paid): +31 (45) 5664621

rs.casemeatechservice@medtronic.com

UC202310658-Affera-customer-presentationen-we-8686723 ©2023 Medtronic. All Rights Reserved. Printed in Europe.

