


Barrx™ radiofrequency ablation

Condition	Catheter	Ablation/ Cleaning protocol	Energy setting
Barrett's esophagus	Barrx™ 360 express RFA balloon catheter	1 – clean – 1	10 J/cm ²
	Barrx™ channel RFA endoscopic catheter	2 – clean – 2	12 J/cm ² (NDBE & Dysplasia)
	Barrx™ 60 RFA focal catheter	2 – clean – 2	
	Barrx™ 90 RFA focal catheter	2 – clean – 2	
	Barrx™ ultra long RFA focal catheter	1 – clean – 1	
GAVE	Barrx™ channel RFA	1–4 (no cleaning)	12 J/cm ²
	Endoscopic catheter		
	Barrx™ 60 RFA focal catheter		
	Barrx™ 90 RFA focal catheter		
	Barrx™ ultra long RFA focal catheter		
Radiation proctitis	Barrx™ channel RFA endoscopic catheter	1–2 (no cleaning)	12 J/cm ²
	Barrx™ 60 RFA focal catheter		
	Barrx™ 90 RFA focal catheter		

No saline or lubrication

Items needed:

- 4 × 4s
- Mucomyst
- Bite block
- Sterile water alcohol pad (Ultra long)

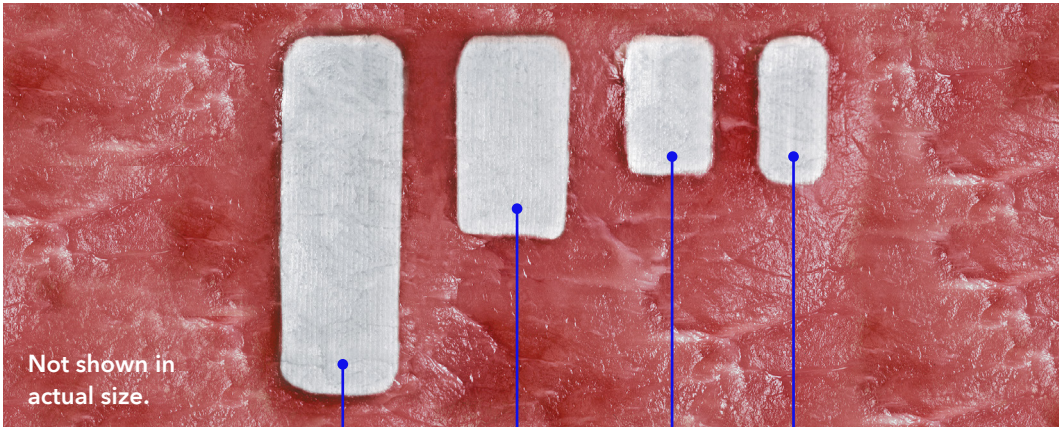

Treatment area footprint

- Barrx™
ultra long
RFA focal
catheter
- Barrx™
90 RFA
focal
catheter
- Barrx™
60 RFA
focal
catheter
- Barrx™
channel RFA
endoscopic
catheter

Catheter	Product code	Specifications	
Barrx™ 360 express RFA balloon catheter 	64082	Balloon length: 8 cm Electrode length: 4 cm Catheter shaft diameter: 7 mm Catheter shaft working length: 85 cm Maximum no. of ablations: 16	
Barrx™ channel RFA endoscopic catheter 	TTS-1100	Recommended scope sizes: endoscopes with working channel diameter of 2.8 mm or larger	Electrode: 15.7 mm length – 7.5 mm width (when “wings” are unfolded) Catheter shaft diameter: 2.5 mm Catheter shaft length: 135 cm Maximum no. of ablations: 120
Barrx™ 60 RFA focal catheter 	90-9300	Recommended scope sizes: 8.6 mm to 9.8 mm	Electrode: 15 mm length – 10 mm width Catheter shaft diameter: 4 mm Catheter shaft length: 160 cm Maximum no. of ablations: 80
Barrx™ 90 RFA focal catheter 	90-9100	Recommended scope sizes: 8.6 mm to 12.8 mm	Electrode: 20 mm length – 13 mm width Catheter shaft diameter: 4 mm Catheter shaft length: 160 cm Maximum no. of ablations: 80
Barrx™ ultra long RFA focal catheter 	90-9200	Recommended scope sizes: 8.6 mm to 9.8 mm	Electrode: 40 mm length – 13 mm width Catheter shaft diameter: 4 mm Catheter shaft length: 160 cm Maximum no. of ablations: 80



Questions? Call or email your Medtronic representative.

For full details, please refer to the product's *Instructions For Use* (IFU).

Risk Information:

The following are transient side effects that may be expected after treatment: chest pain, difficulty swallowing, painful swallowing, throat pain, and/or fever. Complications observed at very low frequency include: mucosal laceration, minor and major acute bleeding, esophageal stricture, perforation, cardiac arrhythmia, pleural effusion, aspiration, and infection. Potential complications that have not been observed include: death. Please refer to the product user manual for additional detailed information.

Important: Please refer to the package insert for complete instructions, contraindications, warnings and precautions.

© 2022 Medtronic. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. EMEA-SE-2200003 - 22-weu-barrx-rfa-procedural-quick-guide-6300075