

Medtronic

Neuromodulation

MRI safety status

December 2023



- * Under specific conditions of use.
Refer to approved labelling.
- ** Pump resumes programmed therapy after MRI scan; requires interrogation to confirm pump status.
- *** SureScan® MRI Systems that do not meet the Full Body Eligibility requirements may still be Head Scan eligible.
- † No B1+RMS limit for Transmit/Receive Lower Extremity Coil.
- †† No MRI if 3023 neurostimulator serial number is:
 - Less than NBV132955H
 - Between NBV133037H and NBV133063H
 - Between NBV628045S and NBV628263S

Brief Statement:

See the device manual for detailed information regarding the instructions for use, the implant procedure, indications, contraindications, warnings, precautions, and potential adverse events. If using an MRI SureScan™ device, see the MRI SureScan™ technical manual before performing an MRI. For further information, contact your local Medtronic representative and/or consult the Medtronic website at www.medtronic.com.

Consult instructions for use at this website. Manuals can be viewed using a current version of any major Internet browser. For best results, use Adobe Acrobat Reader™* with the browser.

Medtronic

Europe

Medtronic International Trading Sàrl.
Route du Molliau 31
Case postale
CH-1131 Tolochenaz
www.medtronic.eu
Tel: +41 (0)21 802 70 00
Fax: +41 (0)21 802 79 00

United Kingdom/Ireland

Medtronic Limited
Building 9
Croxley Park
Hatters Lane
Watford
Herts WD18 8WW
www.medtronic.co.uk
Tel: +44 (0)1923 212213
Fax: +44 (0)1923 241004

UC201102607n-neuro-mri-safety-status-chart-en-emea-11249321 © Medtronic 2024.
All rights reserved. Medtronic and Medtronic logo are trademarks of Medtronic.™*
Third-party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company.