


Medtronic

Adult application guide


Nellcor™ OxySoft™ pulse oximetry sensor




To apply the Nellcor™ OxySoft™ pulse oximetry sensor on adult patients

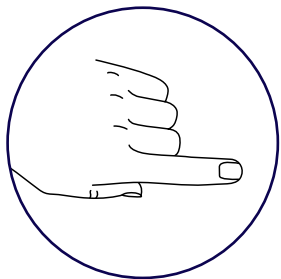


1 Select sensor application site. Choose a location away from blood pressure cuff or infusion lines. The preferred sensor site is an index finger. Alternatively, other fingers may be used.

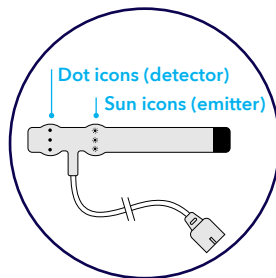

2 Clean and remove any substances such as nail polish from the sensor application site. Remove the plastic backing from the sensor and locate the optical components beneath the sun (☀) and dot (●) icons.

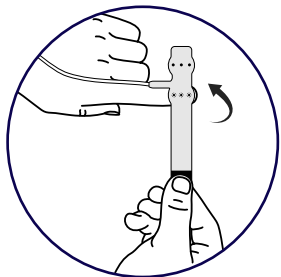

3 Place the sun icons on the nail bed (do not place on a joint) and wrap the sensor around the finger so the dot icons oppose the sun icons. Note that the cable will be positioned on the left side of the selected finger.

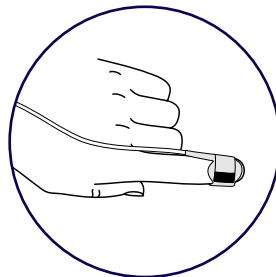

4 Wrap the length of the bandage firmly around the sensor application site, but not too tightly. Plug the sensor into the oximeter and verify proper operation as described in the oximeter operator's manual.



Read the sensor's "Directions for Use" insert for complete instructions for use, including all warnings, cautions, and specifications, prior to use of this product.

The Nellcor™ pulse oximetry monitoring system should not be used as the sole basis for diagnosis or therapy and is intended only as an adjunct in patient assessment.

© 2022 Medtronic. EMEA-PM-2200070-nellcor-oxysoft-adult-positioning-guide-en-we-7505293

[medtronic.co.uk/pmri](https://www.medtronic.co.uk/pmri)



Questions? Call or email your local Medtronic representative.

Medtronic