


The LigaSure™ curved jaw open sealer/divider

LF3225/LF3225c

Assembly instructions

Step 1

Make sure the LigaSure™ instrument is not connected to the generator.

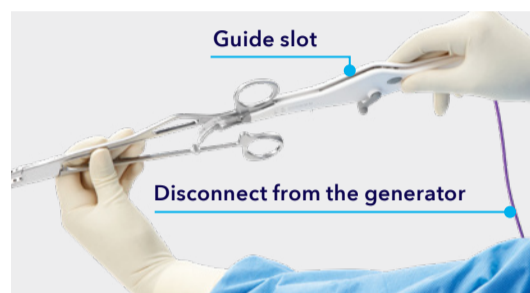

Step 2

Insert the clamp tail into the electrode guide slot.



Step 3

Push the electrode along the clamp until it reaches a hard stop and clicks into place.

Recommendation

Be sure you hear the click from the release button and that the electrode is firmly fixed to the clamp.

Step 4


Align the four pins on the electrode plate with the four holes in the clamp jaws, and push firmly into place.

Recommendation

For each electrode plate, use your finger to push the two pins evenly, parallel to the clamp jaw.

Precaution

Use care with the electrode so the pins do not break or bend. If the pins are bent or broken, the instrument will not function correctly and the electrode should be discarded.

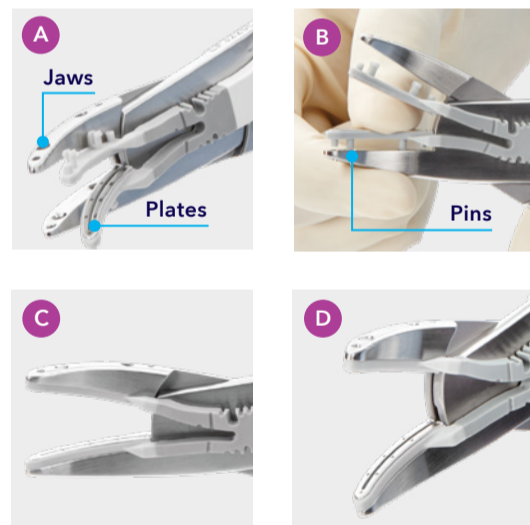


Step 5

Gently close the clamp on a folded four-inch by four-inch gauze pad until you reach the second click from the Inline Activation Button.




Step 6

Remove the gauze pad and visually inspect the jaws to ensure that there is no space between the clamp jaws and the gray jaw electrode plates.

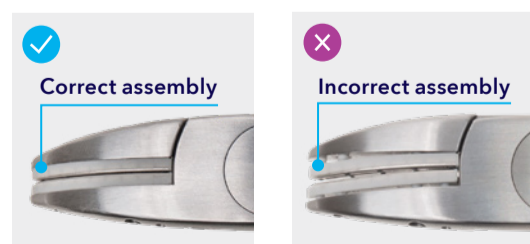

Important features

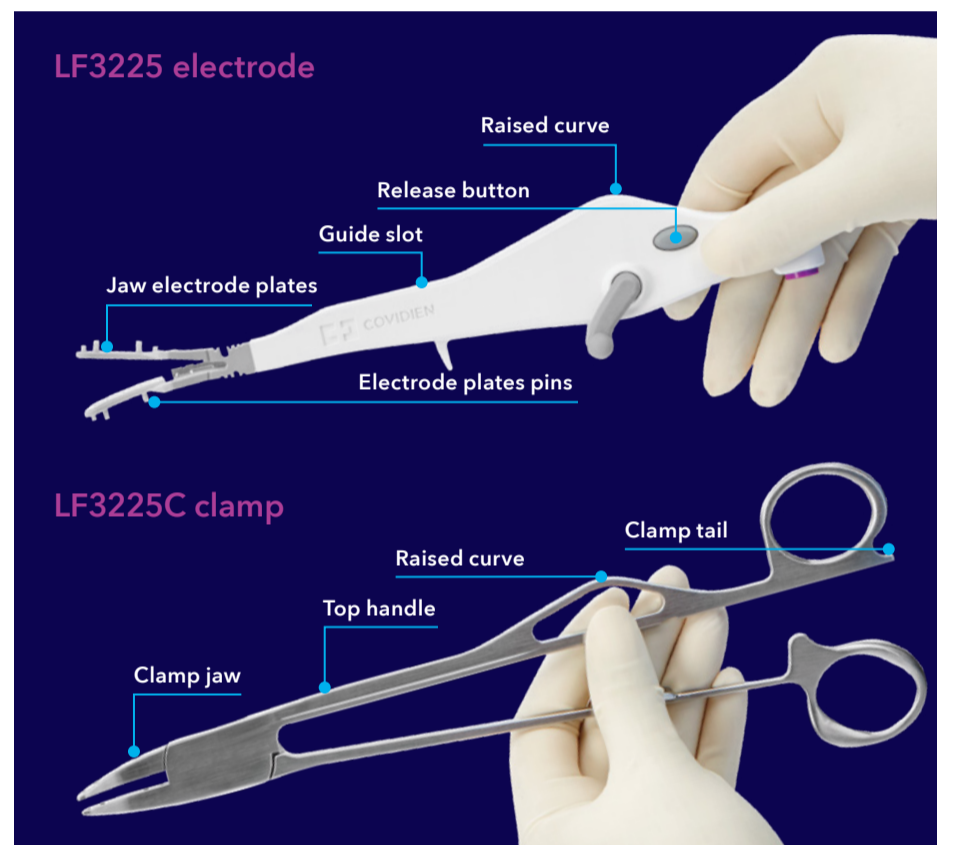

Disassembly instructions

Step 1


Unplug the LigaSure™ instrument from the generator.

Step 2

Unsnap the electrodes plates from the clamp jaws.

Recommendation

Push the outer dark gray area with your finger.

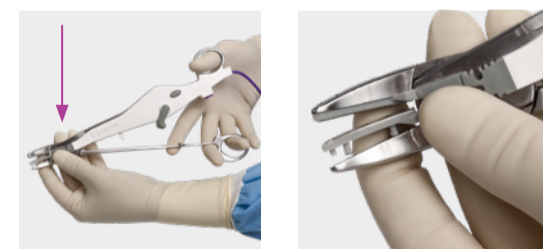

Step 3

From both sides of the clamp, press and firmly hold the release button while pulling the electrode away from the clamp.



Step 4

Dispose of the electrode. Do not resterilize it.